

“LEADER IN ANIMAL CARE PRODUCTS FOR OVER 60 YEARS”



SSCI

SUBURBAN SURGICAL Co., Inc.

275 Twelfth Street, Wheeling, IL 60090

All intellectual material, including the text, graphics and other components of this catalog are Copyrighted by SSCI.
Any use or duplication of these intellectual materials, without written consent by SSCI, is strictly forbidden.


For Product Information Call TOLL-FREE 1-800-323-7366 • (847) 537-9320 • Fax (847) 537-9061

Hours: 8:30am - 5:00pm CST Monday - Friday

Visit our website at: www.suburbansurgical.com

STAINLESS STEEL CAGES

REGAL: *SIMPLY THE BEST CAGES YOU CAN BUY*

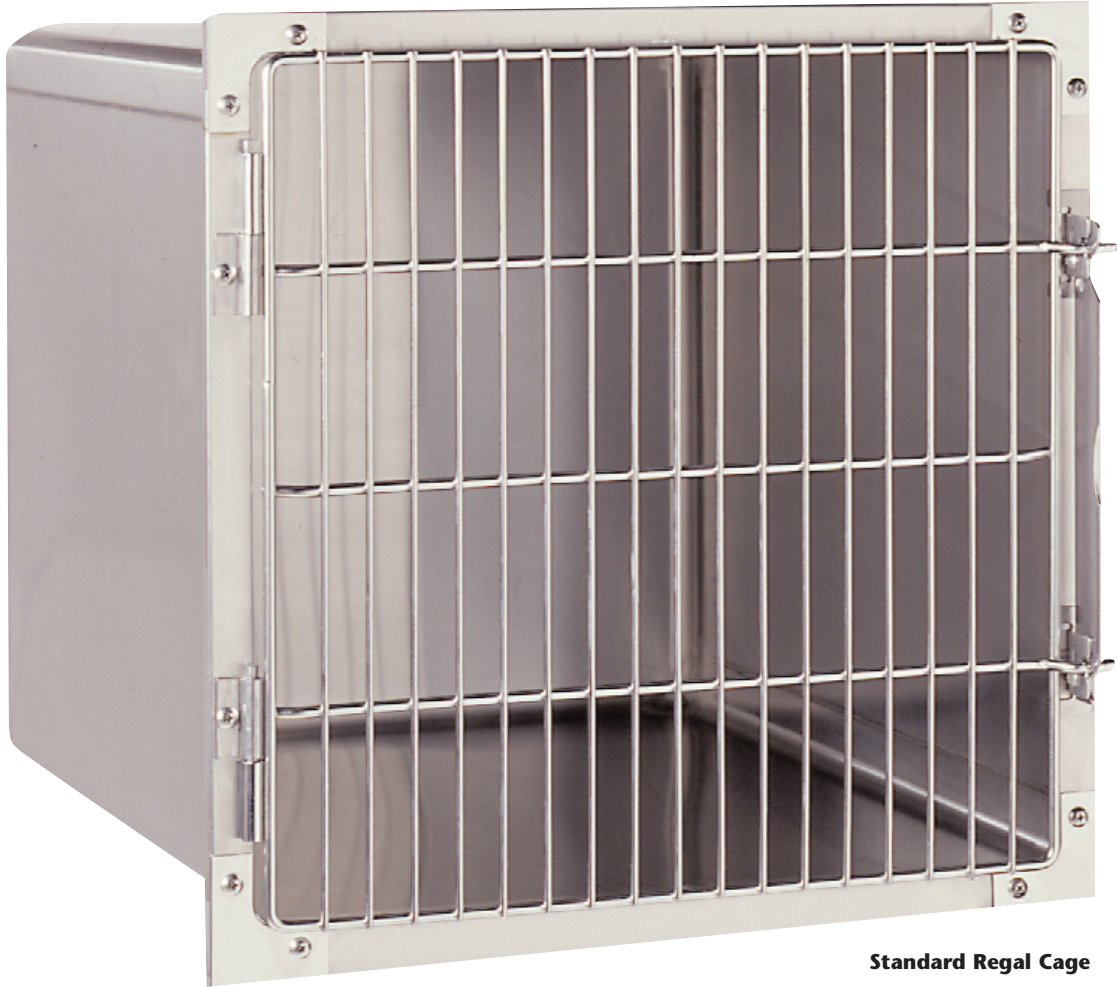

Standard Regal Cage

REGAL CAGES ARE AVAILABLE IN A WIDE VARIETY OF STYLES



Standard Double Door Regal Cage



Full-View Back Regal Cage

Regal Cages are meticulously engineered and crafted with design features you won't find anywhere else. Yet, it's our doors that really set our cages apart. Made entirely of stainless steel, the frame, crossmembers and vertical spokes are welded together at every intersection for exceptional structural integrity and strength. Our cage door mounts on the front of the cage, fitting close along the top and sides and overlapping the bottom of the cage opening to keep the animal's paws from becoming trapped between the door and the cage. This distinctive design also prevents paper liners from slipping out under the door.

Take a look at our latch and hinges. They're the strongest available, made from heavy 12-gauge stainless steel. Extra-strong barrel-style hinges surround the hinge pins to keep the doors fitting right and swinging smoothly for years to come. The self-latching door latch positively secures the door at two points without creating crevices for debris to hide. Latch disassembles easily for adjustment and cleaning. Our cage doors are ambidextrous and are shipped with doors hinged left but may be easily changed to be hinged right if desired without additional parts.

Our cages begin with a 22-gauge Type 304 stainless steel body, seam-welded into one piece to create the strongest cage on the market. The cage interior features .75-in. (1.90cm) radius side and back bends with coved back corners to eliminate any areas that could harbor dirt and fluids, making our cages easy to clean and disinfect. What's more, the floors of all of our cages have a fluid-retaining lip at the front edge to help keep urine and fluids from seeping out.

All Regal Cages can be used alone or readily assembled together to form the perfect cage arrangement for your facility. All installation hardware is included for your individual arrangement. Better yet, Regal Cage arrangements can easily be added to or rearranged at any time.

When you purchase Regal Cages for your facility, you'll know you've made an investment for your future.

Regal Cage Arrangements may be made with any style Regal Cage.



Regal Cage Resting Shelf Options

Make your cages more versatile with our stainless steel Shelves. Shelves are available for cages up to 36 in. wide. Shelves can be used to make room for multiple animals in one cage, or simply make cats more comfortable by allowing them to climb and rest higher than the floor. Cages can be purchased with factory-installed shelves or shelves can be purchased separately for installation by customer on new or existing cages. Shelves are mechanically-fastened thru the sides of the cage for stability. This prevents the shelf from ever tipping out of position. See Regal cage Finishing Accessories.

Regal Cages with factory-installed shelves are positioned 1.250-in. (3.17cm) from the back spanning the width of the cage and the shelf is midway between the floor and the ceiling of the cage. *Select which style Regal Cage you want from the following pages, then see the footnote below each chart to specify cages with shelves.*



Front & Back Door Regal Cage



Heated Regal Cage



Therapy Transport Regal Cage

Regal Cages

ALL STYLES OF REGAL CAGES ARE DESIGNED WITH THESE QUALITY FEATURES:

Door

Stainless steel door spokes are deburred smooth to eliminate sharp ends.

Stainless steel, electropolished removable ambidextrous door may be remounted with hinges on the right side.

Door extends below cage floor to help keep paper liners in place.

Door fits snug to the cage body to eliminate areas where an animal's paws could be pinched.

.1875-in. (.48cm) diameter stainless steel vertical spokes are spaced 1-in. apart and welded at every intersection to crossmembers and frame.

.3125-in. (.79cm) diameter stainless steel door frame and crossmembers are welded together for strength.

Hinges

.3125-in. (.79cm) diameter stainless steel hinge pins allow easy removal of door.

Heavy 12-gauge stainless steel die-formed barrel hinges operate smoothly for the life of the cage.

Hardware

Stainless Steel installation hardware is the strongest available. Vertical molding strips and 14-gauge bolt-on corner plates mount easily with .250-20 stainless steel truss-head machine screws, fasten into the cages front frame's weldnuts, which ensures the secure attaching of cages hardware.

Frame

18-gauge stainless steel die-formed floor support brace keeps floor rigid for animal comfort.

18-gauge stainless steel die-formed rear support keeps cage level.

Welded 18-gauge stainless steel die-formed front frame keeps the front of the cage square and extra strong.

The frame is also fitted with weldnuts, which ensures the secure attaching of cages hardware.

Latch

Stainless Steel Latch easily disassembles for sanitizing and adjusting.

.3125-in. (.79cm) diameter stainless steel latch keeper latches door securely.

Heavy 12-gauge stainless steel latch brackets.

Latch connector bar is notched for easy opening.

Heavy 12-gauge stainless steel self-latching door latch connector is designed to accommodate a padlock for greater security.

Body


22-gauge Type 304 stainless steel body is heli-arc welded into one piece to create the strongest cage available.

.750 in. (1.90cm) covered rear corners on top and bottom eliminate accumulation of dirt and bacteria.

.750-in. (1.90cm) radius bends on all sides and back provides a smooth, easy-to-clean interior surface.

Bottom lip overlaps lower cage when used in a Regal Cage arrangement

Distinctive floor design with fluid-retaining front lip is standard on all Regal Cages.

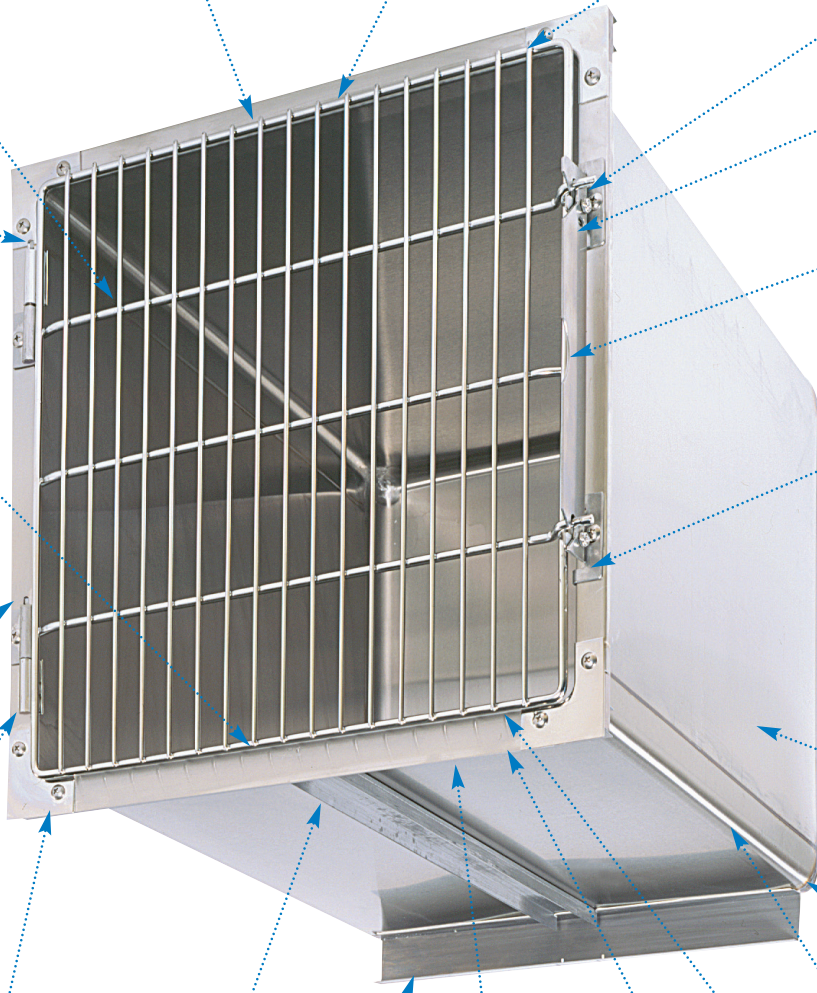


Standard Regal Cages

Two smaller doors allow you to have a large cage with less aisle space needed to open doors all the way.

14-gauge stainless steel latch: top and bottom plate latches door in two places.

Stainless steel channel holds Divider Panel in place.

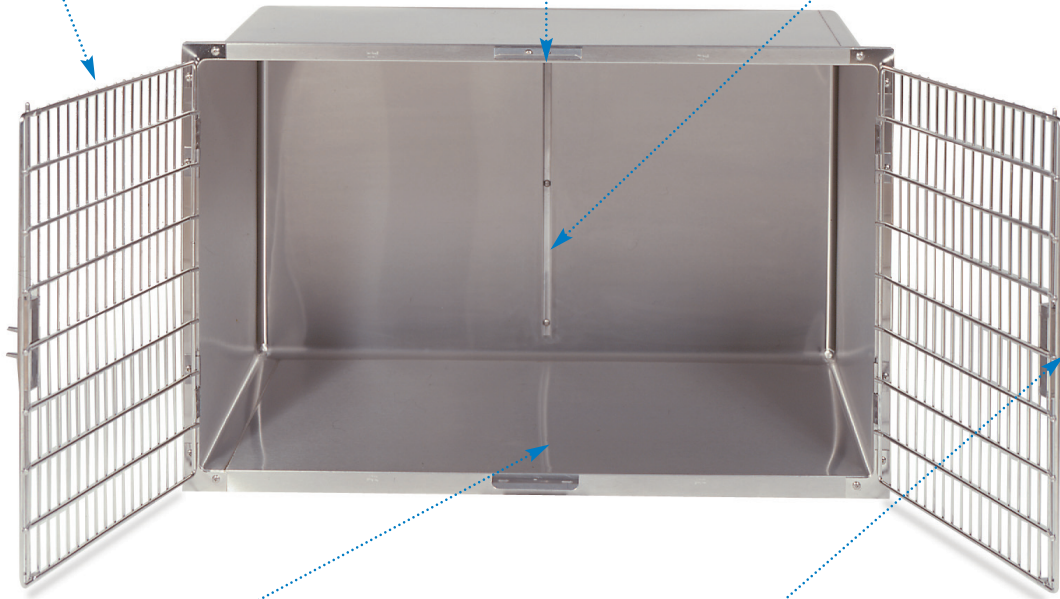

There's no center post to get in the way when placing animals in the cage.

Self-locking door latch secures door at top and bottoms.

**AVAILABLE IN
21 DIFFERENT SIZES —
MORE THAN ANYWHERE ELSE**

Standard Regal Cages

P/N	CAGE SIZE	
	Inches	Centimeters
12000-00-CTCTEI	18W x 18H	45.72W x 45.72H
12000-00-CTDREI	24W x 18H	60.96W x 45.72H
12000-00-DRCTEI	18W x 24H	45.72W x 60.96H
12000-00-DRDREI	24W x 24H	60.96W x 60.96H
12000-00-DREPEI	30W x 24H	76.20W x 60.96H
12000-00-DRFNEI	36W x 24H	91.44W x 60.96H
12001-00-DRHJEI	48W x 24H DD*	121.92W x 60.96H DD*
12000-00-EPDREI	24W x 30H	60.96W x 76.20H
12000-00-EPEPEI	30W x 30H	76.20W x 76.20H
12000-00-EPFNEI	36W x 30H	91.44W x 76.20H
12000-00-EPGLEI	42W x 30H	106.68W x 76.20H
12000-00-EPHJEI	48W x 30H	121.92W x 76.20H
12001-00-EPHJEI	48W x 30H DD*	121.92W x 76.20H DD*
12000-00-FNDREI	24W x 36H	60.96W x 91.44H

Standard Double Door (DD) Regal Cages

Double Door Cages allow you to have a large cage without a large door. Our double door cages have no center door-latching post for easy use and animal comfort. Large animals can be easily placed inside without having to maneuver around a center post. Doors have self-latching pins that slide into top and bottom latches. An optional stainless steel divider panel can be used to create two smaller cages for housing two animals. Channel included with each Double Door Cage is attached to back of cage to stabilize Divider Panel. See Regal Cage Finishing Accessories for Divider Panels.

P/N	CAGE SIZE	
	Inches	Centimeters
12000-00-FNEPEI	30W x 36H	76.20W x 91.44H
12000-00-FNFNEI	36W x 36H	91.44W x 91.44H
12000-00-FNGLEI	42W x 36H	106.68W x 91.44H
12000-00-FNHJEI	48W x 36H	121.92W x 91.44H
12001-00-FNHJEI	48W x 36H DD*	121.92W x 91.44H DD*
12001-00-FNJFEI	60W x 36H DD*	152.40W x 91.44H DD*
12001-00-FNLBEI	72W x 36H DD*	182.88W x 91.44H DD*

To order cages with factory-installed Resting Shelf, change the cage P/N 7th position 0, to: A Straight resting shelf

B Corner resting shelf in left back corner

C Corner resting shelf in right back corner

D Corner resting shelves in left and right back corners

Specify the height of the resting shelves above the cage floor, change the cage

P/N, 8th and 9th position 0- to: BF approximately midway in 18-in high cages

BR approximately midway in 24-in high cages

CD approximately midway in 30-in high cages

CP approximately midway in 36-in high cages

See dimension chart for other Resting Shelf heights

*DD after the height indicates a double door cage.

All cage bodies measure 28.25-in. (71.75cm) front to back.

Cages measure 30.188-in. (76.68cm) deep overall, including latch.

For Product Information Call TOLL-FREE 1-800-323-7366 • (847) 537-9320 • Fax (847) 537-9061

Hours: 8:30am - 5:00pm CST Monday - Friday

Visit our website at: www.suburbansurgical.com

Full-View Back Regal Cages

Vinyl gasket cushions back panel, providing a tight seal.

Clear back panel available in polycarbonate or tempered glass.


Full-perimeter stainless steel frame attaches rear full-view panel to cage for greater durability and a more finished appearance.

Full-width and height clear back panel offers an unobstructed view allowing observation without direct interaction in post-op, boarding and adoption areas.

Features the same high quality as our standard Regal Cages.

Vertical Molding Strips cover the space between two cages to keep liquids from seeping into hard-to-clean areas.

Corner Plates to join multiple cages together in the back to give a finished look.

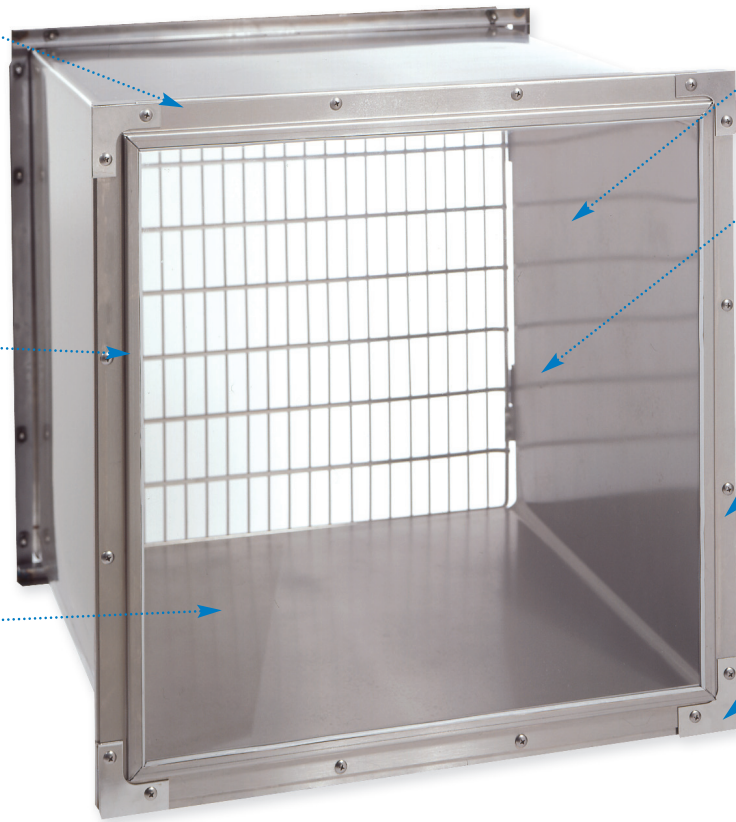

Full-View Back Regal Cages

These cages offer all the fine features of our standard Regal Cage with the addition of a uniquely designed full-view clear back panel designed to provide an unobstructed view of the animal inside the cage. Ideal for medical observation or for facilities where animals are on display to the public, these cages allow animals to be seen from either side of the cage.

Your choice of a clear polycarbonate or clear tempered glass panel is attached to the back of the cage with a full-perimeter, stainless steel frame and a vinyl gasket to provide a tight, leak-resistant seal.

POLYCARBONATE BACK PANEL	TEMPERED GLASS BACK PANEL	CAGE SIZE	
		Inches	Centimeters
12002-00-CTCTEI	12004-00-CTCTEI	18W x 18H	45.72W x 45.72H
12002-00-CTDREI	12004-00-CTDREI	24W x 18H	60.96W x 45.72H
12002-00-DRCTEI	12004-00-DRCTEI	18W x 24H	45.72W x 60.96H
12002-00-DRDREI	12004-00-DRDREI	24W x 24H	60.96W x 60.96H
12002-00-DREPEI	12004-00-DREPEI	30W x 24H	76.20W x 60.96H
12002-00-DRFNEI	12004-00-DRFNEI	36W x 24H	91.44W x 60.96H
12003-00-DRHJEI	12005-00-DRHJEI	48W x 24H DD*	121.92W x 60.96H DD*
12002-00-EPDREI	12004-00-EPDREI	24W x 30H	60.96W x 76.20H
12002-00-EPEPEI	12004-00-EPEPEI	30W x 30H	76.20W x 76.20H
12002-00-EPFNEI	12004-00-EPFNEI	36W x 30H	91.44W x 76.20H
12002-00-EPGLEI	12004-00-EPGLEI	42W x 30H	106.68W x 76.20H
12002-00-EPHJEI	12004-00-EPHJEI	48W x 30H	121.92W x 76.20H
12003-00-EPHJEI	12005-00-EPHJEI	48W x 30H DD*	121.92W x 76.20H DD*
12002-00-FNDREI	12004-00-FNDREI	24W x 36H	60.96W x 91.44H
12002-00-FNEPEI	12004-00-FNEPEI	30W x 36H	76.20W x 91.44H
12002-00-FNFNEI	12004-00-FNFNEI	36W x 36H	91.44W x 91.44H
12002-00-FNGLEI	12004-00-FNGLEI	42W x 36H	106.68W x 91.44H
12002-00-FNHJEI	12004-00-FNHJEI	48W x 36H	121.92W x 91.44H
12003-00-FNHJEI	12005-00-FNHJEI	48W x 36H DD*	121.92W x 91.44H DD*
12003-00-FNJFEI	12005-00-FNJFEI	60W x 36H DD*	152.40W x 91.44H DD*
12003-00-FNLBEI	12005-00-FNLBEI	72W x 36H DD*	182.88W x 91.44H DD*

To order cages with factory-installed Resting Shelf, change the cage P/N 7th position 0, to: A Straight resting shelf

B Corner resting shelf in left back corner

C Corner resting shelf in right back corner

D Corner resting shelves in left and right back corners

Specify the height of the resting shelves above the cage floor, change the cage

P/N, 8th and 9th position 0- to: BF approximately midway in 18-in high cages

BR approximately midway in 24-in high cages

CD approximately midway in 30-in high cages

CP approximately midway in 36-in high cages

See dimension chart for other Resting Shelf heights


*DD after the height indicates a double door cage.

All cage bodies measure 28.25-in. (71.75cm) front to back.

Cages measure 30.188-in. (76.68cm) deep overall, including latch.

Front & Back Door Regal Cages

Features the same high quality design as our standard Regal Cages.

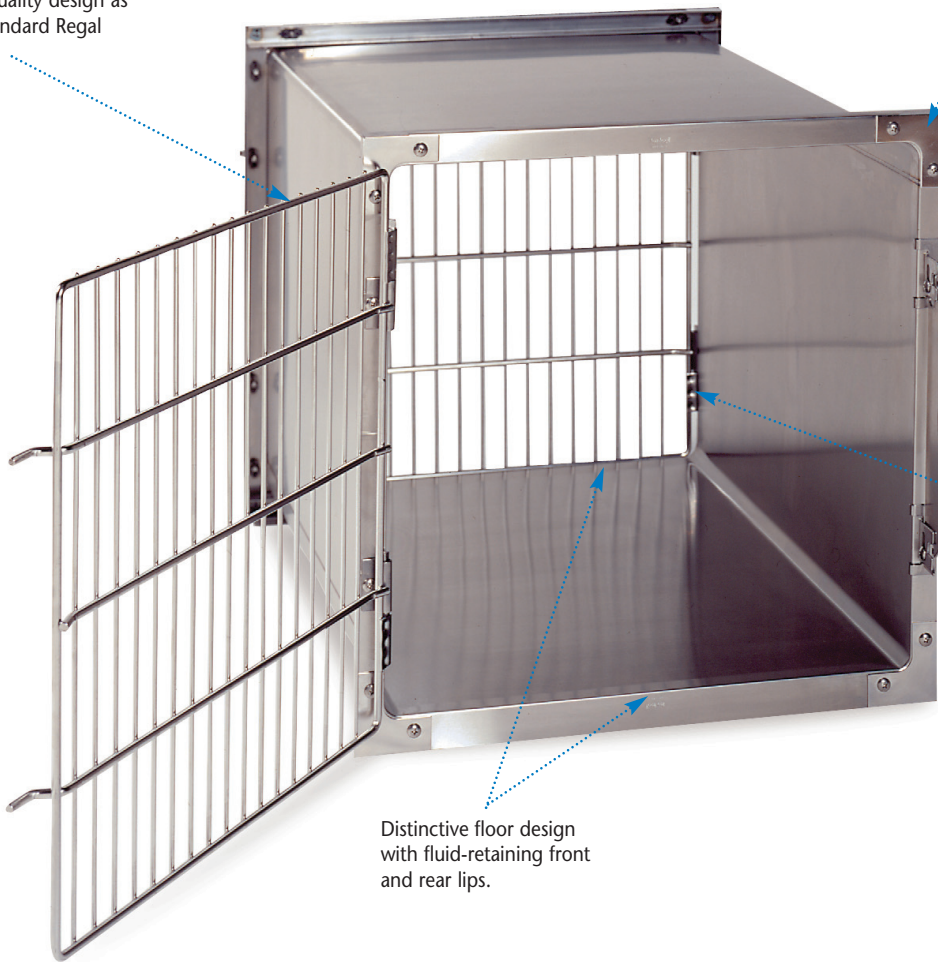

Corner Plates to join multiple cages together in the back to give a finished look.

Vertical Molding Strips cover the space between two cages to keep liquids from seeping into hard-to-clean areas.

Heavy, 12-gauge stainless steel latches and hinges on both front and rear doors offer smooth operation and secure latching.

Distinctive floor design with fluid-retaining front and rear lips.

Front & Back Door Regal Cages

These Regal Cages may be built into a wall or a reception counter to allow you to access animals from either side of the cage. They are ideal for transferring animals to owners in the waiting room, or for after-hours drop-off.

Our cage doors are ambidextrous and are shipped with doors hinged left but may be easily changed to be hinged right if desired without additional parts.

P/N	CAGE SIZE	
	Inches	Centimeters
12010-00-CTCTEI	18W x 18H	45.72W x 45.72H
12010-00-CTDREI	24W x 18H	60.96W x 45.72H
12010-00-DRCTEI	18W x 24H	45.72W x 60.96H
12010-00-DRDREI	24W x 24H	60.96W x 60.96H
12010-00-DREPEI	30W x 24H	76.20W x 60.96H
12010-00-DRFNEI	36W x 24H	91.44W x 60.96H
12011-00-DRHJEI	48W x 24H DD*	121.92W x 60.96H DD*
12010-00-EPDREI	24W x 30H	60.96W x 76.20H
12010-00-EPEPEI	30W x 30H	76.20W x 76.20H
12010-00-EPFNEI	36W x 30H	91.44W x 76.20H
12010-00-EPGLEI	42W x 30H	106.68W x 76.20H
12010-00-EPHJEI	48W x 30H	121.92W x 76.20H
12011-00-EPHJEI	48W x 30H DD*	121.92W x 76.20H DD*
12010-00-FNDREI	24W x 36H	60.96W x 91.44H

P/N	CAGE SIZE	
	Inches	Centimeters
12010-00-FNEPEI	30W x 36H	76.20W x 91.44H
12010-00-FNFNEI	36W x 36H	91.44W x 91.44H
12010-00-FNGLEI	42W x 36H	106.68W x 91.44H
12010-00-FNHJEI	48W x 36H	121.92W x 91.44H
12011-00-FNHJEI	48W x 36H DD*	121.92W x 91.44H DD*
12011-00-FNJFEI	60W x 36H DD*	152.40W x 91.44H DD*
12011-00-FNLBEI	72W x 36H DD*	182.88W x 91.44H DD*

To order cages with factory-installed Resting Shelf, change the cage P/N 7th position 0, to: A Straight resting shelf

B Corner resting shelf in left back corner

C Corner resting shelf in right back corner

D Corner resting shelves in left and right back corners

Specify the height of the resting shelves above the cage floor, change the cage P/N, 8th and 9th position 0- to: BF approximately midway in 18-in high cages

BR approximately midway in 24-in high cages

CD approximately midway in 30-in high cages

CP approximately midway in 36-in high cages

See dimension chart for other Resting Shelf heights

*DD after the height indicates a double door cage.

All cage bodies measure 28.25-in. (71.75cm) front to back.

Cages measure 32.125-in. (81.60cm) deep overall, including latch.

For Product Information Call TOLL-FREE 1-800-323-7366 • (847) 537-9320 • Fax (847) 537-9061

Hours: 8:30am - 5:00pm CST Monday - Friday

Visit our website at: www.suburbansurgical.com

Heated Regal Cages



Features the same high quality design as our standard Regal Cages.

Heated Regal Cages

Place your patients in our Heated Regal Cages for extra warmth and comfort during recovery from surgery or treatments. Specially designed electric heating element is laminated to the underside of the Cage to distribute steady, uniform heat over the entire floor, helping to maintain the animal's body temperature for greater comfort and faster recovery.

The floor temperature of each Heated Regal Cage in an Arrangement is controlled by a Temperature Controller. An easy-to-read LCD display allows you to adjust and monitor the floor temperature for individual patients. Built-in thermostat ensures steady, even heat. Temperature controller for each cage is conveniently located on control panel mounted on top of the cage arrangement.

Heater

Heater is laminated to underside of cage floor to provide even, controlled heat to keep animal comfortable during recovery.

Heated Regal Cages are available preassembled in a select number of Heated Regal Cage Arrangements. Look for this symbol () on Arrangements. Heated Regal Cages may also be used in any other customer-assembled Heated Regal Cage Arrangements.*

P/N	Volts/Amps	CAGE SIZE	
		Inches	Centimeters
12012-00-CTCTE20V AC/1.03A▲	18W x 18H	45.72W x 45.72H	
12012-00-CTDRE20V AC/1.03A▲	24W x 18H	60.96W x 45.72H	
12012-00-DRCTE20V AC/1.03A▲	18W x 24H	45.72W x 60.96H	
12012-00-DRDRE20V AC/1.03A▲	24W x 24H	60.96W x 60.96H	
12012-00-DREPE20V AC/1.38A▲	30W x 24H	76.20W x 60.96H	
12012-00-DRFNE20V AC/1.71A▲	36W x 24H	91.44W x 60.96H	
12013-00-DRHJE20V AC/2.06A▲	48W x 24H DD*	121.92W x 60.96H DD*	
12012-00-EPDRE20V AC/1.03A▲	24W x 30H	60.96W x 76.20H	
12012-00-EPEPE20V AC/1.38A▲	30W x 30H	76.20W x 76.20H	
12012-00-EPFNE20V AC/1.71A▲	36W x 30H	91.44W x 76.20H	
12012-00-EPGLE20V AC/1.71A▲	42W x 30H	106.68W x 76.20H	
12012-00-EPHJE20V AC/2.06A▲	48W x 30H	121.92W x 76.20H	
12013-00-EPHJE20V AC/2.06A▲	48W x 30H DD*	121.92W x 76.20H DD*	
12012-00-FNDRE20V AC/1.03A▲	24W x 36H	60.96W x 91.44H	

▲ 220V requires step-down transformer. Contact SSCI.
 *DD after the height indicates a double door cage.
 All cage bodies measure 28.25-in. (71.75cm) front to back.
 Cages measure 30.188-in. (76.68cm) deep overall, including latch.

P/N	Volts/Amps	CAGE SIZE	
		Inches	Centimeters
12012-00-FNEPEI	120V AC/1.38A▲	30W x 36H	76.20W x 91.44H
12012-00-FNFNEI	120V AC/1.71A▲	36W x 36H	91.44W x 91.44H
12012-00-FNGLEI	120V AC/1.71A▲	42W x 36H	106.68W x 91.44H
12012-00-FNHJEI	120V AC/2.06A▲	48W x 36H	121.92W x 91.44H
12013-00-FNHJEI	120V AC/2.06A▲	48W x 36H DD*	121.92W x 91.44H DD*
12013-00-FNJFEI	120V AC/2.76A▲	60W x 36H DD*	152.40W x 91.44H DD*
12013-00-FNLBEI	120V AC/3.42A▲	72W x 36H DD*	182.88W x 91.44H DD*

To order cages with factory-installed Resting Shelf, change the cage P/N 7th position 0, to: A Straight resting shelf
 B Corner resting shelf in left back corner
 C Corner resting shelf in right back corner
 D Corner resting shelves in left and right back corners
 Specify the height of the resting shelves above the cage floor, change the cage P/N, 8th and 9th position 0- to: BF approximately midway in 18-in high cages
 BR approximately midway in 24-in high cages
 CD approximately midway in 30-in high cages
 CP approximately midway in 36-in high cages
 See dimension chart for other Resting Shelf heights

Regal Cage Arrangement Panels

Heat Cage Control Panel with 110V Outlets, Electrical Panel with 110V Outlets, Oxygen Manifold Panel and Oxygen Manifolds*, are available for many Regal Cage Arrangements.

Each panel is designed to support the specific number of cages in the arrangement and is the overall width of the cage arrangement.

Heated Cage Control Panel with 110V Outlets, allow you to control the temperature of the cage floor. Control Panel Includes: Easy-to-Read Temperature Display(s) for Each Heated Cage and One 110V outlet for accessories, each with an individual fuse. These Features Allow You to Monitor the Temperature of the Heated Floor with the Control Dial to a Maximum Temperature of 105°F. Amber Light Indicates Power Supply to Floor and Illuminated ON/OFF Button Lights When Unit is Turned On.

Electric Panel supplies convenient access to 110V electric outlets. There is one outlet for each cage in the arrangement.


Oxygen Panel for Mounting the Oxygen Manifold

Oxygen manifold provides you a convenient supply of oxygen. There is an outlet for each cage in the arrangement and mounts on top of the chosen panel.

Stainless Steel Heated Cage Control Panel with 110V Outlets



Stainless Steel Electric Panel with 110V Outlets

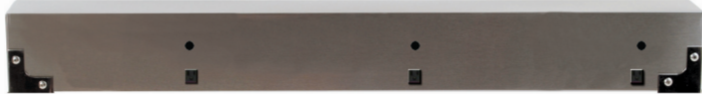

Stainless Steel Oxygen Panel for Mounting Oxygen Manifold



P/N	DESCRIPTION
12180-__-__ AVAT	SS Heated Cage Control Panel with Controller(s) and Electric Outlet(s) for Regal Cage Arrangements
12190-__-__ ALAT	SS Electric outlet panel with electrical outlet(s) for Regal Cage Arrangements
12191-00-__ ALAT	SS oxygen panel for Regal Cage Arrangements

Stainless Steel Oxygen Manifold

P/N	DESCRIPTION
12185-__-__ AEAK	SS Oxygen Manifold with flow meter outlet(s) for Regal Cage arrangements
<i>Flow Meters are sold separately</i>	
853456	Oxygen Flow Meter-15 LPM



Regal Cage Arrangements

Bolt-on 14-gauge stainless steel corner plates secure cages together to create a single, solid unit.

Bottom front lip overlaps the cage below to eliminate fluid seepage between cages.

Shown with optional stainless steel mobile Base. Increases arrangement's overall height by 6.75-in. (17.15cm).

Vertical molding strips eliminate gaps between cages where dirt can hide.



*For
Bases, Top and
End Panels, see
Regal Cage
Finishing
Accessories*

**Regal Cage Arrangement
shown with
Stainless Steel Mobile Base**

The beauty of our Regal Cages is that the arrangement possibilities are endless. Regal cages are available in more sizes than other cages, more sizes allow you to create ideal cage arrangement(s) for your facility. Regal cages are also available in various types, standard, full-view back, front and back door, and heated, to fulfill the requirements for various areas in your facility. Some types of Regal cages can be combined in the same cage arrangement(s) for more flexibility. Better yet, if your needs or available space change, the arrangement can be added to or rearranged to fit the new requirements.

You can even reverse the doors to open the other way if you find it would work better for your needs. And when you add some of our finishing accessories shown on the following pages, you'll find that Regal Cages are not only the best, but the most versatile you can buy.

Each cage arrangement includes vertical molding strips and bolt-on corner plates made of heavy 14-gauge stainless steel designed to give the entire arrangement structural integrity with a finished appearance. The lower edge of the front of each cage overlaps the top of the cage below or the mounting base to insure that no liquids seep between the cages.

Cages may be mounted on a SSCI stainless steel Base with or without casters, or on your wooden or concrete platform.

A variety of Cage Arrangements are shown on the following pages. But, if you don't see an Arrangement that's ideal for your facility, we will work with you to create the perfect arrangement of cage styles and sizes for your needs.



Design Service For Regal Cage Arrangements

THE PERFECT REGAL CAGE ARRANGEMENT IS JUST A PHONE CALL AWAY

SSCI's Design Service is offered to customers after discussing and agreeing on a viable project with an SSCI Representative. After a methodical review of the customer's needs for their facility, the SSCI Representative will present the project to our staff of designers to create a set of approval drawings. The drawings ensure that SSCI will supply exactly what the customer

has discussed with the SSCI Representative. Once the customer has committed to purchase from SSCI and, if the drawings require revisions, the Final Approval Drawings will be created and presented to the customer for their approval. Other Customer Requirements may be required before manufacturing time can begin.

CREATING THE PERFECT REGAL CAGE ARRANGEMENT TAKES JUST 5 EASY STEPS:

- 1 Measure your space.**
Obtain overall width, height and depth of the available space in your facility where you plan to place your Regal Cage Arrangement. To allow for the cage, the minimum depth of the area must be 29-in. (73.66cm) plus the width of the widest cage door in the arrangement. This will allow cage doors to open fully. Make sure you have sufficient space in front of the cage arrangement for your staff to handle patients safely.
- 2 Choose the style of the Regal Cages in your Arrangement.**
Our Regal Cage styles give four options.
- 3 Identify the Regal Cage sizes you require.**
A variety of sizes can be used in a single Arrangement, as long as the overall width and height of each tier is the same. An idea of the size of animals you will house in your Cages will help you select the sizes you need.
- 4 Determine how your Cages will be arranged.**
Some Regal Cage Arrangements are shown on the following pages. If you don't see an Arrangement that will work for you, call us with your space measurements, and we will help you design a Regal Cage Arrangement that will include the largest cages for the total number of animals you wish to accommodate.
- 5 Regal Cage Arrangements should be set up on a base.**

Optional SSCI Stainless Steel Base:

(add casters to create a mobile base).

Customer-Supplied Bases:


Wood Platform Bases should be constructed level front to back and right to left of 2x4s measuring the width of your Arrangement and 29-in. (73.66cm) front to back.

Trowel-finished Concrete Pad should be poured level 3.5-in. (8.90cm) high and 29-in. (73.66cm) front to back.

If one of our illustrated Arrangements shown on the following pages won't fit your facility perfectly, let our designers help you create the ideal Arrangement for your needs. SSCI's design team will work with your specifications to create a Cage Arrangement suited to your unique needs, space requirements and budget. And if you already own some cages, even some of our competitors' cages, we can often incorporate those into your Cage Arrangement design.


It all starts with a phone call. Contact one of our

representatives to discuss your requirements. We'll create a CAD-generated drawing of your Arrangement complete with all hardware needed for installation.

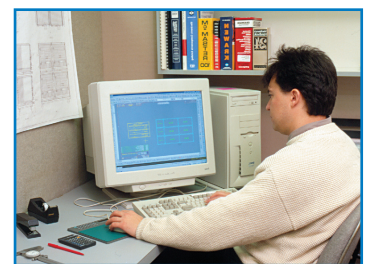

Our team of experienced designers will work with your measurements, cage style and size and budget to create the perfect Cage Arrangement for your requirements.

FOR EVERY REGAL ARRANGEMENT PURCHASE SSCI CREATES A DRAWING TO ENSURE THE CUSTOMER'S SATISFACTION

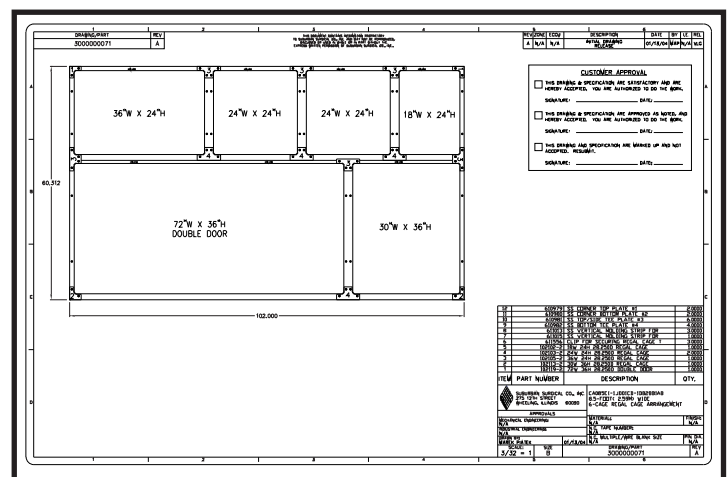

SSCI creates an Approval Drawing for each cage arrangement ordered. These detailed drawings of your cage arrangement list cage sizes, quantities and all appropriate hardware and illustrate how the arrangement is to be assembled. If you choose to alter the positions of the cages in the arrangement, contact your SSCI representative, so that a new approval drawing can be created. The new arrangement might require different hardware. If the cage positions are changed after the drawings are approved and the cages and hardware have been shipped, you will be required to purchase additional hardware to assemble your arrangement.

In addition to other customer requirements, the Final Approval Drawings must be signed and received by SSCI prior to the beginning of manufacturing lead time.

For Product Information Call TOLL-FREE 1-800-323-7366 • (847) 537-9320 • Fax (847) 537-9061

Hours: 8:30am - 5:00pm CST Monday - Friday

Visit our website at: www.suburbansurgical.com

HEATED REGAL CAGE ARRANGEMENTS

HEATED REGAL CAGE ARRANGEMENTS UP TO TWO TIERS HIGH AND 6-FT. (1.83M) WIDE ARE SHIPPED PREASSEMBLED.



Heated Regal Cage Arrangement on Stainless Steel Mobile Base

We preassemble a select number of Heated Regal Cage arrangements that can be shipped and moved into most facilities. These arrangements include a full-width top-mounted control panel with a temperature controller for each cage in the arrangement. The pre-wired and pre-assembled arrangement plugs into a 120V AC ▲ facility wall outlet for electric power. Each cage's floor temperature is regulated by a dedicated adjustable temperature controller that displays the cage's floor temperature.


Preassembled Heated Arrangement's are only sold with Stainless Steel, Top and End Panel, Finishing Accessories to protect the wiring and for a finished appearance. The Arrangements are either mounted on a stainless steel base or Mobile stainless steel base, unitizing the arrangement for shipping.

Preassembled Arrangements are available in two-tier high Arrangements up to 6-ft. (1.83m) wide only. Look for this symbol (*) after the part number and description of the Arrangements on the following pages to see which Heated Regal Cage arrangements are shipped preassembled.

Please Note: Heated Regal Cages and control panels may be ordered for all Regal Cage Arrangements; however, assembly of the cages and connection of the pre-wired control panel to each cage's floor heater will be required at your facility.

For customer-assembled arrangements, we recommend you purchase optional stainless steel top and end panels.

Regal Cage Arrangements

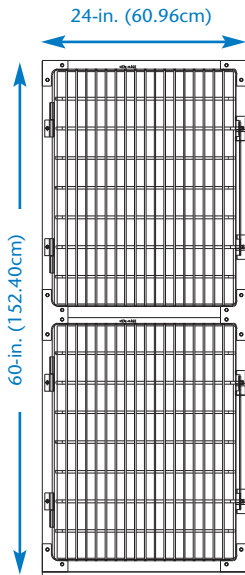

2-Foot (.61m) Wide, 2-Cage Regal Cage Arrangement


Includes:

Top:

One 24-in.W x 30-in.H (60.96cmW x 76.20cmH) Cage

Bottom:

One 24-in.W x 30-in.H (60.96cmW x 76.20cmH) Cage

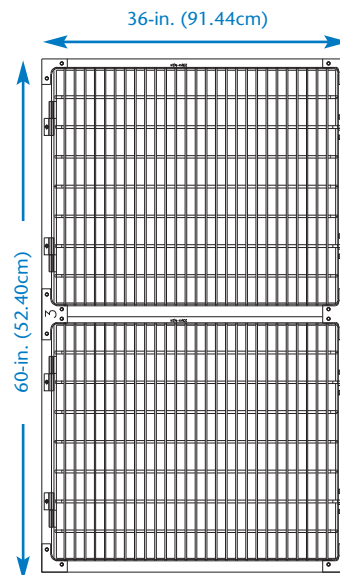

3-Foot (.91m) Wide, 2-Cage Regal Cage Arrangement

Includes:

Top:

One 36-in.W x 30-in.H (91.44cmW x 76.20cmH) Cage

Bottom:

One 36-in.W x 30-in.H (91.44cmW x 76.20cmH) Cage

P/N	Description
300000170	Regal Cage Arrangement
3000003001	Full-View Polycarbonate Back Regal Cage Arrangement
3000004000	Full-View Tempered Glass Back Regal Cage Arrangement
3000005000	Front & Back Door Regal Cage Arrangement
3000006026	Heated Regal Cage Arrangement: 120V AC ▲, Fahrenheit Thermostat, on S.S. Base*
3000006006	Heated Regal Cage Arrangement: 120V AC ▲, Fahrenheit Thermostat, on S.S. Base with Casters*

P/N	Description
3000000202	Regal Cage Arrangement
3000003017	Full-View Polycarbonate Back Regal Cage Arrangement
3000004015	Full-View Tempered Glass Back Regal Cage Arrangement
3000005015	Front & Back Door Regal Cage Arrangement
3000006028	Heated Regal Cage Arrangement: 120V AC ▲, Fahrenheit Thermostat, on S.S. Base*
3000006007	Heated Regal Cage Arrangement: 120V AC ▲, Fahrenheit Thermostat, on S.S. Base with Casters*


4-Foot (1.22m) Wide, 3-Cage Regal Cage Arrangement

Includes:

Top: Two 24-in.W x 24-in.H (60.96cmW x 60.96cmH) cages

Bottom: One 48-in.W x 30-in.H (121.92cmW x 72.20cmH) DD cage

P/N	Description
3000000021	Regal Cage Arrangement
3000003002	Full-View Polycarbonate Back Regal Cage Arrangement
3000004001	Full-View Tempered Glass Back Regal Cage Arrangement
3000005001	Front & Back Door Regal Cage Arrangement
3000006030	Heated Regal Cage Arrangement: 120V AC ▲, Fahrenheit Thermostat, on S.S. Base*
3000006012	Heated Regal Cage Arrangement: 120V AC ▲, Fahrenheit Thermostat, on S.S. Base with Casters*

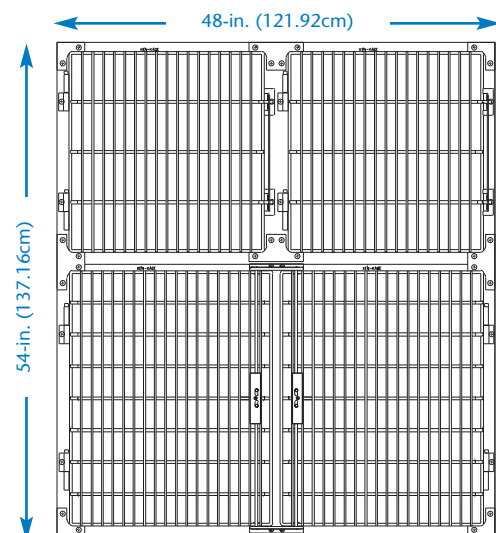

*Arrangements shipped pre-assembled. ▲ 220V requires step-down transformer. Contact SSCI.

Regal Cage Arrangements



4-Foot (1.22m) Wide, 3-Cage Regal Cage Arrangement

Includes:

Top: Two 24-in.W x 30-in.H (60.96cmW x 76.20cmH) cages

Bottom: One 48-in.W x 30-in.H (121.92cmW x 76.20cmH) DD cage

P/N	Description
3000000004	Regal Cage Arrangement
3000003003	Full-View Polycarbonate Back Regal Cage Arrangement
3000004002	Full-View Tempered Glass Back Regal Cage Arrangement
3000005002	Front & Back Door Regal Cage Arrangement
3000006045	Heated Regal Cage Arrangement: 120V AC ▲, Fahrenheit Thermostat, on S.S. Base*
3000006013	Heated Regal Cage Arrangement: 120V AC ▲, Fahrenheit Thermostat, on S.S. Base with Casters*

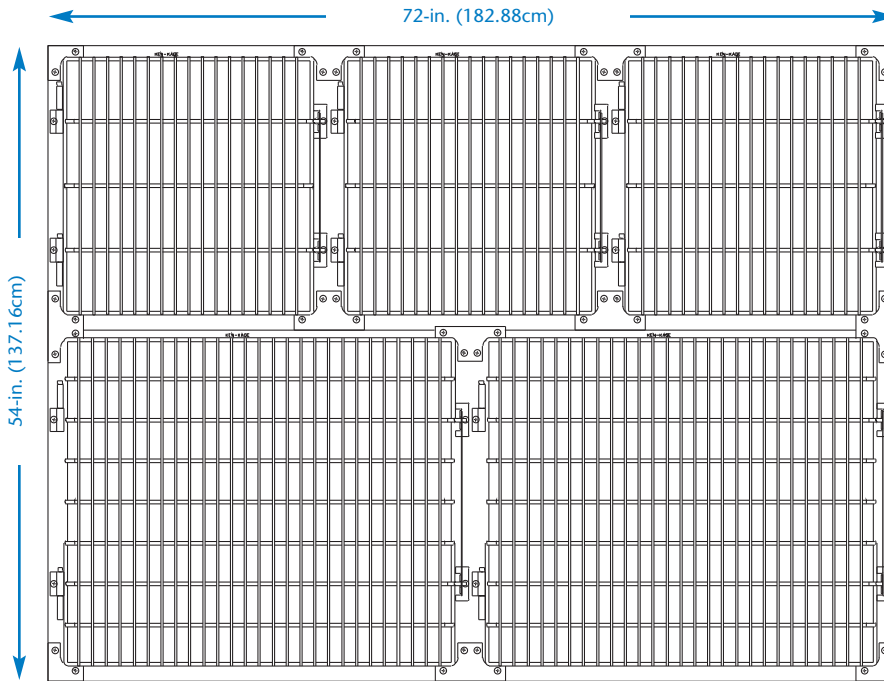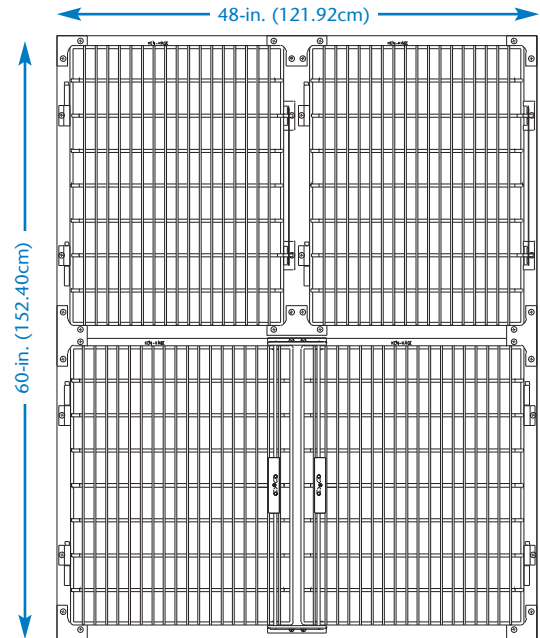

6-Foot (1.83m) Wide, 5-Cage Regal Cage Arrangement

Includes:


Top: Three 24-in.W x 24-in.H (60.96cmW x 60.96cmH) cages

Bottom: Two 36-in.W x 30-in.H (91.44cmW x 76.20cmH) cages

P/N	Description
3000000000	Regal Cage Arrangement
3000003004	Full-View Polycarbonate Back Regal Cage Arrangement
3000004003	Full-View Tempered Glass Back Regal Cage Arrangement
3000005003	Front & Back Door Regal Cage
3000006034	Heated Regal Cage Arrangement: 120V AC ▲, Fahrenheit Thermostat, on S.S. Base*
3000006021	Heated Regal Cage Arrangement: 120V AC ▲, Fahrenheit Thermostat, on S.S. Base with Casters*

*Arrangements shipped pre-assembled. ▲ 220V requires step-down transformer. Contact SSCI.

Regal Cage Arrangements

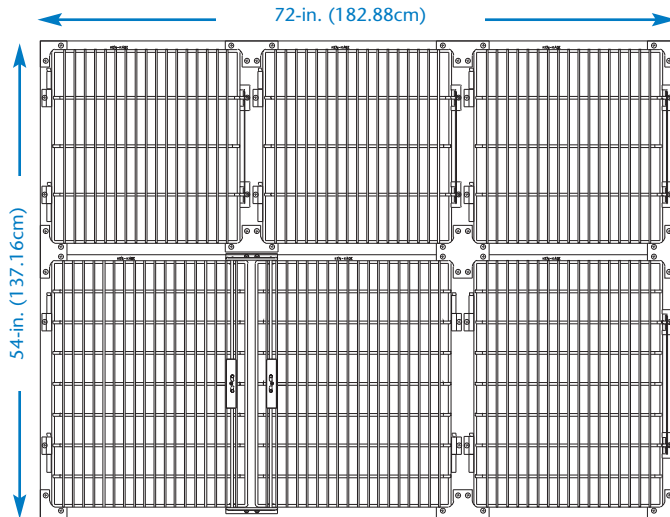

6-Foot (1.83m) Wide, 5-Cage Regal Cage Arrangement

Includes:

- Top:** Three 24-in.W x 24-in.H (60.96cmW x 60.96cmH) cages
- Bottom:** One 48-in.W x 30-in.H (121.92cmW x 76.20cmH) DD cage
One 24-in.W x 30-in.H (60.96cmW x 76.20cmH) cage



P/N	Description
300000117	Regal Cage Arrangement
3000003005	Full-View Polycarbonate Back Regal Cage Arrangement
3000004004	Full-View Tempered Glass Back Regal Cage Arrangement
3000005004	Front & Back Door Regal Cage Arrangement
3000006046	Heated Regal Cage Arrangement: 120V AC ▲, Fahrenheit Thermostat, on S.S. Base*
3000006023	Heated Regal Cage Arrangement: 120V AC ▲, Fahrenheit Thermostat, on S.S. Base with Casters*

6-Foot (1.83m) Wide, 5-Cage Regal Cage Arrangement

Includes:

- Top:** Three 24-in.W x 30-in.H (60.96cmW x 76.20cmH) cages
- Bottom:** Two 36-in.W x 30-in.H (91.44cmW x 76.20cmH) cages

P/N	Description
3000000023	Regal Cage Arrangement
3000003006	Full-View Polycarbonate Back Regal Cage Arrangement
3000004005	Full-View Tempered Glass Back Regal Cage Arrangement
3000005005	Front & Back Door Regal Cage Arrangement
3000006047	Heated Regal Cage Arrangement: 120V AC ▲, Fahrenheit Thermostat, on S.S. Base*
3000006022	Heated Regal Cage Arrangement: 120V AC ▲, Fahrenheit Thermostat, on S.S. Base with Casters*

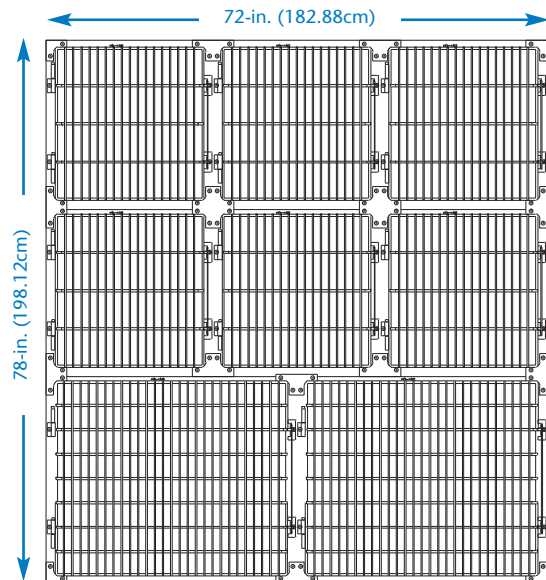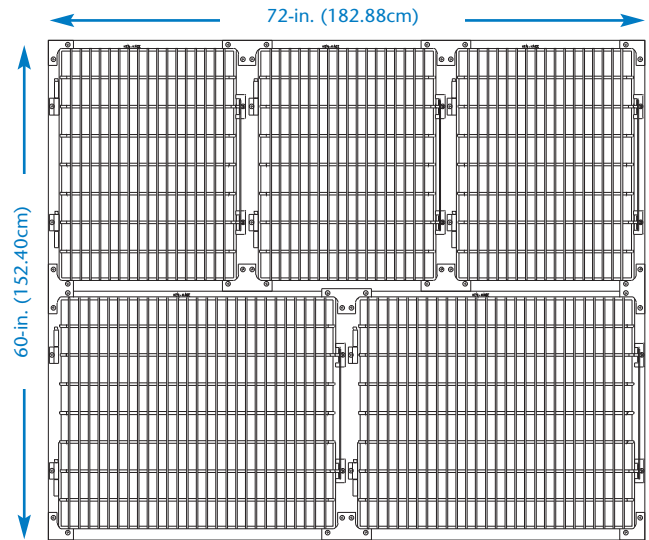

6-Foot (1.83m) Wide, 8-Cage Regal Cage Arrangement

Includes:


- Top:** Three 24-in.W x 24-in.H (60.96cmW x 60.96cmH) cages
- Middle:** Three 24-in.W x 24-in.H (60.96cmW x 60.96cmH) cages
- Bottom:** Two 36-in.W x 30-in.H (91.44cmW x 76.20cmH) cages

P/N	Description
3000000092	Regal Cage Arrangement
3000003007	Full-View Polycarbonate Back Regal Cage Arrangement
3000004006	Full-View Tempered Glass Back Regal Cage Arrangement
3000005006	Front & Back Door Regal Cage Arrangement
3000006048	Heated Regal Cage Arrangements, 120V AC ▲*

*Arrangements shipped pre-assembled. ▲ 220V requires step-down transformer. Contact SSC1.

For Product Information Call TOLL-FREE 1-800-323-7366 • (847) 537-9320 • Fax (847) 537-9061
Hours: 8:30am - 5:00pm CST Monday - Friday
Visit our website at: www.suburbansurgical.com

Regal Cage Arrangements

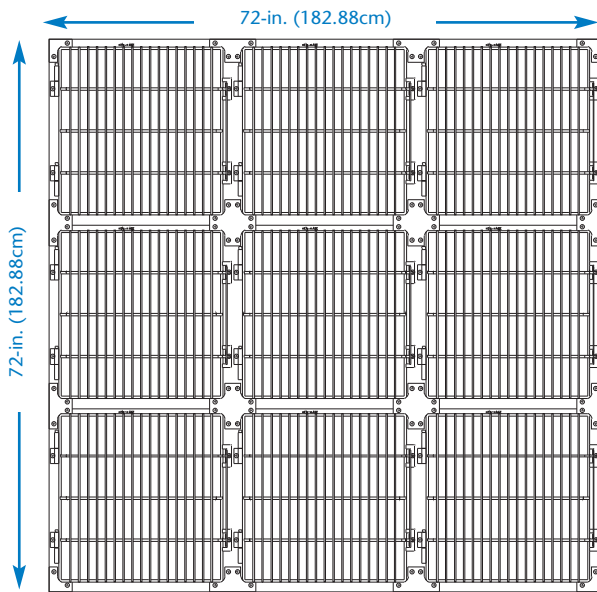

6-Foot (1.83m) Wide, 9-Cage Regal Cage Arrangement

Includes:

Top: Three 24-in.W x 24-in.H (60.96cmW x 60.96cmH) cages

Middle: Three 24-in.W x 24-in.H (60.96cmW x 60.96cmH) cages

Bottom: Three 24-in.W x 24-in.H (60.96cmW x 60.96cmH) cages

P/N	Description
300000015	Regal Cage Arrangement
3000003016	Full-View Polycarbonate Back Regal Cage Arrangement
3000004007	Full-View Tempered Glass Back Regal Cage Arrangement
3000005007	Front & Back Door Regal Cage Arrangement
3000006049	Heated Regal Cage Arrangement, 120V AC ▲ *

8-Foot (2.44m) Wide, 9-Cage Regal Cage Arrangement

Includes:


Top: Four 24-in.W x 24-in.H (60.96cmW x 60.96cmH) cages

Middle: Two 36-in.W x 30-in.H (91.44cmW x 76.20cmH) cages

One 24-in.W x 30-in.H (60.96cmW x 76.20cmH) cage

Bottom: Two 48-in.W x 30-in.H (121.92cmW x 76.20cmH) DD cages

P/N	Description
3000000019	Regal Cage Arrangement
3000003009	Full-View Polycarbonate Back Regal Cage Arrangement
3000004008	Full-View Tempered Glass Back Regal Cage Arrangement
3000005008	Front & Back Door Regal Cage Arrangement
3000006050	Heated Regal Cage Arrangement, 120V AC ▲ *

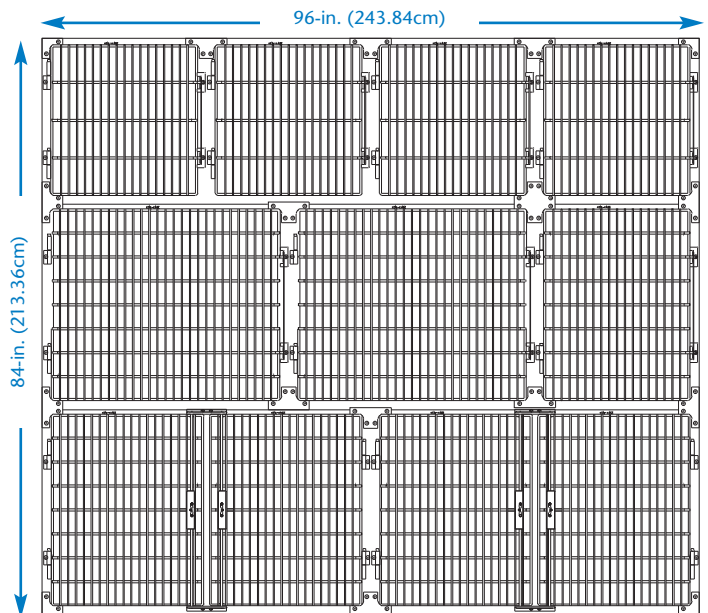

*Arrangements shipped pre-assembled. ▲ 220V requires step-down transformer. Contact SSCI.


Regal Cage Arrangements

8-Foot (2.44m) Wide, 10-Cage Regal Cage Arrangement

Includes:

- Top:** Four 24-in.W x 24-in.H (60.96cmW x 60.96cmH) cages
- Middle:** Four 24-in.W x 24-in.H (60.96cmW x 60.96cmH) cages
- Bottom:** Two 48-in.W x 30-in.H (121.92cmW x 76.20cmH) DD cages

P/N	Description
300000175	Regal Cage Arrangement
3000003010	Full-View Polycarbonate Back Regal Cage Arrangement
3000004009	Full-View Tempered Glass Back Regal Cage Arrangement
3000005009	Front & Back Door Regal Cage Arrangement
3000006051	Heated Regal Cage Arrangement, 120V AC ▲ *

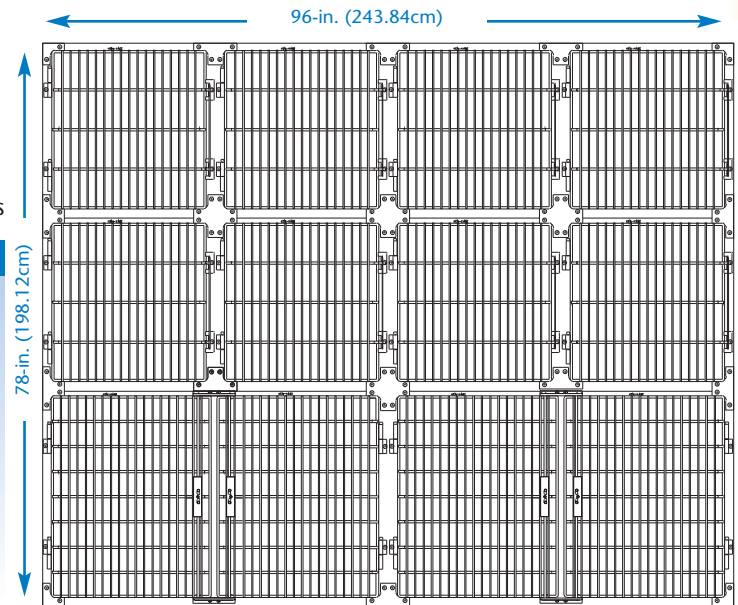


8-Foot (2.44m) Wide, 10-Cage Regal Cage Arrangement

Includes:

- Top:** Four 24-in.W x 30-in.H (60.96cmW x 76.20cmH) cages
- Middle:** Four 24-in.W x 30-in.H (60.96cmW x 76.20cmH) cages
- Bottom:** Two 48-in.W x 30-in.H (121.92cmW x 76.20cmH) DD cages

P/N	Description
300000176	Regal Cage Arrangement
3000003011	Full-View Polycarbonate Back Regal Cage Arrangement
3000004010	Full-View Tempered Glass Back Regal Cage Arrangement
3000005010	Front & Back Door Regal Cage Arrangement
3000006052	Heated Regal Cage Arrangement, 120V AC ▲ *

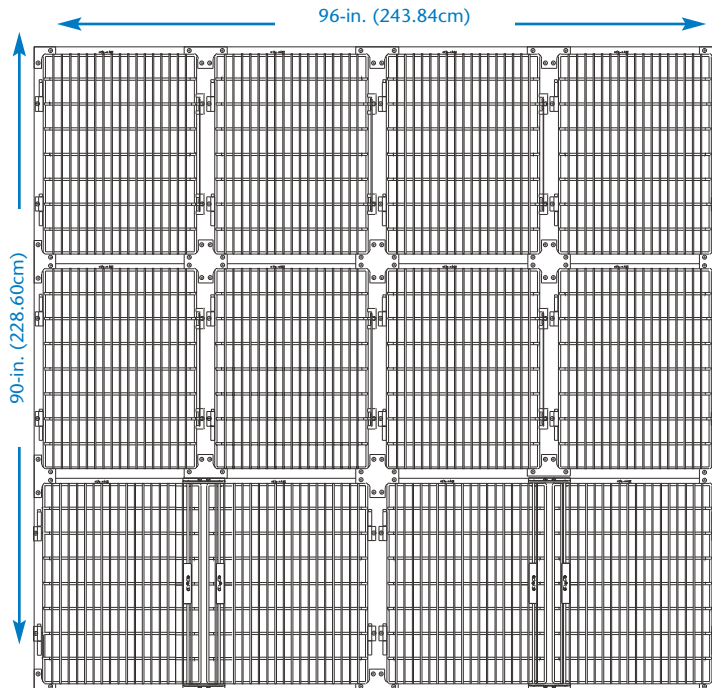

*Arrangements shipped pre-assembled. ▲ 220V requires step-down transformer. Contact SSCI.


Regal Cage Arrangements

10-Foot (3.05m) Wide, 12-Cage Regal Cage Arrangement

Includes:

- Top:** Five 24-in.W x 24-in.H (60.96cmW x 60.96cmH) cages
- Middle:** Four 30-in.W x 24-in.H (76.20cmW x 60.96cmH) cages
- Bottom:** One 48-in.W x 30-in.H (121.92cmW x 76.20cmH) DD cage
Two 36-in.W x 30-in.H (91.44cmW x 76.20cmH) cages

P/N	Description
300000183	Regal Cage Arrangement
3000003014	Full-View Polycarbonate Back Regal Cage Arrangement
3000004013	Full-View Tempered Glass Back Regal Cage Arrangement
3000005013	Front & Back Door Regal Cage Arrangement
3000006053	Heated Regal Cage Arrangement, 120V AC ▲*

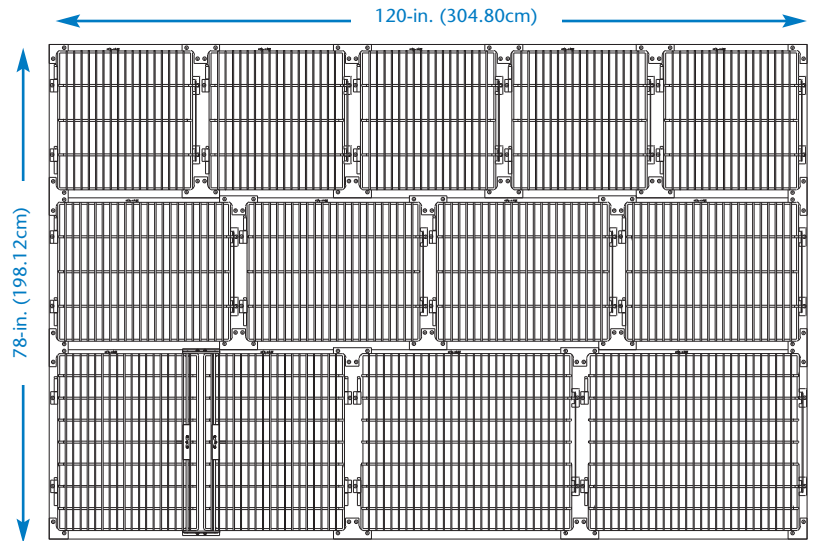


10-Foot (3.05m) Wide, 12-Cage Regal Cage Arrangement

Includes:

- Top:** Five 24-in.W x 24-in.H (60.96cmW x 60.96cmH) cages
- Middle:** Four 30-in.W x 30-in.H (76.20cmW x 76.20cmH) cages
- Bottom:** One 48-in.W x 30-in.H (121.92cmW x 76.20cmH) DD cage
Two 36-in.W x 30-in.H (91.44cmW x 76.20cmH) cages

P/N	Description
300000180	Regal Cage Arrangement
3000003012	Full-View Polycarbonate Back Regal Cage Arrangement
3000004011	Full-View Tempered Glass Back Regal Cage Arrangement
3000005011	Front & Back Door Regal Cage Arrangement
3000006054	Heated Regal Cage Arrangement, 120V AC ▲*

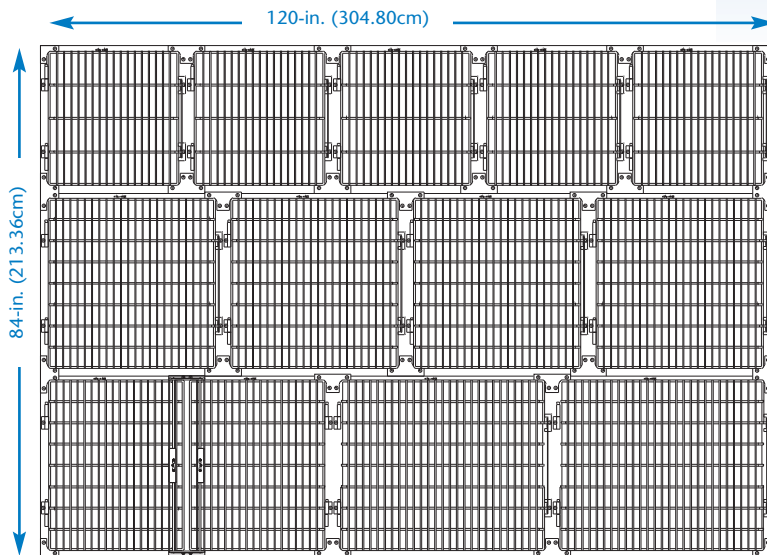

* Arrangements shipped pre-assembled. ▲ 220V requires step-down transformer. Contact SSCI.


Regal Cage Arrangements

12-Foot (3.66m) Wide, 14-Cage Regal Cage Arrangement

Includes:

- Top:** Six 24-in.W x 24-in.H (60.96cmW x 60.96cmH) cages
- Middle:** Four 30-in.W x 30-in.H (76.20cmW x 76.20cmH) cages
One 24-in.W x 30-in.H (60.96cmW x 76.20cmH) cages
- Bottom:** Three 48-in.W x 30-in.H (121.92cmW x 76.20cmH) DD cage

P/N	Description
300000171	Regal Cage Arrangement
3000003013	Full-View Polycarbonate Back Regal Cage Arrangement
3000004012	Full-View Tempered Glass Back Regal Cage Arrangement
3000005012	Front & Back Door Regal Cage Arrangement
3000006055	Heated Regal Cage Arrangement, 120V AC ▲ *

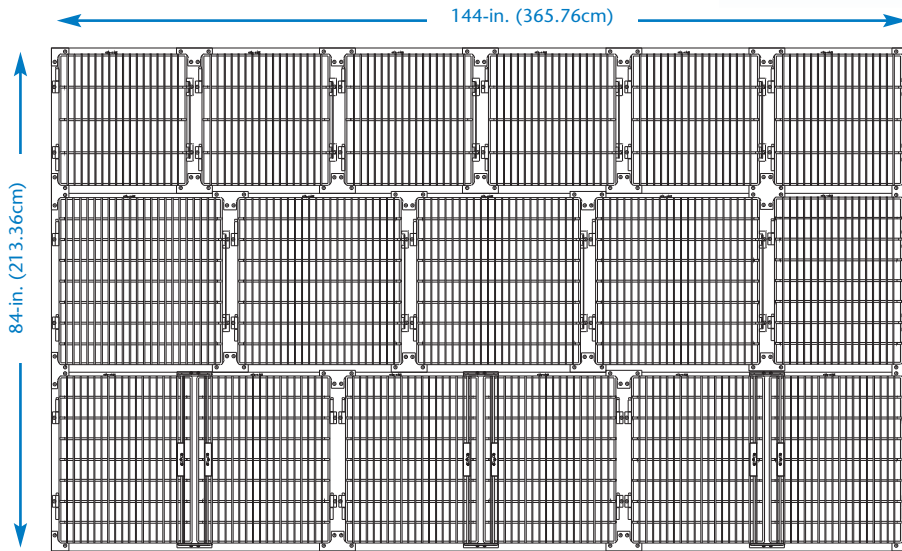


14-Foot (4.27m) Wide, 17-Cage Regal Cage Arrangement

Includes:

- Top:** Seven 24-in.W x 24-in.H (60.96cmW x 60.96cmH) cages
- Middle:** Four 30-in.W x 30-in.H (76.20cmW x 76.20cmH) cages
Two 24-in.W x 30-in.H (60.96cmW x 76.20cmH) cages
- Bottom:** Two 36-in.W x 30-in.H (91.44cmW x 76.20cmH) cages
Two 48-in.W x 30-in.H (121.92cmW x 76.20cmH) DD cages

P/N	Description
300000181	Regal Cage Arrangement
3000003015	Full-View Polycarbonate Back Regal Cage Arrangement
3000004014	Full-View Tempered Glass Back Regal Cage Arrangement
3000005014	Front & Back Door Regal Cage Arrangement
3000006056	Heated Regal Cage Arrangement, 120V AC ▲ *

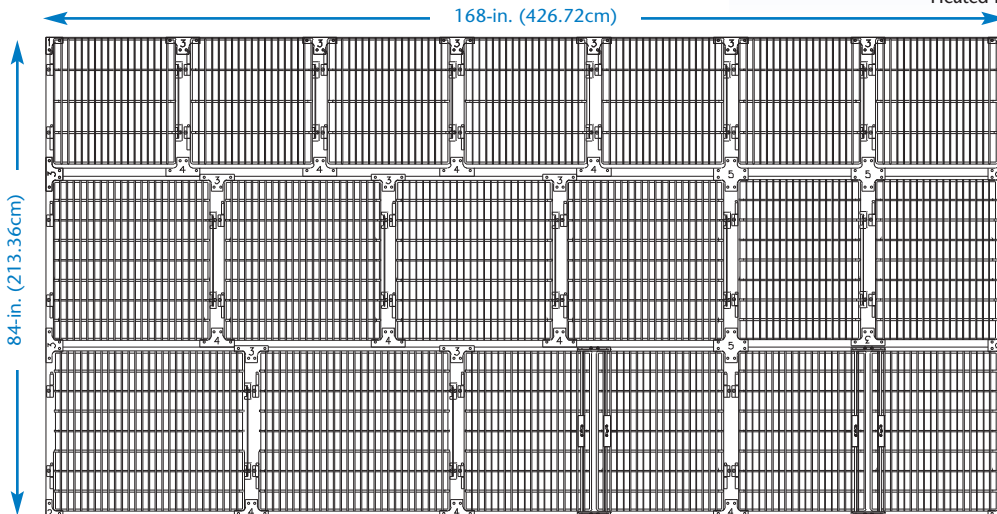

* Arrangements shipped pre-assembled. ▲ 220V requires step-down transformer. Contact SSCI.

Installation Hardware For Regal Cage Arrangements

QUALITY STAINLESS STEEL HARDWARE IS INCLUDED WITH YOUR NEW CAGE ARRANGEMENT.

Each of our Cage Arrangements is shipped complete with our custom-matched stainless steel installation hardware made of heavy-gauge stainless steel. Corner Plates install easily to attach cages together. Vertical Molding Strips cover the space between two cages to keep liquids and debris from seeping into hard-to-clean areas.


Each Cage Arrangement drawing, which you reviewed and approved prior to purchasing the cages, shows the exact placement of every cage and each piece of hardware. If you decide to reposition the cages when setting up your new cage arrangements, this modified arrangement may require different pieces of hardware. SSCI is available to help you determine what additional hardware is required.

Call SSCI for availability and pricing of the additional hardware to setup your modified cage arrangement.

Using your simple hand tools, just follow the Installation Guide and Cage Arrangement Approval Drawing(s) to assemble your new Cage Arrangement.



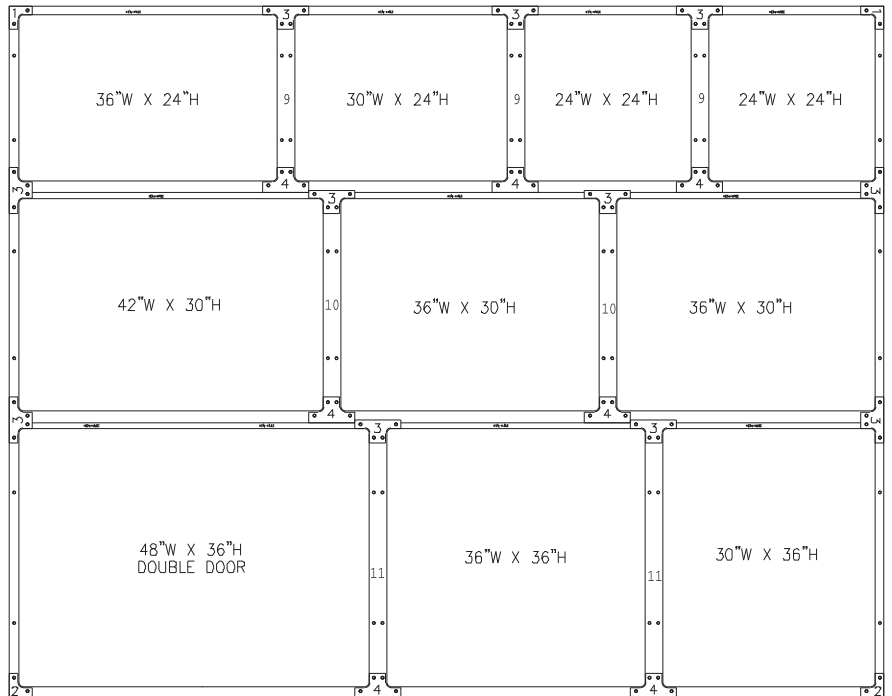


HARDWARE IS AVAILABLE FOR PURCHASE TO REARRANGE EXISTING CAGE ARRANGEMENTS.




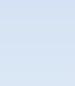



All hardware can also be purchased separately to enable you to reposition cages in existing arrangements, if additional hardware is required. So, if you need to reconfigure the arrangement you already have, SSCI can help you determine what additional hardware is required.



Typical Cage Arrangement Drawing

Numbers next to photos of hardware below refer to numbers on this drawing above.

P/N	Description	
12100-CT-AAAAAA	Corner Top Plate #1	 1
12100-CB-AAAAAA	Corner Bottom Plate #2	 2
12100-TS-AAAAAA	Top/Side T-Plate #3	 3
12100-BT-AAAAAA	Bottom T-Plate #4	 4
12100-IC-AAAAAA	Cross Plate #5	 5
008545	Double-Faced Tape 10-ft. (3.05m) Rolls	

P/N	Description	
850157	Cap Screws	
12106-00-AAAAAA	Anchoring Tab Bracket for Securing Cages to Non-SSCI Bases	
12102-00-AAAAAA	Securing Clip	 11
12104-00-AACTAA	Vertical Molding Strip 18-in. high (45.72cm)	 10
12104-00-AADRAA	Vertical Molding Strip 24-in. high (60.96cm)	 9
12104-00-AAEPAA	Vertical Molding Strip 30-in. high (76.20cm)	 8
12104-00-AAFNAA	Vertical Molding Strip 36-in. high (91.44cm)	

Finishing Accessories For Regal Cage

Finishing Accessories and Cage Accessories, give your Cage Arrangement a finished appearance and greater versatility. Regal Cage Finishing Accessories will only fit SSCI Regal Cages. SSCI Cage Accessories fit all types of SSCI Cages.




For Product Information Call TOLL-FREE 1-800-323-7366 • (847) 537-9320 • Fax (847) 537-9061
Hours: 8:30am - 5:00pm CST Monday - Friday
Visit our website at: www.suburbansurgical.com

Finishing Accessories For Regal Cage

Removable Divider Panel for Double Door Regal Cages

Use this rugged stainless steel Divider to turn one large cage into two smaller ones. Panel slides into Channel included with all Double Door Regal Cages, and the front pin fits into top and bottom cage-door latches to hold panel securely in place. Panel removes quickly and easily when not required.

P/N	FITS CAGE SIZE	
	Inches	Centimeters
12110-00-AADREI	24H	60.96H
12110-00-AAEPEI	30H	76.20H
12110-00-AAFNEI	36H	91.44H

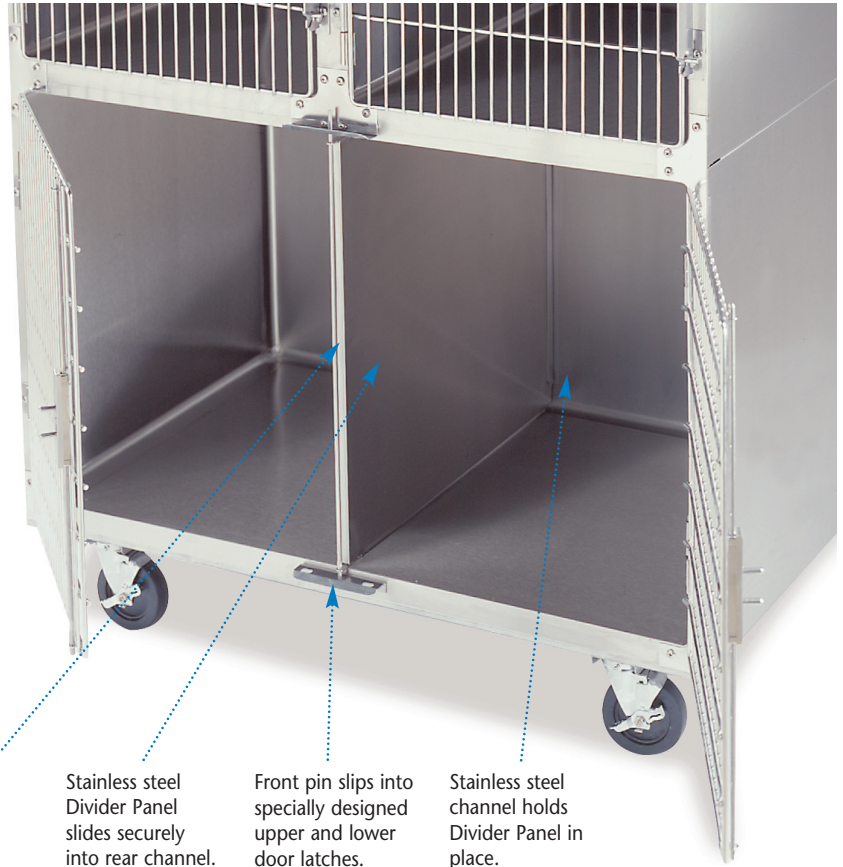

When the Divider Panel is removed, there's no center post to get in the way when placing animals in the cage.

Stainless steel Divider Panel slides securely into rear channel.

Front pin slips into specially designed upper and lower door latches.

Stainless steel channel holds Divider Panel in place.

Stainless Steel Resting Shelves

Make your cages more versatile with our stainless steel Resting Shelves. Shelf can be used to make room for multiple animals in one cage, or simply to make cats more comfortable by allowing them to climb and rest higher than the floor. Shelves, factory-installed, are available when ordering new cages, or shelves may be ordered separately for customer installation in new or existing cages. Shelves are mechanically-fastened thru the sides of the cage for stability. This prevents the shelf from ever tipping out of position. Existing arrangements must be disassembled to install shelves. Two types of shelves are available: straight and corner. Straight shelves are positioned 1.250-in. (3.17cm) from the back spanning the width of the cage, corner shelves are positioned against the back and side of the cage, typically, the shelves are installed approximately midway between the floor and the ceiling of the cage. See chart associated with Regal Cage type for cage P/N.



Shelf is fastened thru sides for stability

P/N	FITS CAGE SIZE		SHELF SIZE	
	Inches	Centimeters	Inches	Centimeters
12165-00-CTACAX	18W	45.72W	15.75Wx6D	40.01Wx15.24D
12165-00-DRACAX	24W	60.96W	21.75Wx6D	55.24Wx15.24D
12165-00-EPACAX	30W	76.20W	27.75Wx6D	70.48Wx15.24D
12165-00-FNACAX	36W	91.44W	33.75Wx6D	85.72Wx15.24D
12170-00-BVACBV	Corner Shelf fits any cage			

Finishing Accessories For Regal Cage

Give your Regal Cage Arrangement a finished look with our stainless steel Top and End Panels. These Panels cover the spaces between cages where dust and dirt can hide. Made of stainless steel for long life.

A Top Panels

Order the Top Panel or combination of Top Panels you need to span the entire width of your Cage Arrangement.

For example: for a 78-in. (198.12cm) wide Arrangement, you need to order two 24-in. (60.96cm) wide Panels and one 36-in. (91.44cm) wide Panel.

See Top Panel Chart for Combination Examples.

P/N	WIDTH	
	In.	Cm
12140-00-CTAAEI	18W	45.72W
12140-00-DRAAEI	24W	60.96W
12140-00-EPAAEI	30W	76.20W
12140-00-FNAAEI	36W	91.44W

Combinations of the above panels will accommodate all widths of cage arrangements.

CHART FOR TOP PANEL COMBINATIONS

Cage Arrangements	Panel 1	Panel 2	Panel 3	Panel 4
18" Wide	18			
24" Wide	24			
30" Wide	30			
36" Wide	18	18		
42" Wide	24	18		
48" Wide	24	24		
54" Wide	24	30		
60" Wide	30	30		
66" Wide	30	36		
72" Wide	36	36		
78" Wide	24	30	24	
84" Wide	24	36	24	
90" Wide	30	30	30	
96" Wide	30	36	30	
102" Wide	36	30	36	
108" Wide	36	36	36	
114" Wide	36	30	24	24
120" Wide	30	30	30	30

B End Panels

Select the appropriate height End Panel for exposed ends of each tier of cages for Standard, Heated, Front & Back Door, or Full View Cages. Order the appropriate End Panel Uprights below.

P/N	HEIGHT	
	In.	Cm
12120-00-AACTEI	18H	45.72H
12120-00-AADREI	24H	60.96H

P/N	HEIGHT	
	In.	Cm
12120-00-AAEPEI	30H	76.20H
12120-00-AAFNEI	36H	91.44H

C End Panel Uprights

Uprights hold the back edge of End Panels and attach to each tier of cages with the End Panel Bracket Assembly. Order the Upright height equal to the combined height of the Cages in your Arrangement.


P/N FOR USE WITH STANDARD AND HEATED CAGES	P/N FOR USE WITH FRONT & BACK DOOR AND FULL-VIEW BACK CAGES	HEIGHT	
		In.	Cm
12125-00-AACTAA	12126-00-AACTAA	18H	45.72H
12125-00-AADRAA	12126-00-AADRAA	24H	60.96H
12125-00-AAEPAA	12126-00-AAEPAA	30H	76.20H
12125-00-AAFNAA	12126-00-AAFNAA	36H	91.44H
12125-00-AAGLAA	12126-00-AAGLAA	42H	106.68H
12125-00-AAHJAA	12126-00-AAHJAA	48H	121.92H
12125-00-AAIHAA	12126-00-AAIHAA	54H	137.16H

P/N FOR USE WITH STANDARD AND HEATED CAGES	P/N FOR USE WITH FRONT & BACK DOOR AND FULL-VIEW BACK CAGES	HEIGHT	
		In.	Cm
12125-00-AAJFAA	12126-00-AAJFAA	60H	152.40H
12125-00-AAKDAA	12126-00-AAKDAA	66H	167.64H
12125-00-AALBAA	12126-00-AALBAA	72H	182.88H
12125-00-AALZAA	12126-00-AALZAA	78H	198.12H
12125-00-AAMXAA	12126-00-AAMXAA	84H	213.36H
12125-00-AANVAA	12126-00-AANVAA	90H	228.60H
12125-00-AAOTAA	12126-00-AAOTAA	96H	243.84H

D End Panel Bracket

For attaching End Panel Upright to Regal Standard and Heated Cage Arrangements (order one bracket for each end panel ordered). Not necessary for other cage arrangements.

P/N
12130-00-AAAAA

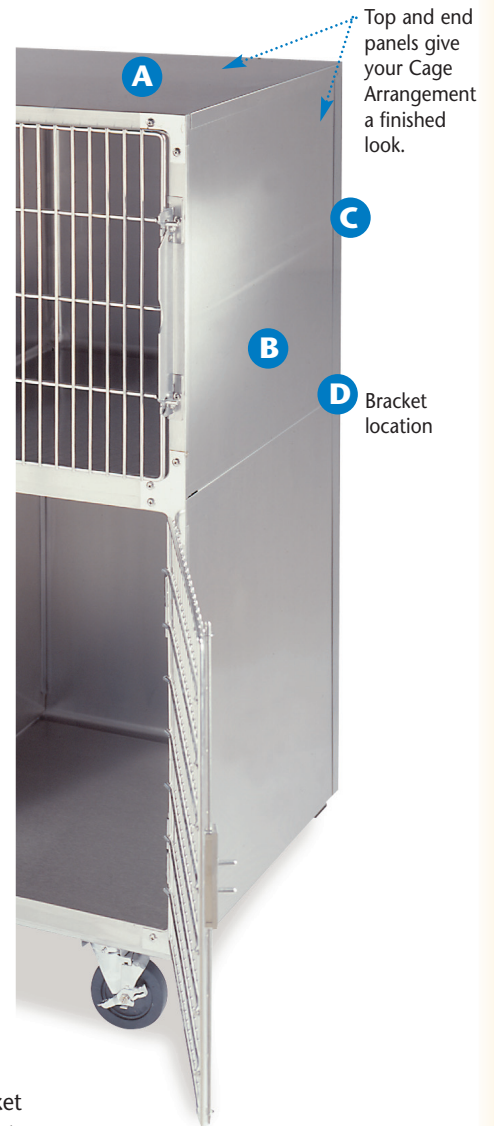


Finishing Accessories For Regal Cage

Stainless Steel Stationary Bases

Stainless Steel Bases give your Cage Arrangement stability. Leg levelers ensure your Arrangement can be level even if your floor is slightly uneven.

Base adds just 1.5-in. (3.81cm) to Cage Arrangement's height .

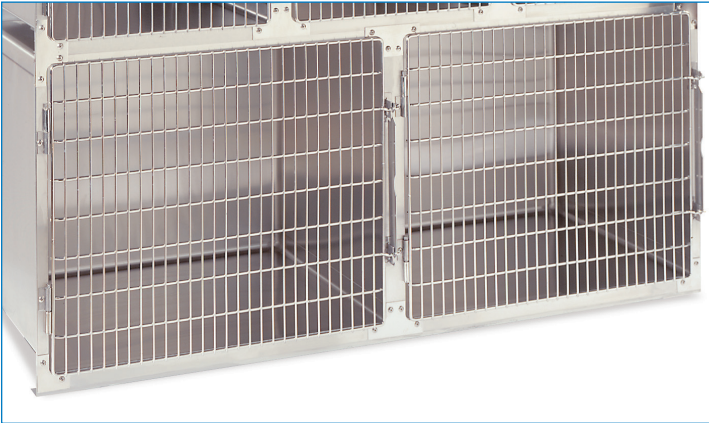

For Standard and Heated Cages	For Full-View Back and Front & Back Door Cages	ARRANGEMENT WIDTH	
		Feet	Meters
12115-00-CTAHEI	12116-00-CTAHEI	1'6"W	.46W
12115-00-DRAHEI	12116-00-DRAHEI	2'W	.61W
12115-00-EPAHEI	12116-00-EPAHEI	2'6"W	.76W
12115-00-FNAHEI	12116-00-FNAHEI	3'W	.91W
12115-00-GLAHEI	12116-00-GLAHEI	3'6"W	1.07W
12115-00-HJAHEI	12116-00-HJAHEI	4'W	1.22W
12115-00-IHAHEI	12116-00-IHAHEI	4'6"W	1.37W
12115-00-JFAHEI	12116-00-JFAHEI	5'W	1.52W
12115-00-KDAHEI	12116-00-KDAHEI	5'6"W	1.68W
12115-00-LBAHEI	12116-00-LBAHEI	6'W	1.83W
12115-00-LZAHEI	12116-00-LZAHEI	6'6"W	1.98W
12115-00-MXAHEI	12116-00-MXAHEI	7'W	2.13W
12115-00-NVAHEI	12116-00-NVAHEI	7'6"W	2.29W
12115-00-OTAHEI	12116-00-OTAHEI	8'W	2.44W
12115-00-PRAHEI	12116-00-PRAHEI	8'6"W	2.59W
12115-00-QPAHEI	12116-00-QPAHEI	9'W	2.74W
12115-00-RNAHEI	12116-00-RNAHEI	9'6"W	2.90W
12115-00-SLAHEI	12116-00-SLAHEI	10'W	3.05W

Stainless Steel Mobile Bases

Stainless Steel Mobile Base give your Cage Arrangement mobility. Base adds 6.75-in. (17.15cm) to Cage Arrangement's height. Swivel casters with 5-in. (12.70cm) diameter wheels allow for easy movement throughout your facility.

Bases up to 6-ft. (1.83m) long have four casters, two with wheel brakes. Bases 6-ft. 6-in. (1.98m) long or longer have six casters, three with wheel brakes.



For Standard and Heated Cages	For Full-View Back and Front & Back Door Cages	ARRANGEMENT WIDTH	
		Feet	Meters
12115-50-CTAHEI	12116-50-CTAHEI	1'6"W	.46W
12115-50-DRAHEI	12116-50-DRAHEI	2'W	.61W
12115-50-EPAHEI	12116-50-EPAHEI	2'6"W	.76W
12115-50-FNAHEI	12116-50-FNAHEI	3'W	.91W
12115-50-GLAHEI	12116-50-GLAHEI	3'6"W	1.07W
12115-50-HJAHEI	12116-50-HJAHEI	4'W	1.22W
12115-50-IHAHEI	12116-50-IHAHEI	4'6"W	1.37W
12115-50-JFAHEI	12116-50-JFAHEI	5'W	1.52W
12115-50-KDAHEI	12116-50-KDAHEI	5'6"W	1.68W
12115-50-LBAHEI	12116-50-LBAHEI	6'W	1.83W
12115-50-LZAHEI	12116-50-LZAHEI	6'6"W	1.98W
12115-50-MXAHEI	12116-50-MXAHEI	7'W	2.13W
12115-50-NVAHEI	12116-50-NVAHEI	7'6"W	2.29W
12115-50-OTAHEI	12116-50-OTAHEI	8'W	2.44W
12115-50-PRAHEI	12116-50-PRAHEI	8'6"W	2.59W
12115-50-QPAHEI	12116-50-QPAHEI	9'W	2.74W
12115-50-RNAHEI	12116-50-RNAHEI	9'6"W	2.90W
12115-50-SLAHEI	12116-50-SLAHEI	10'W	3.05W

Cage Handle for Mobile Arrangements

Stainless Steel Handle makes it easy to move your Mobile Regal Cage Arrangement wherever you need it within your facility. Fully welded stainless steel Handle replaces the Side T plate in your Arrangement. Simply remove the Side T plate, and attach handle in its place.



P/N	Description
12108-00-AAAAA	Cage Handle for Mobile Arrangements

Finishing Accessories For Regal Cage

STRAPPING REQUIREMENT CHART

Cage Size Inches	Strapping Required	Cage Size Centimeters	Strapping Required
18W x 18H	12'	45.72W x 45.72H	3.66m
24W x 18H	14'	60.96W x 45.72H	4.27m
18W x 24H	14'	45.72W x 60.96H	4.27m
24W x 24H	16'	60.96W x 60.96H	4.88m
30W x 24H	18'	76.20W x 60.96H	5.49m
36W x 24H	20'	91.44W x 60.96H	6.10m
48W x 24H DD	24'	121.92W x 60.96H DD	7.32m
24W x 30H	18'	60.96W x 76.20H	5.49m
30W x 30H	20'	76.20W x 76.20H	6.10m
36W x 30H	22'	91.44W x 76.20H	6.71m
42W x 30H	24'	106.68W x 76.20H	7.32m
48W x 30H	26'	121.92W x 76.20H	7.92m
48W x 30H DD	26'	121.92W x 76.20H DD	7.92m
24W x 36H	20'	60.96W x 91.44H	6.10m
30W x 36H	22'	76.20W x 91.44H	6.71m
36W x 36H	24'	91.44W x 91.44H	7.32m
42W x 36H	26'	106.68W x 91.44H	7.92m
48W x 36H	28'	121.92W x 91.44H	8.53m
48W x 36H DD	28'	121.92W x 91.44H DD	8.53m
60W x 36H DD	32'	152.40W x 91.44H DD	9.75m
72W x 36H DD	36'	182.88W x 91.44H DD	10.97m

Noise Reduction Strapping

This specially designed vinyl Strapping wraps around Regal Cages to reduce vibrations and kennel noise. Adhesive backing allows you to easily apply strapping to cages before assembling your cage arrangement.


P/N	Feet	Meters
12135-00-CHAAAA	15'L Strip	4.57m
12135-00-PJAAAA	100'L Coil	30.48m



Squeeze Back Panel

Stainless steel with electropolished finish Squeeze Back Panel slides forward to bring frightened or hard-to-handle animals to the front of the cage. Side rails extend out through the cage door to allow you to slide panel and animal forward without opening the door. Curved sides allow panel to fit snugly into back of a Regal Cage until needed.

For Doors Hinged Left	For Doors Hinged Right	FITS CAGE SIZE	
		Inches	Centimeters
12175-00-CTCEI	12176-00-CTCEI	18W x 18H	45.72W x 45.72H
12175-00-CTDREI	12176-00-CTDREI	24W x 18H	60.96W x 45.72H
12175-00-DRCTEI	12176-00-DRCTEI	18W x 24H	45.72W x 60.96H
12175-00-DRDREI	12176-00-DRDREI	24W x 24H	60.96W x 60.96H
12175-00-DREPEI	12176-00-DREPEI	30W x 24H	76.20W x 60.96H
12175-00-DRFNEI	12176-00-DRFNEI	36W x 24H	91.44W x 60.96H
12175-00-EPDREI	12176-00-EPDREI	24W x 30H	60.96W x 76.20H
12175-00-EPEPEI	12176-00-EPEPEI	30W x 30H	76.20W x 76.20H
12175-00-EPFNEI	12176-00-EPFNEI	36W x 30H	91.44W x 76.20H
12175-00-FNDREI	12176-00-FNDREI	24W x 36H	60.96W x 91.44H
12175-00-FNEPEI	12176-00-FNEPEI	30W x 36H	76.20W x 91.44H
12175-00-FNFNEI	12176-00-FNFNEI	36W x 36H	91.44W x 91.44H

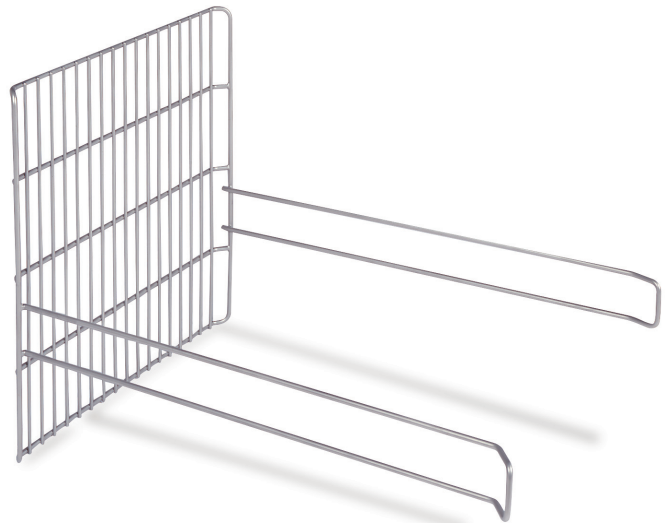


Finishing Accessories For Regal Cage

Thermometer/hygrometer monitors temperature and humidity.

Melted ice used for cooling oxygen is drained from compartment through convenient valve.

Clear polycarbonate panel resists scratches and discoloration.

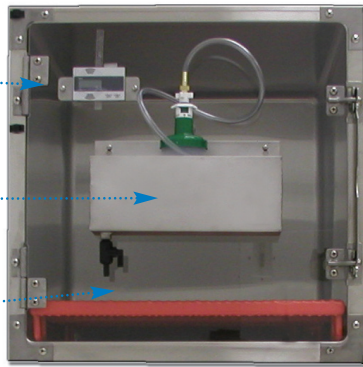

Therapy Doors

Transforms Regal Cage into a respiratory therapy cage. Allows for the following therapies:

- 1) Oxygen
- 2) Oxygen with moisture
- 3) Oxygen with moisture and medication
- 4) Compressed air with moisture
- 5) Compressed air with moisture and medication

Clear door allows for patient observation and features rubber gasket seal which is open at the corners to create a positive atmosphere, yet allows for venting. A continuous-flow nebulizer mounted on door delivers a regulated flow of the preferred therapy. Ice compartment mounted on door holds ice to cool therapy liquid, if needed. Thermometer/ hygrometer, mounted on door interior, indicates temperature and humidity of cage. Door mounts hinged left onto front of cage in place of the regular door; uses regular hinges and latches to close securely with .25-in. (.64cm) inner diameter x .062-in. (.16cm) thick tubing. Only available for single-door cages.

P/N	FITS CAGE SIZE	
	Inches	Centimeters
12156-00-DRDRAA	24W x 24H	60.96W x 60.96H
12156-00-DREPAA	24W x 30H	60.96W x 76.20H
12156-00-EPEPAA	30W x 30H	76.20W x 76.20H
12156-00-EPDRAA	30W x 24H	76.20W x 60.96H
12156-00-FNEPAA	36W x 30H	91.44W x 76.20H
12156-00-HJEPAA	48W x 30H	121.92W x 76.20H
Additional Parts		
212126	Nebulizers with Supply Hose	

Animal Transfer Door

An ideal solution for hard to handle animals. Limited access door limits the opportunity for your staff to be injured by angry strays or feral animals. Door mounts onto front of cage, hinged left and latches just like our regular doors, yet also features a guillotine door that opens upward for greater animal control. Bowl holder rotates out to make changing food and water safer for your staff and the animal. Latches securely in both in and out positions. Two stainless steel bowls are included.

P/N	Description
12160-00-DRDRAA	Fits 24W x 24H (60.96cm W x 60.96cm H) Regal Cage
12160-00-DRDRAA	Fits 30W x 24H (76.20cm W x 60.96cm H) Regal Cage
Additional Parts	
12308-00-A5AAAA	1.5-quart Bowl with bowl-holder tab



Short Hinge Pin IV Hook

Stainless steel Hinge Pin IV hooks replace the Regal Cage hinge pin to provide secure positioning of an IV bag. Shorter height may be placed on lower tiers without interfering with the doors of cages above or for facilities with limited ceiling height.

P/N	Description
12395-00-AAAAAA	Short Hinge Pin IV Hook



Tall Hinge Pin IV Hook


Stainless steel Hinge Pin IV hooks replace the Regal Cage hinge pin to provide secure positioning of an IV bag. Taller height is designed for use on top tiers of cages, or on lower tiers wherever cages are vertically aligned, allowing upper cage door to open. Reduces chance of animal interference.

P/N	Description
12396-00-AAAAAA	Tall Hinge Pin IV Hook

Clamp-on Adjustable Height IV Hook

Stainless Steel Clamp on IV Hook attaches securely to vertical spokes in cage door. Height adjusts total amount shown below.


P/N	Inches	Centimeters
12397-00-AACXAA	19H	48.26H
12397-00-AAFJAA	35H	88.90H

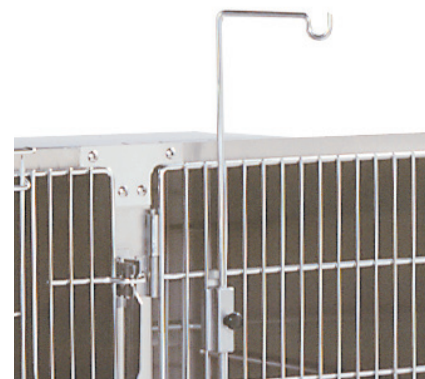

Cage Accessories

Card Holders

Use to display animal's name, weight, medication needs, release date, or other pertinent information. Attaches securely to the cage door to hold identification cards on three sides. Made of durable, easy to sanitize stainless steel. Cards slide into holder from top.

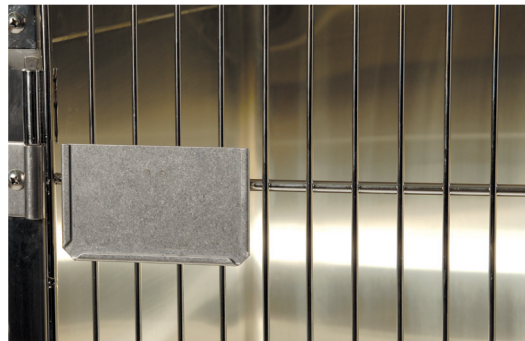

P/N	DIMENSIONS	
	Inches	Centimeters
12300-00-ATALAA	5W x 3H	12.70W x 7.62H
12300-00-AXAPAA	6W x 4H	15.24W x 10.16H
12300-00-BFATAA	8W x 5H	20.32W x 12.70H



Clipboards

Holds information about the animal in your care. Yet lifts off quickly and easily to use as a writing surface to update the chart. Clip holds chart securely to keep all pertinent information at hand and easily accessible. Made of durable, easy to sanitize stainless steel.

P/N	DIMENSIONS	
	Inches	Centimeters
12302-00-BJAXAA	9W x 6H	22.86W x 15.24H
12302-00-BJBVAA	9W x 12H	22.86W x 30.48H

Clipboard with Bin

Bin/clipboard combination lifts on and off easily, lets you keep medicines, special foods or patient's personal belongings, leash, collar, etc. and dispensing schedule right on patient's cage. Helps to remind you when the next dose is due. Made of durable, easy to sanitize stainless steel.

P/N	BIN HEIGHT		OVERALL DIMENSIONS	
	In.	Cm	In.	Cm
12304-00-AXBJAL	4.188H	10.64H	6W x 9H x 3D	15.24W x 22.86H x 7.62D
12304-00-BJBVAL	5.188H	13.18H	9W x 12H x 3D	22.86W x 30.48H x 7.62D



Food and Water Bowls

Stainless steel bowls are easy to clean and sanitize to ensure animal health.


P/N	CAPACITY	
	U.S. Liquid	Metric
12305-00-A2AAAA	1 Pint	0.47 Liters
12305-00-A4AAAA	1 Quart	0.95 Liters
12305-00-A6AAAA	2 Quarts	1.89 Liters

Food and Water Bowl Holders

Attach securely to cage door to hold stainless steel bowls sold above. Designed to keep bowl off the floor and reduce spills. Made of stainless steel with electropolished finish.

P/N	FITS BOWL SIZE	
	U.S. Liquid	Metric
12335-00-A2AAAA	1 Pint	0.47 Liters
12335-00-A4AAAA	1 Quart	0.95 Liters
12335-00-A6AAAA	2 Quarts	1.89 Liters



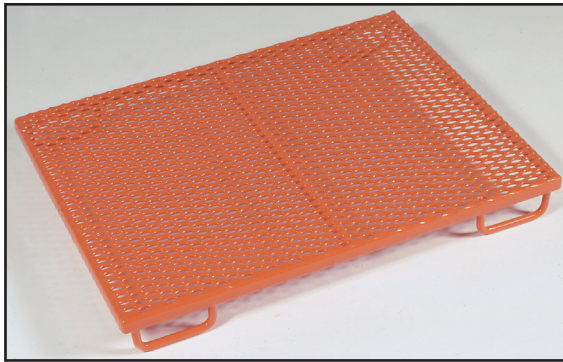

Plastisol-Coated Removable Cage Floors


Orange plastisol-coated expanded metal Cage Floor. Allows animals to rest 3-in. (7.62cm) above the cage bottom to stay clean, dry and comfortable. For double-door cages, order 2 Cage Floors of the appropriate size.

P/N	FITS CAGES (W)		FLOOR SIZE (WxHxD)	
	In.	Cm	Inches	Centimeters
12385-00-CTALEI	18W	45.72W	15.438W x 3H x 28D	38.98W x 7.62H x 71.12D
12385-00-DRALEI	24W	60.96W	21.438W x 3H x 28D	54.45W x 7.62H x 71.12D
12385-00-EPALEI	30W	76.20W	27.438W x 3H x 28D	69.69W x 7.62H x 71.12D
12385-00-FNALEI	36W	91.44W	33.438W x 3H x 28D	84.93W x 7.62H x 71.12D

Stainless Steel Removable Cage Floors

Round rod stainless steel Cage Floor is welded at all intersections for superior strength and durability. Rods are deburred and spaced .750-in. (1.90cm) apart and have electropolished finish. Allows animals to rest 1.5-in. (3.81cm) above the cage bottom to stay clean, dry and comfortable. For double-door cages, order 2 Cage Floors of the appropriate size.

P/N	FITS CAGES (W)		FLOOR SIZE (WxHxD)	
	In.	Cm	Inches	Centimeters
12375-00-CTAFEI	18W	45.72W	15.188W x 1.412H x 27.75D	38.58W x 3.59H x 70.49D
12375-00-DRAFEI	24W	60.96W	21.188W x 1.412H x 27.75D	53.82W x 3.59H x 70.49D
12375-00-EPAFEI	30W	76.20W	27.188W x 1.412H x 27.75D	69.06W x 3.59H x 70.49D
12375-00-FNAFEI	36W	91.44W	33.188W x 1.412H x 27.75D	84.30W x 3.59H x 70.49D

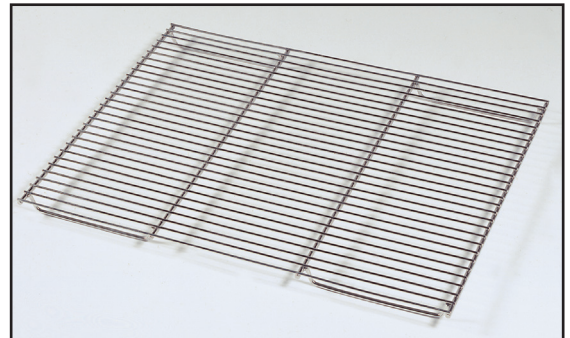


Tapered Litter Pan

Our stainless steel Litter Pan is just the right size to fit neatly into your cage. Features tapered sides for nesting. Easily cleaned and disinfected.

P/N	DESCRIPTION	MEASUREMENTS (WxHxD)	
		Inches	Centimeters
12350-00-BZAPBJ	Tapered Litter Pan	13W x 4H x 9D	33.02W x 10.16H x 22.86D

Regal Transport Cages

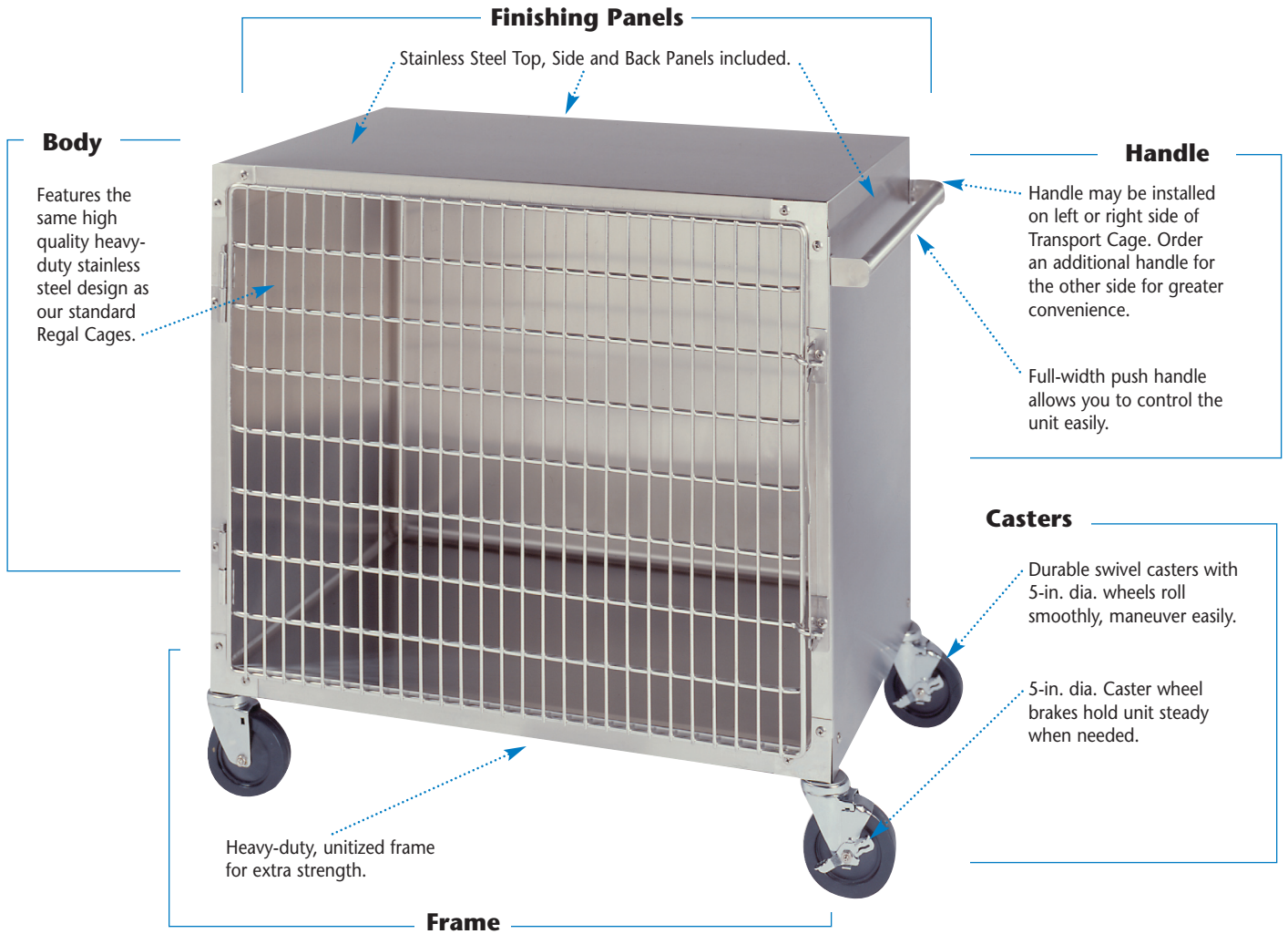

Regal Transport Cage

Versatile Transport Regal Cages can be used to move animals from their cages to surgery, transfer boarders from the waiting room to the kennel, temporarily hold patients before treatment.

These solid stainless steel cages start with the same rugged construction as our standard Regal Cages. Then we add a stainless steel frame with full-width rails and crossmembers for rigidity. Casters have threaded stem that thread into the frame, securing them so they will not wobble or fall out. Swivel bearing and axle have grease fittings. 5-in. (12.70cm) diameter wheels allow for easy movement

throughout your facility.

Two casters have wheel brakes for safety. Stainless steel Top, Ends and Back Finishing Panels give the cage a finished appearance. Solid stainless steel top can be used as a mobile work surface if desired.

Door is hinged left, but may be easily reversed to hinge on the right if desired with no additional hardware. A full-width stainless steel push handle is included for one end.

For handles on both ends, order additional handle below.

P/N	CAGE SIZE		OVERALL SIZE	
	In.	Cm	Inches	Centimeters
12014-00-GCFPDY	36W x 30H	91.44W x 76.20H	39.75W x 36.50H x 25.75D	100.97W x 92.71H x 65.41D
12014-00-HYFPDY	48W x 30H	121.92W x 76.20H	51.75W x 36.50H x 25.75D	131.45W x 92.71H x 65.41D
Additional Parts				
201483	Push Handle			



Regal Transport Cage with Therapy Door


Our Regal Transport Cage with Therapy Door is the ideal way to move patients to area for continuous observation. It's designed with all the features of our standard Regal Transport Cage with a clear polycarbonate door. See Therapy Door description. A Stainless Steel Regal Cage Door can be purchased separately for more versatility.

P/N	CAGE SIZE		OVERALL SIZE	
	In.	Cm	Inches	Centimeters
12016-00-GCFPEH	36W x 30H	91.44W x 76.20H	39.75W x 36.50H x 25.75D	100.97W x 92.71H x 73.03D
12016-00-HYFPEH	48W x 30H	121.92W x 76.20H	51.75W x 36.50H x 25.75D	131.45W x 92.71H x 73.03D

Additional Parts

201483	Push Handle
209798	Stainless Steel Regal Cage Door 36W x 30H
209800	Stainless Steel Regal Cage Door 48W x 30H


Regal Transport Cage Finishing Accessories

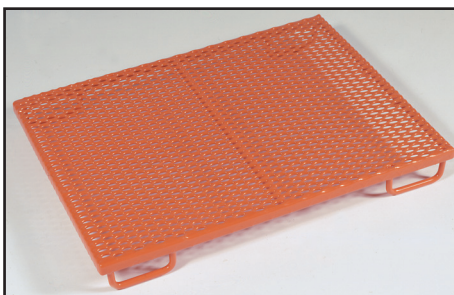

Plastisol-Coated Removable Cage Floors

Orange plastisol-coated expanded metal Cage Floor. Allows animals to rest 3-in. (7.62cm) above the cage bottom to stay clean, dry and comfortable.

P/N	FITS CAGES		FLOOR SIZE (WxHxD)	
	In.	Cm	Inches	Centimeters
12385-00-FNALDO	36W	91.44W	33.438 x 3 x 23.25	84.93 x 7.62 x 59.06
12385-00-HJALDO	48W	121.92W	45.438 x 3 x 23.25	115.41 x 7.62 x 59.06

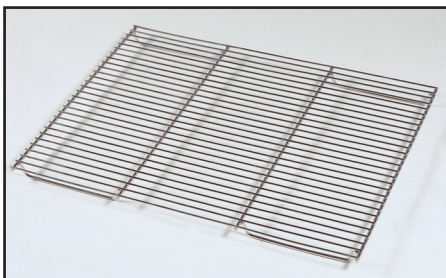

Stainless Steel Removable Cage Floors

Round rod stainless steel Cage Floor is welded at all intersections for superior strength and durability. Rods are deburred and spaced .750-in. (1.90cm) apart and have electropolished finish. Allows animals to rest 1.5-in. (3.81cm) above the cage bottom to stay clean, dry and comfortable.

P/N	FITS CAGES		FLOOR SIZE (WxHxD)	
	In.	Cm	Inches	Centimeters
12375-00-FNAFDN	36W	91.44W	33.188 x 1.412 x 23	84.30 x 3.59 x 58.42
12375-00-HJAFDN	48W	121.92W	45.188 x 1.412 x 23	114.78 x 3.59 x 58.42

Regal Intensive Care Unit

Cage Body

Convenient access ports may be used for IV lines or electronic monitoring equipment.



Plastisol-coated Cage Floors are included for patient comfort.

Removable wire grill and clear acrylic panel (included) slide into rear channel to create two compartments.

Built-in floor heaters provide even, steady heat to help maintain the patient's body temperature.

Thermometer/hygrometer continuously measures the temperature and humidity inside the compartment.

Alarm sounds when oxygen falls below your desired concentration.

Oxygen meter monitors oxygen concentration.

Monitoring window lets you visually check when the carbon-dioxide filter needs to be changed.

Oxygen flowmeter regulates the flow of oxygen delivery.

Separate lights for each function indicate when function is on and operating.

Controls

Regal Intensive Care Unit

Our Intensive Care Unit has been uniquely engineered to provide a temperature and oxygen controlled environment to care for your most seriously ill patients. This versatile unit can accommodate one large animal or provide an intensive care environment for two smaller patients simultaneously.

The unit controls temperature in two ways: by controlling air temperature and by warming the floor with two separate floor heaters. Continually recirculating air flow can be electrically warmed if necessary, or cooled and humidity-controlled with the built-in refrigeration condensing unit. Electric floor heaters have individual thermostats to help maintain the patient's body temperature during recovery from surgery or other treatments. To ensure your patients are comfortable, a built-in thermometer/hygrometer monitors the temperature and humidity inside the unit. A display window on the control panel lets you check the temperature and humidity quickly and easily.

To control air levels, filtered air or oxygen-enriched air from a pressure-regulated oxygen supply can be recirculated throughout the compartment as often as every 30 seconds. The oxygen concentration level and flow volume are accurately controlled, continuously monitored and displayed on the control panel. An audible alarm alerts

you when oxygen levels fall below the level you have set. Carbon dioxide is removed when circulating air passes through a soda-lime filter, and a filter monitoring window on the front panel allows you to see when filter needs to be changed.

Our rugged Intensive Care Unit is made of durable, easy-to-sanitize stainless steel that is unaffected by moisture, high humidity or temperature. Unit features clear, polycarbonate compartment doors to give you an unobstructed view of your patients. Double-door compartment has outer door latches to eliminate a center door post for easy access to the animals inside. A full rubber gasket on the doors provides a complete, leakproof seal. Add the wire grill or clear acrylic divider to create two smaller compartments. Two patients may be treated simultaneously with the wire grill divider in place. IV access ports allow you to change the IV supply without opening the compartment or disturbing the patient.

Requires 120V AC▲, and a pressure-regulated oxygen supply. Order your choice of a Mobile Stainless Steel Base or a Imperial/Regal Modular Cabinetry Base shown on next page.

P/N	CAGE SIZE		OVERALL DIMENSIONS (WxHxD)	
	In.	Cm	Inches	Centimeters
12019-00-NGFREP	60W	121.92W	86.125x37x30	218.76x93.98x76.20

▲ 220V requires step-down transformer. Contact SSCI.

Finishing Accessories For Regal Intensive Care Unit



Mobile Base for Intensive Care Unit

Fully welded stainless steel base moves easily on Swivel Casters with 5-in. (12.70 cm) diameter wheels. Brakes on all four wheels hold unit stationary for safety.

P/N	DIMENSIONS	
	Inches	Centimeters
12024-00-NIDREJ	86.75W x 30H x 29.25D	220.35W x 76.20H x 74.30D



Imperial/Regal Modular Cabinetry Base for Intensive Care Unit

Our quality-crafted Imperial Laminated or Regal Stainless Steel Cabinetry can be used to create an ideal base for your Intensive Care Unit along with easy-access storage. The unit shown includes two cabinets with a pair of doors and one cabinet with two drawers. A counter top and end panels give the unit a finished appearance. Order two cabinets that are

30-in. (76.20cm) wide and one cabinet that is 24-in. (60.96cm) wide.

Cabinet with counter top overall measurements:

87.25-in.W x 26-in.H x 28.25-in.D

221.62cmW x 63.50cmH x 71.75cmD

See next page for Cabinetry Bases.

Finishing Accessories For Regal Intensive Care Unit

Imperial Cabinet body panels are constructed of 45-lb. industrial-grade substrate, panels are machined on a CNC machine for accuracy. Panels are dadoed and rabbetted for tight-fitting joints. Joints are glued with adhesive impervious to liquids and mechanically-fastened to ensure a solid, sturdy cabinet.

All cabinet interiors are finished with Thermo-Fused Melamine, making them water, odor and chemical-resistant.

Drawer's steel sides feature integrated roller slides with baked-on, white epoxy finish. Drawer's back/bottom are .625-in. thick; open/close easily with 100 lb. load. 3D micro-adjustment ensures drawer alignment.

Door's positive-action, invisible hinges have self-closing mechanism, also holds door open. 3D micro-adjustment ensures door alignment. Choose Chrome Wire Pulls or full-width Aluminum Pulls for doors and drawer fronts .

All finished, exterior surfaces including the body's edges, drawer and cupboard door edges are laminated with high-pressure plastic laminate, available in SSCI Standard colors, or SSCI optional colors or choose from a spectrum of customer-selected colors to blend with your décor.



Cabinet, Pair of Doors

Height: 24.50-in. (59.69cm) Depth: 28.25-in. (71.75cm)

P/N Imperial	Regal	WIDTH	
		In.	Cm
Cabinet, Chrome Wire Pulls; 1 adjustable shelf			
ARAPAEAAEPDTEI - _ _ _ _ †	N/A	30W	76.20W
Cabinet Full-Width ; 1 adjustable shelf			
ARAPAEAAEPDTEI - _ _ _ _ †	NRAPAEAAEPDTEI	30W	76.20W

Regal Cabinet body panels are constructed of die-formed stainless steel, panels are formed on a CNC machine for accuracy. Panels are engineered for tight-fitting joints. Joints are welded and mechanically-fastened to ensure a solid, sturdy cabinet.

All cabinet interiors are stainless steel, making them impervious to water and absorbing odors.

Drawer's front is double-wall stainless steel construction with a full-width pull incorporated into the design. Welded, one-piece drawer body is stainless steel with roller telescoping slides allow drawers to fully open/close easily with 100 lb. load.

Doors are double-wall stainless steel construction with a full-width pull incorporated into the design and features positive-action invisible hinges which have self-closing mechanism, also holds door open. 3D micro-adjustment ensures door alignment.

Doors and drawer fronts have full width Stainless Steel Pulls.

All stainless steel exterior/interior is durable and easy to clean, satin finish.

Cabinet, 2 "F"-Style Drawers

Height: 24.50-in. (59.69cm) Depth: 28.25-in. (71.75cm)

P/N Imperial	Regal	WIDTH	
		In.	Cm
Cabinet, Chrome Wire Pulls			
ARBOAEAAABDRDTEI - _ _ _ _ †	N/A	24W	60.96W
Cabinet Full-Width Pulls			
ARBOAEAAABDRDTEI - _ _ _ _ †	NRBOAEAAABDRDTEI	24W	60.96W



End Panel for Cabinets


Width: .625-in. (1.59cm) Height: 24.50-in. (62.23cm) Depth: 28.25-in. (71.75cm)



P/N Imperial	Regal	Description
DROKAEAAAACDTEI - _ _ _ _ †	QROKAEAAAACDTEI	Fits on Left or Right End of Cabinet

Countertop for Intensive Care Cabinet Base

Width: 87.25-in. (221.62cm) Height: 1.50-in. (3.81cm) Depth: 30.25-in. (76.84cm)

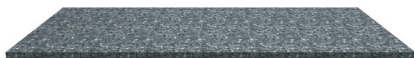


P/N Imperial	Regal	Description
FCAJHAANKAAAAEQ - _ _ _ _ †	SCAJHAANKAAAAEQ	Countertop

Important: These customer order requirements, in addition to other customer order requirements, must be satisfied prior to the beginning of manufacturing lead time:
 †Specify 4-digit Color code. See Color Selections, Section A, pages 1 & 2 for Color Codes.



SSCI laminated products are available in SSCI standard laminates, SSCI optional laminates, or Customer selected laminates.

Regal Cage Doors Kits for Customer Built Cages

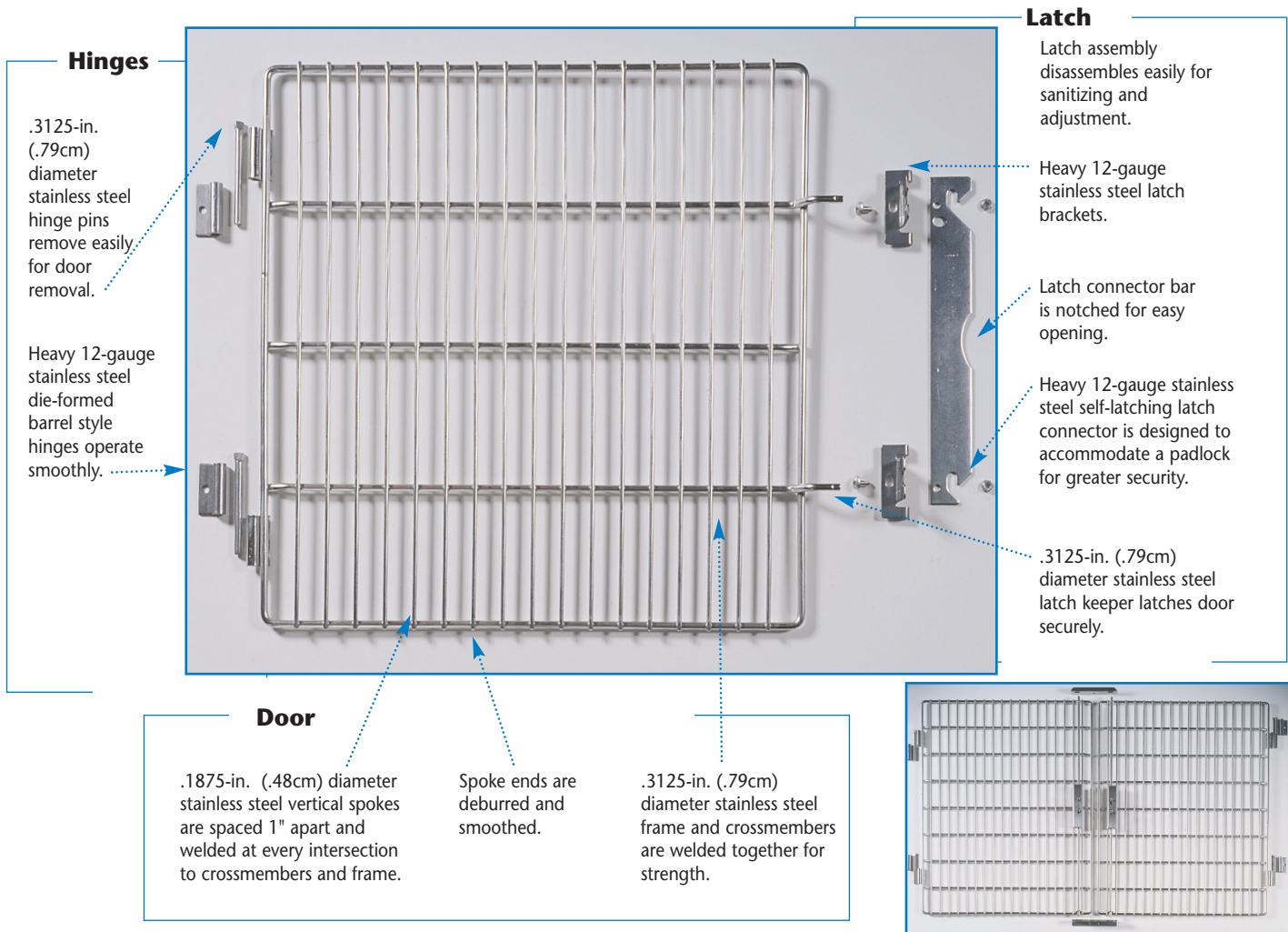

Hinges

.3125-in. (.79cm) diameter stainless steel hinge pins remove easily for door removal.

Heavy 12-gauge stainless steel die-formed barrel style hinges operate smoothly.

Latches

Latch assembly disassembles easily for sanitizing and adjustment.

Heavy 12-gauge stainless steel latch brackets.

Latch connector bar is notched for easy opening.

Heavy 12-gauge stainless steel self-latching latch connector is designed to accommodate a padlock for greater security.

.3125-in. (.79cm) diameter stainless steel latch keeper latches door securely.

Door

.1875-in. (.48cm) diameter stainless steel vertical spokes are spaced 1" apart and welded at every intersection to crossmembers and frame.

Spoke ends are deburred and smoothed.

.3125-in. (.79cm) diameter stainless steel frame and crossmembers are welded together for strength.

Double Door Cage Door

Regal Cage Door Kits

Our Regal cage doors, latches and hinges are so superior, we now offer Regal Cage Door Kits for installation on wood or masonry cages you build yourself. Stainless steel doors feature the exact construction specifications as the doors included on each Regal Cage. Each kit includes our superior-quality stainless steel Cage Door plus our unique latches and hinges. Fasteners are not included.

Before ordering, please call your SSCI or local authorized representative for advice about what installation fasteners are the best for your application.

P/N	FOR CAGE SIZE		FOR CAGE OPENING	
	Inches	Centimeters	Inches	Centimeters
12295-00-CTCTAA	18W x 18H	45.72W x 45.72H	16W x 16H	40.64W x 40.64H
12295-00-CTDRAA	24W x 18H	60.96W x 45.72H	22W x 16H	55.88W x 40.64H
12295-00-DRCTAA	18W x 24H	45.72W x 60.96H	16W x 22H	40.64W x 55.88H
12295-00-DRDRAA	24W x 24H	60.96W x 60.96H	22W x 22H	55.88W x 55.88H
12295-00-DREPAA	30W x 24H	76.20W x 60.96H	28W x 22H	71.12W x 55.88H
12295-00-DRFNAA	36W x 24H	91.44W x 60.96H	34W x 22H	86.36W x 55.88H
12296-00-DRHJAA	48W x 24H DD	121.92W x 60.96H DD	46W x 22H	116.84W x 55.88H
12295-00-EPDRAA	24W x 30H	60.96W x 76.20H	22W x 28H	55.88W x 71.12H
12295-00-EPEPAA	30W x 30H	76.20W x 76.20H	28W x 28H	71.12W x 71.12H
12295-00-EPFNAA	36W x 30H	91.44W x 76.20H	34W x 28H	86.36W x 71.12H
12295-00-EPGLAA	42W x 30H	106.68W x 76.20H	40W x 28H	101.60W x 71.12H
12295-00-EPHJAA	48W x 30H	121.92W x 76.20H	46W x 28H	116.84W x 71.12H
12296-00-EPHJAA	48W x 30H DD	121.92W x 76.20H DD	46W x 28H	116.84W x 71.12H
12295-00-FNDRAA	24W x 36H	60.96W x 91.44H	22W x 34H	55.88W x 86.36H
12295-00-FNEPAA	30W x 36H	76.20W x 91.44H	28W x 34H	71.12W x 86.36H
12295-00-FNFNAA	36W x 36H	91.44W x 91.44H	34W x 34H	86.36W x 86.36H
12295-00-FNGLAA	42W x 36H	106.68W x 91.44H	40W x 34H	101.60W x 86.36H
12295-00-FNHJAA	48W x 36H	121.92W x 91.44H	46W x 34H	116.84W x 86.36H
12296-00-FNHJAA	48W x 36H DD	121.92W x 91.44H DD	46W x 34H	116.84W x 86.36H
12296-00-FNJFAA	60W x 36H DD	152.40W x 91.44H DD	58W x 34H	147.32W x 86.36H
12296-00-FNLBAA	72W x 36H DD	182.88W x 91.44H DD	70W x 34H	177.80W x 86.36H

For Product Information Call TOLL-FREE 1-800-323-7366 • (847) 537-9320 • Fax (847) 537-9061

Hours: 8:30am - 5:00pm CST Monday - Friday

Visit our website at: www.suburbansurgical.com

Stainless Steel Cat Cages

Deluxe Stacking Cat Cages

Experience shows cats often require unique handling solutions. That's why we designed our Deluxe Stacking Cat Cages. Constructed entirely of stainless steel, these cages have several unique features to allow you to safely care for even the most feral cats without sacrificing their comfort.

Locking bowl holder system rotates the removable stainless steel feeding bowls out of the cage to let your attendants change food and water without disturbing the animal. Collection pan slides out easily for cleaning without opening the cage door. Stainless steel floor grill also removes for cleaning when the cage door is open.

Our uniquely designed round rod stainless steel grill top and side(s) offer cats a more open, better ventilated, less threatening environment. Solid stainless steel shelves provide additional resting surfaces up off the floor. Self-latching cage door latches securely, and features a 5-in. wide x 3-in. high (12.70cm wide x 7.62cm high) card holder for quick patient identification. For easy set-up in your facility, select the Cat Cage Rack on the facing page that best suits your needs.

Door

.162-in. (.411cm) diameter stainless steel vertical spokes are spaced 1-in. apart and welded at every intersection to crossmembers and frame.

.250-in. (.635cm) diameter stainless steel door frame and crossmembers are welded together for strength.

Includes a Card Holder; 5Wx3H-in. (12.70Wx7.62Hcm)

Latch

Self-latching door latch accommodates a padlock for greater security.

Interior

Solid shelves give cats additional resting surfaces.

Floor removes easily when cage door is open.


Removable collection pan slides out easily for cleaning.

Bowl Holder

Two stainless steel bowls rotate out to fill for safe, quick and easy feeding.


Rotating feeder latches securely in open or closed position for safety.

Spring-loaded latch mechanism locks feeder in position for safety.

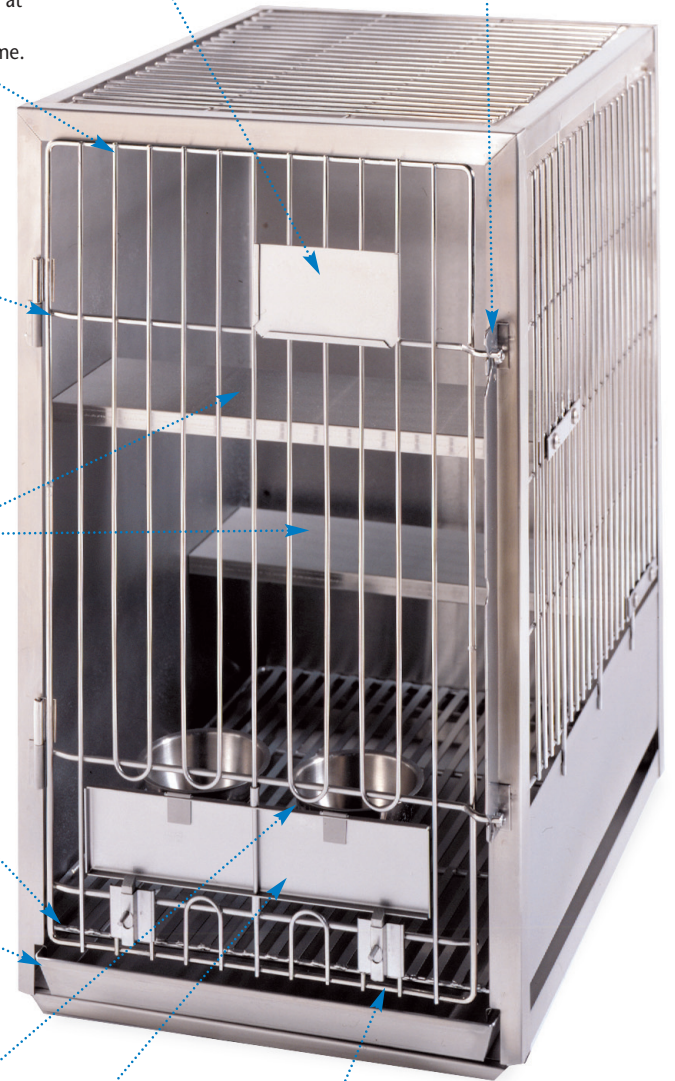

P/N	Floor Area		Inside Height		Shelves	Overall Width		Overall Height	
	Sq. Ft.	Sq. Meters	Inches	Centimeters		Inches	Centimeters	Inches	Centimeters
101447-00	3	.28	27H	68.58H	2	18W	45.72W	30.875H	78.42H
101448-00	4	.37	24H	60.96H	1	24W	60.96W	27.50H	69.85H
101449-00	5	.46	27H	68.58H	2	30W	76.20W	30.875H	78.42H

Stainless Steel Cat Cage Racks

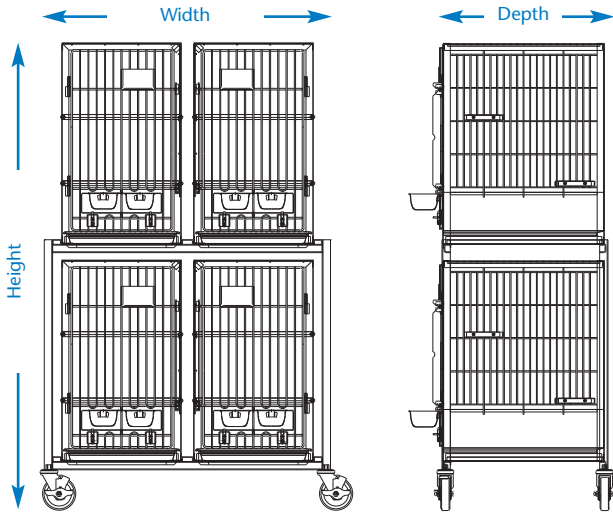

Cage Rack

- Rack is uniform non-glare finish.
- Stainless steel channel between cages prevents shifting.
- Sturdy 5-in. (12.70cm) swivel casters allow for easy mobility.

Deluxe Stacking Cat Cage Racks

Solid stainless steel Cat Cage Racks are designed to offer you quick set-up and easy cage access along with the ability to move the cages where you need them in your facility. Sturdy full-length runners front and back and welded stainless steel channels between each cage hold cages securely in place on the rack. All frame joints are butted and welded for strength, then all welds are smoothed to be free of crevices. The rack rolls easily on nickel-plated Swivel Casters with 5-in. diameter (12.70cm) wheels to allow you to move the entire bank of cages when needed. Two swivel casters have brakes for safety.

P/N	HOLDS CAGES		No. of Tiers	DIMENSIONS (INCHES)		DIMENSIONS (CENTIMETERS)	
	Total	Per tier		Without cages	With Cages	Without cages	With Cages
For 3 sq. ft. cages							
104520-00	4	2	2	44.281W x 40.906H x 26.625D	70.625H x 27.250D	112.47W x 103.90H x 67.63D	179.39H x 69.22D
104521-00	6	2	3	44.281W x 73.656H x 26.625D	103.438H x 27.250D	112.47W x 187.09H x 67.63D	262.73H x 69.22D
104522-00	6	3	2	64.188W x 40.906H x 26.625D	70.625H x 27.250D	163.04W x 103.90H x 67.63D	179.39H x 69.22D
104523-00	9	3	3	64.188W x 73.656H x 26.625D	103.438H x 27.250D	163.04W x 187.09H x 67.63D	262.73H x 69.22D
For 4 sq. ft. cages							
104526-00	4	2	2	56.281W x 37.406H x 26.625D	63.625H x 27.250D	142.95W x 95.01H x 67.63D	161.61H x 69.22D
104527-00	6	2	3	56.281W x 66.656H x 26.625D	92.875H x 27.250D	142.95W x 169.31H x 67.63D	235.90H x 69.22D
104528-00	6	3	2	82.188W x 37.406H x 26.625D	63.625H x 27.250D	208.76W x 95.01H x 67.63D	161.61H x 69.22D
104529-00	9	3	3	82.188W x 66.656H x 26.625D	92.875H x 27.250D	208.76W x 169.31H x 67.63D	235.90H x 69.22D
For 5 sq. ft. cages							
104532-00	4	2	2	68.281W x 40.906H x 26.625D	70.625H x 27.250D	173.43W x 103.90H x 67.63D	179.39H x 69.22D
104533-00	6	2	3	68.281W x 73.656H x 26.625D	103.438H x 27.250D	173.43W x 187.09H x 67.63D	262.73H x 69.22D
104534-00	6	3	2	100.188W x 40.906H x 26.625D	70.625H x 27.250D	254.48W x 103.90H x 67.63D	179.39H x 69.22D
104535-00	9	3	3	100.188W x 73.656H x 26.625D	103.438H x 27.250D	254.48W x 187.09H x 67.63D	262.73H x 69.22D

For Product Information Call TOLL-FREE 1-800-323-7366 • (847) 537-9320 • Fax (847) 537-9061
 Hours: 8:30am - 5:00pm CST Monday - Friday
 Visit our website at: www.suburbansurgical.com