




Premier Laminated Exam Tables

- *Professional, organized, efficient*
- *Stainless steel, laminated, & scale tops*
- *Selection of cupboard & drawer arrangements*
- *Top quality laminated construction*

Model Numbers: Model numbers vary with table styles - refer to current SSCI product catalog

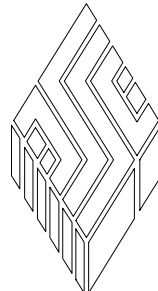

SSCI

Wheeling, IL (800) 323 7366

Table of Contents

General Information 4

- Introduction 4
- About this Manual 4
- Models 5
- Exam Table Tops 8
- Accessories 8
- SSCI Contact Information 9
- Cleaning Requirements 9
- Returning the Exam Table for Repairs 9
 - RMA Numbers 9
 - Packing & Shipment 9
- Warranty 10

Unpacking & Inspection 10

Use & Care 11

- Using Your Exam Table 11
 - Using the Drawers 11
 - Opening & Closing the Drawers 11
 - Removing & Replacing a Drawer 11
 - Using the Doors 12
 - Opening & Closing the Doors 12
 - Hinges - Overview 12
 - Removing a Door 12
 - Installing a Door 13
 - Adjusting the Door Left-to-Right 13
 - Adjusting the Door In-&-Out 14
 - Using the Shelf 15
 - Overview 15
 - Installation 15
 - Adjusting the Shelf Position 15

© Copyright 2007 by **Suburban Surgical Co., Inc.** All rights reserved.

No part of this document may be reproduced or utilized in any form or by any means, electronic or mechanical, including photocopying, recording, or by any information storage and retrieval system without written permission.

Inquiries should be addressed to **Suburban Surgical Co., Inc.**
Wheeling, Illinois 60090, USA

Care & Cleaning of Laminated Surfaces	16
Introduction	16
Cleaning Laminated Surfaces	16
Sharp Objects	16
Dusting	16
Cleaning Tips	16
Substances to Avoid	16
Recommended Cleaners	17
Cleaners to Avoid	18
Care & Cleaning of Stainless Steel	18
Introduction	18
Cleaning & Cleansers	18
Deodorizing Agents, Disinfectants, & Sanitizers	19
Effect on Warranty	19
Parts Replacement	20
Replacement Parts	20
General Information	21
Parts Ordering Procedure	21
Parts Replacement Procedures	22
Door, Hinged Left or Hinged Right	22
Overview	22
Tool Required	22
Procedure	22
Door Hinge	23
Overview	23
Tool Required	23
Procedure	23
Door Hinge Mounting Plate	24
Overview	24
Tools Required	24
Procedure	24
Door Hinge Suburban Surgical Logo Cap	25
Overview	25
Procedure	25
Drawer	25
Overview	25
Procedure	25
Stainless Steel Exam Top	26
Overview	26
Tool Required	26
Procedure	26
Laminated Exam Top	27
Overview	27
Tool Required	27
Procedure	27

General Information

Introduction

SSCI Premier Exam Tables are two-section laminated cabinets that give your exam room a custom appearance without the custom cost. Choose a table with the combination of drawers, cupboards, or refrigerator space that best fits your needs. Tables are offered with stainless steel, laminated, or electronic platform scale tops.



Figure 1. Premier Exam Table with Cupboard & 3 Drawers



Figure 2. Premier Exam Table with Refrigerator Space & 3 Drawers

Cabinets are finished with a durable, high-pressure laminate shaped on a CNC machine for accuracy. All joints are glued with an adhesive impervious to liquids, then mechanically fastened to guarantee a solid, sturdy cabinet. Cabinet interiors are finished with Thermo-Fused Melamine, making them water, odor, and chemical-resistant. All finished, exterior surfaces are covered with high-pressure plastic laminates that are available in a wide spectrum of customer-selected colors to blend with your facility's decor.

The particleboard substrate is the ideal solution for applications where high humidity and intermittent water contact are common, and is more resistant to moisture and water exposure than conventional industrial particleboard. SSCI's particleboard is manufactured with highly compressed wood fibers, and a specially formulated resin to produce a substrate with superior strength, rigidity, and an extra hard surface. Screw holding strength is substantially greater than typical industrial particleboard, while swelling caused by water soaking is much less.

All drawers open and close easily even with heavy loads. A positive stop on each drawer prevents the drawer from being pulled out accidentally. Drawers are easily removed and reinstalled. The door's positive-action, invisible hinges have self-closing mechanisms and 3D micro-adjustment.

About this Manual

Every attempt has been made to insure that the information in this manual is correct and complete. SSCI, however, always welcomes our customer's suggestions for improvements to our products and associated publications.

Models

SSCI Premier Exam Tables come in a wide variety of cabinet styles designed to fit your office requirements. Table 1 lists the cabinet styles available. Each of these cabinets can be ordered with your choice of a stainless steel, laminated, or electronic platform scale top. Table 2 lists the cabinets by part number.

Premier Exam Tables are also available with stainless steel legs with levelers or casters. These tables are not within the scope of this manual. If desired, refer to SSCI *Owner's Manual 702760*. For a copy, call SSCI Customer Service at (800) 323-7366.

Find descriptions, pictures, and part numbers for SSCI products and accessories in our current catalog, or visit our website at www.suburbansurgical.com. To order products, parts, or accessories, refer to *Parts Ordering Procedure* on Page 21.

Drawers	Cupboard	Refrigerator Space	Cupboard Door Hinged On	Additional Drawer Above Cupboard
3, Left Side	Right Side	No	Right	No
3, Right Side	Left Side	No	Left	No
3, Left Side	Right Side	No	Right	Yes
3, Right Side	Left Side	No	Left	Yes
4, Left Side	Right Side	No	Right	No
4, Right Side	Left Side	No	Left	No
4, Left Side	Right Side	No	Right	Yes
4, Right Side	Left Side	No	Left	Yes
3, Left Side 3, Right Side	No	No	N/A	N/A
4, Left Side 4, Right Side	No	No	N/A	N/A
3, Left Side	No	Right Side	N/A	N/A
3, Right Side	No	Left Side	N/A	N/A
4, Left Side	No	Right Side	N/A	N/A
4, Right Side	No	Left Side	N/A	N/A

Table 1. Premier Exam Table Cabinet Styles

SSCI Part Number	Top Style	Laminate Color	
		Cabinet	Top
Cupboard on Right, Door Hinged Right, & 3 Drawers			
L40DA01U0A0E0A2	Stainless Steel	- ____ *	N/A
L40DG01U0A0E0A2	Laminated	- ____ *	- ____ *
L40CA01U0A0E0A2	Electronic Scale	- ____ *	N/A
Cupboard on Left, Door Hinged Left, & 3 Drawers			
L40DA01D0A0U0A2	Stainless Steel	- ____ *	N/A
L40DG01D0A0U0A2	Laminated	- ____ *	- ____ *
L40CA01D0A0U0A2	Electronic Scale	- ____ *	N/A
Cupboard on Right, Door Hinged Right, 3 Drawers, & Drawer Above Cupboard			
L40DA01U0A0M0A2	Stainless Steel	- ____ *	N/A
L40DG01U0A0M0A2	Laminated	- ____ *	- ____ *
L40CA01U0A0M0A2	Electronic Scale	- ____ *	N/A
Cupboard on Left, Door Hinged Left, 3 Drawers, & Drawer Above Cupboard			
L40DA01K0A0U0A2	Stainless Steel	- ____ *	N/A
L40DG01K0A0U0A2	Laminated	- ____ *	- ____ *
L40CA01K0A0U0A2	Electronic Scale	- ____ *	N/A
Cupboard on Right, Door Hinged Right, & 4 Drawers			
L40DA01V0A0E0A2	Stainless Steel	- ____ *	N/A
L40DG01V0A0E0A2	Laminated	- ____ *	- ____ *
L40CA01V0A0E0A2	Electronic Scale	- ____ *	N/A
Cupboard on Left, Door Hinged Left, & 4 Drawers			
L40DA01D0A0V0A2	Stainless Steel	- ____ *	N/A
L40DG01D0A0V0A2	Laminated	- ____ *	- ____ *
L40CA01D0A0V0A2	Electronic Scale	- ____ *	N/A
Cupboard on Right, Door Hinged Right, 4 Drawers, & Drawer Above Cupboard			
L40DA01V0A0M0A2	Stainless Steel	- ____ *	N/A
L40DG01V0A0M0A2	Laminated	- ____ *	- ____ *
L40CA01V0A0M0A2	Electronic Scale	- ____ *	N/A

* Four-digit laminate color code - refer to SSCI product catalog

Table 2. Models - Premier Exam Tables (cont'd on Page 7)

SSCI Part Number	Top Style	Laminate Color	
		Cabinet	Top
Cupboard on Left, Door Hinged Left, 4 Drawers, & Drawer Above Cupboard			
L40DA01K0A0V0A2	Stainless Steel	- _ _ _ _ *	N/A
L40DG01K0A0V0A2	Laminated	- _ _ _ _ *	- _ _ _ _ *
L40CA01K0A0V0A2	Electronic Scale	- _ _ _ _ *	N/A
6 Drawers (No Cupboard)			
L40DA01U0A0U0A2	Stainless Steel	- _ _ _ _ *	N/A
L40DG01U0A0U0A2	Laminated	- _ _ _ _ *	- _ _ _ _ *
L40CA01U0A0U0A2	Electronic Scale	- _ _ _ _ *	N/A
8 Drawers (No Cupboard)			
L40DA01V0A0V0A2	Stainless Steel	- _ _ _ _ *	N/A
L40DG01V0A0V0A2	Laminated	- _ _ _ _ *	- _ _ _ _ *
L40CA01V0A0V0A2	Electronic Scale	- _ _ _ _ *	N/A
Refrigerator Space Right, & 3 Drawers			
L42DA01U0A0B2A2	Stainless Steel	- _ _ _ _ *	N/A
L42DG01U0A0B2A2	Laminated	- _ _ _ _ *	- _ _ _ _ *
L42CA01U0A0B2A2	Electronic Scale	- _ _ _ _ *	N/A
Refrigerator Space Left, & 3 Drawers			
L42DA01B2A0U0A2	Stainless Steel	- _ _ _ _ *	N/A
L42DG01B2A0U0A2	Laminated	- _ _ _ _ *	- _ _ _ _ *
L42CA01B2A0U0A2	Electronic Scale	- _ _ _ _ *	N/A
Refrigerator Space Right, & 4 Drawers			
L42DA01V0A0B2A2	Stainless Steel	- _ _ _ _ *	N/A
L42DG01V0A0B2A2	Laminated	- _ _ _ _ *	- _ _ _ _ *
L42CA01V0A0B2A2	Electronic Scale	- _ _ _ _ *	N/A
Refrigerator Space Left, & 4 Drawers			
L42DA01B2A0V0A2	Stainless Steel	- _ _ _ _ *	N/A
L42DG01B2A0V0A2	Laminated	- _ _ _ _ *	- _ _ _ _ *
L42CA01B2A0V0A2	Electronic Scale	- _ _ _ _ *	N/A

* Four-digit laminate color code - refer to SSCI product catalog

Table 2. Models - Premier Exam Tables (cont'd from Page 6)

Exam Table Tops

A key component of your SSCI Premier Exam Table is the top. SSCI provides three different top styles to perfectly suit your personal preferences (Figure 3).



SSCI's stainless steel table tops feature unique details such as raised edges on all four sides to contain fluids; hemmed sides to eliminate sharp edges; and a Thermo-Fused Melamine-finished substrate. This adds up to a table top that is durable, attractive, and easy to maintain.

SSCI's laminated tops feature rounded corners and are finished with high-pressure plastic laminates on the top surface and all edges. Laminates are available in SSCI standard and optional colors, or you can pick from a wide spectrum of customer-selected colors to perfectly blend with your office decor.

If you prefer, you might select SSCI's easy-to-use, Regal 300 Electronic Platform Scale with a stainless steel top and convenient control console. This scale allows you to quickly weigh an animal during examination.

Whichever top style you choose, you can be sure of SSCI's continuing commitment to the highest standards of quality and service. Our more than sixty-years of experience is your guarantee of complete satisfaction.

Figure 3. Top Styles for Premier Exam Tables

Accessories

A variety of accessories is available for SSCI Premier Exam Tables to increase your convenience and efficiency.

- Corner edge protectors - 12811-00-AAAAAA
- Pull-out tray - DRPAEEAAACJAODB
- Black vinyl ribbed mat - 12800-00-GNAACV
- Tie downs (stainless steel top) - 12805-00-AAAAAA
- Tie downs (laminated top) - 12806-00-AAAAAA
- Towel drop for cupboard door - Refer to catalog
- Privacy key lock - Refer to catalog

Find descriptions, pictures, and part numbers for SSCI products and accessories in our current catalog, or visit our website at www.suburbansurgical.com. To order products, parts, or accessories, refer to *Parts Ordering Procedure* on Page 21.

SSCI Contact Information

Contact SSCI Customer Service by mail, telephone, or fax. The department is available from 8:30am to 5:00pm, Central Time, Monday through Friday. Closed holidays.

Address: Suburban Surgical Co., Inc.
275 Twelfth Street
Wheeling, Illinois 60090

Telephone: Illinois - (847) 537-9320, ext. 3518
Toll Free - (800) 323-7366

Fax: (847) 537-9061

Web: www.suburbansurgical.com

Cleaning Requirements

Clean your exam table exactly in accordance with the cleaning instructions provided in the *Use & Care* section of this manual. *Failure to follow these instructions can void your warranty.*

Returning the Exam Table for Repairs

RMA Numbers

If your exam table should require return to SSCI for repairs, discuss the problem with one of our Customer Service Representatives. Obtain an RMA number (Return Merchandise Authorization) from them before shipping the unit back.

Note: SSCI will *not* accept merchandise returned without an RMA number.

Packing & Shipment

If you were able to keep the table shipping carton, repack the table into the carton and staple or tape the cover securely in place.

If the shipping carton is not available, package the table securely in a suitable container. Ship documentation with the table including:

- Destination
- RMA Number
- Your name, company, and address
- Your telephone number
- A description of the reason for returning the table

Warranty

Suburban Surgical Company, Inc. warrants the original purchaser that our products are of the highest standards in material and workmanship. Our stainless steel components are guaranteed to last a lifetime assuming they are used as intended, properly maintained and cared for. Mechanical, electrical, electronic, hydraulic, and any product's devices carry a one year warranty.

Items purchased by Suburban Surgical Company, Inc. from other manufacturers and incorporated into our equipment are covered by the respective manufacturer's warranties.

Warranties will not apply if it is determined by Suburban Surgical Company, Inc. that the equipment became defective due to an accident, misuse, abuse, improper maintenance or alteration. Warranty freight charges are covered for the first year only.

Unpacking & Inspection

CAUTION: Unpacking the Premier Exam Table is not difficult. The table is heavy, however, and we recommend that unpacking be done by at least two people.

If the shipping container appears damaged in any way, contact the shipping company immediately. Save all damaged packing materials to assist in proving liability for damage.

Carefully inspect the table as you unpack it. If any damage is noted, or if parts appear to be missing, call SSCI Customer Service at (800) 323-7366.

Use & Care

Using Your Exam Table

Using the Drawers

Opening & Closing the Drawers

The following instructions apply to all drawers on any Premier Exam Table. These drawers are designed to work smoothly even with heavy loads.

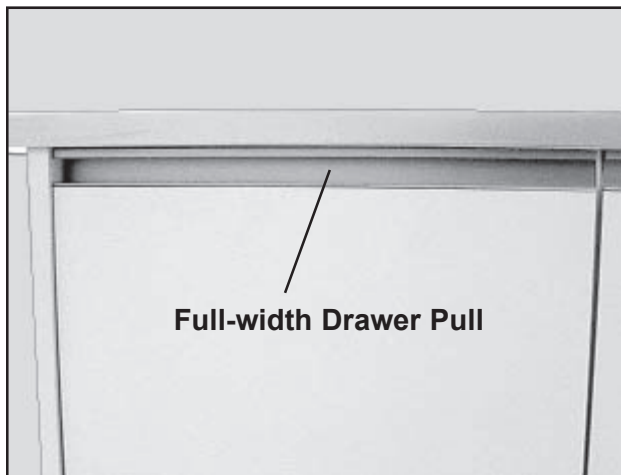


Figure 4. Full-width Drawer Pull

To open a drawer, turn your palm upward and, with your fingertips, grasp the full-width pull (Figure 4), then gently pull the drawer open.

To close the drawer, gently push the drawer in. Avoid overfilling the drawers so that the contents obstruct the movement of the drawers.

Removing & Replacing a Drawer

To remove a drawer, pull the drawer out, then tilt the front of the drawer up, and pull the drawer out of the cabinet.

To replace a drawer, slide the rollers at the rear of the drawer (Figure 5) over the rollers on the cabinet walls (Figure 6), then gently push the drawer in.

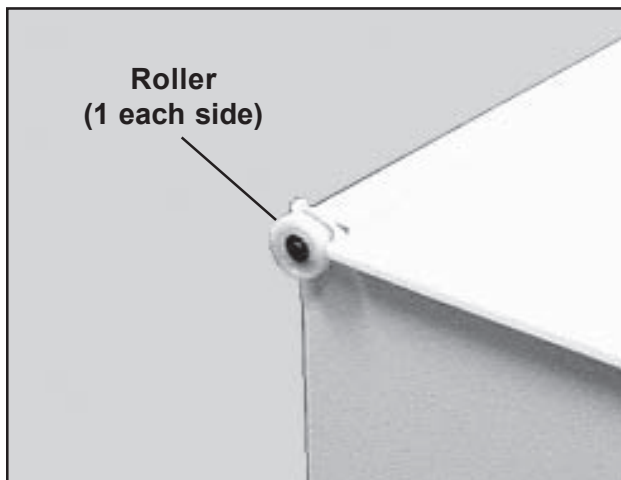


Figure 5. Roller on Rear of Drawer.

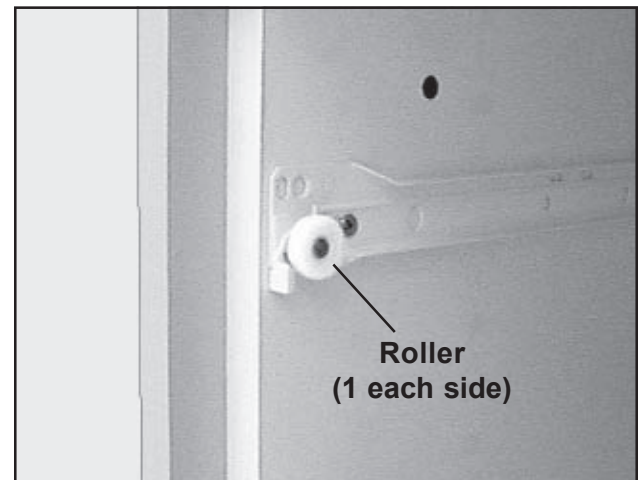


Figure 6. Roller on Cabinet Wall

Using the Doors **Opening & Closing the Doors**

The doors on the Premier Exam Table open approximately 110° for full and easy access to the interior. To open a door, turn your palm upward and, with your fingertips, grasp the full-width pull (Figure 7), and gently pull the door open. The door will hold itself open. To close the door, gently push the door closed and the door will stay closed. The doors feature positive-action invisible hinges, and 3D micro-adjustment ensures correct door alignment.

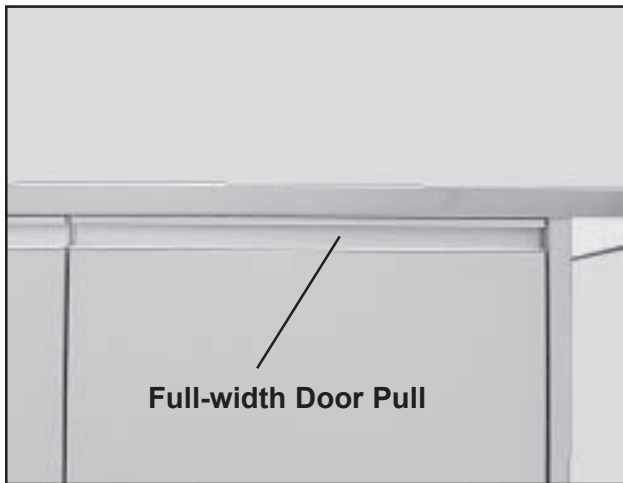


Figure 7. Full-width Door Pull

Hinges - Overview

The hinges used on the doors are sturdy, high quality devices. The following paragraphs will be helpful when removing or installing a door, or when adjusting the fit of a door in the cabinet. Figures 8 and 9 show close-up images of unmounted hinges to give you detailed views which would be difficult to obtain with the hinges mounted in place.

Removing a Door

The design of the hinges makes it easy to remove a door from the cabinet. To remove a door, squeeze the release catch on the hinge (Figure 8) and the hinge releases from the hinge mounting plate. When both hinges are released, the door comes off the cabinet.

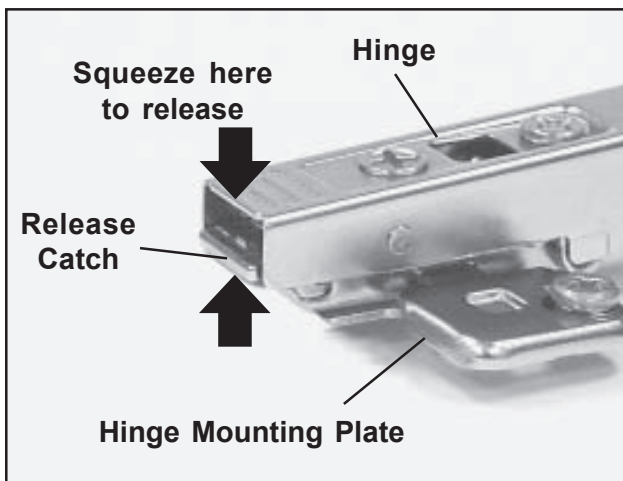


Figure 8. Hinge & Hinge Mounting Plate (unmounted)

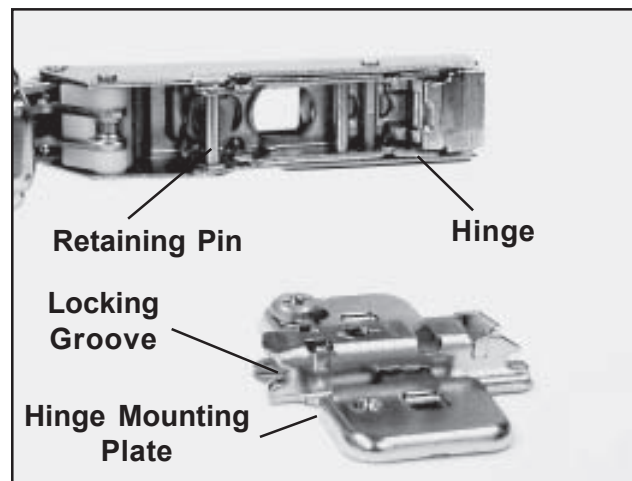


Figure 9. Hinge & Hinge Mounting Plate Separated

Installing a Door

Two hinges must be mounted to the door and two hinge mounting plates to the cabinet. Hook the retaining pin on the hinge into the locking groove in the hinge mounting plate (Figure 9). Then, pivot the hinge closed and press the end of the hinge down until it snaps into place. Do the same to both hinges to secure the door to the cabinet.

Adjusting the Door Left-to-Right

An adjustment on the hinge moves the door left and right within the door frame. By adjusting both hinges, you can center the door for equal spacing on the left and right. The door must be mounted on the cabinet to make this adjustment.

Tool Required: Phillips screwdriver

Procedure:

1. Close the cabinet door.
2. Observe the gaps between the left and right sides of the door and the door frame (Figure 10).
3. Open the door.
4. Pull the *Suburban Surgical* logo cap off the upper hinge.
5. With a Phillips screwdriver, turn the left-right adjustment screw (Figure 11) clockwise to move the door to the right; counterclockwise to move the door to the left.
6. Close the door and check the gaps.
7. Open the door and repeat the adjustment if necessary.
8. Continue as above until the gaps are equal.
9. Snap the *Suburban Surgical* logo cap back on the hinge.
10. Repeat the procedure for the lower hinge.

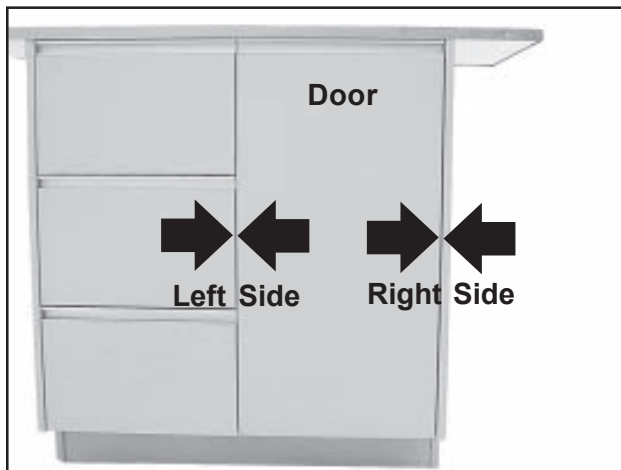


Figure 10. Adjusting the Door Left-Right

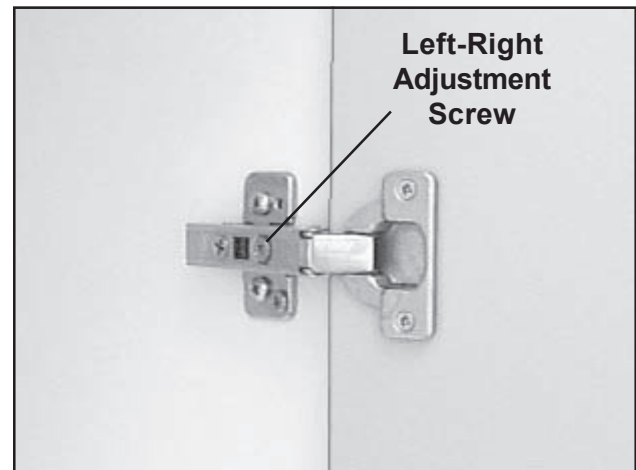


Figure 11. Door Left-Right Adjustment Screw

Adjusting the Door In-&-Out

An adjustment on the hinge moves the door in and out in the door frame. By adjusting both hinges, you can make the hinge side of the door flush with the cabinet frame. The door must be mounted on the cabinet to make this adjustment.

Tool Required: Phillips screwdriver

Procedure:

1. Close the cabinet door.
2. Observe whether or not the hinge side of the door is flush with the cabinet frame (Figure 12).
3. Open the door.
4. Pull the *Suburban Surgical* logo cap off the upper hinge.
5. With a Phillips screwdriver, turn the in-out adjustment screw (Figure 13) clockwise to move the door in; counterclockwise to move the door out.
6. Close the door and check to see if the door is flush with the cabinet frame.
7. Open the door and repeat the adjustment if necessary.
8. Continue as above until the door is flush with the frame.
9. Snap the *Suburban Surgical* logo cap back on the hinge.
10. Repeat the procedure for the lower hinge.



Figure 12. Adjusting the Door In-Out



Figure 13. Door In-Out Adjustment Screw

Using the Shelf Overview

Each Premier Exam Table having a cupboard comes with a single shelf and four shelf pegs (Figure 14). The shelf is easy to install and requires no tools. The height of the shelf in the cupboard can be easily adjusted to fit your storage requirements.

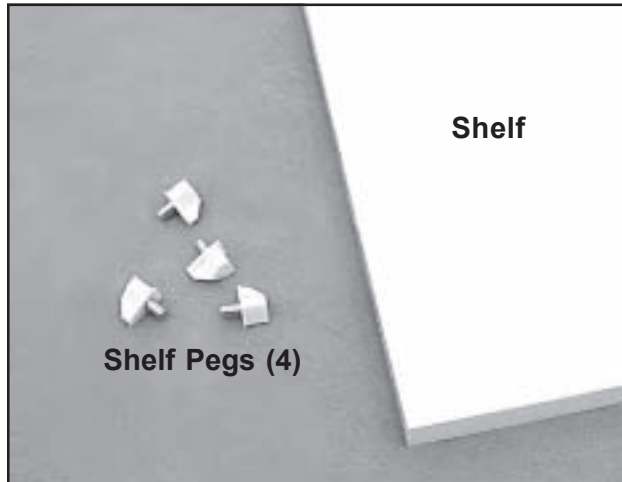


Figure 14. Shelf and Shelf Pegs

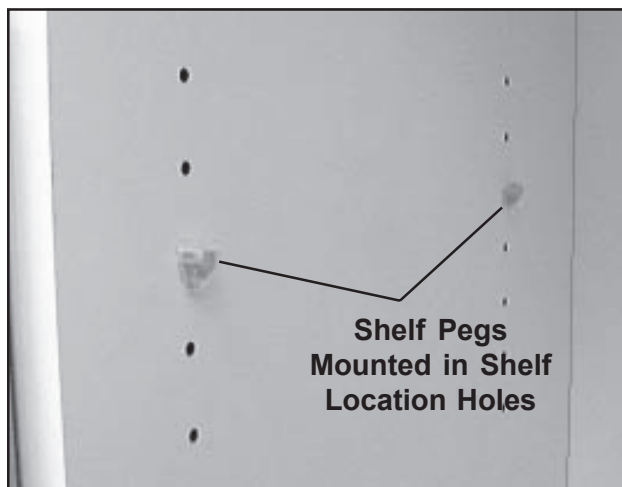


Figure 15. Shelf Pegs Mounted in Cupboard Wall

Installation

1. Open the cupboard door and remove the shelf and the four shelf pegs.
2. There are two vertical columns of nine holes in each side wall of the cupboard. Place the metal portion of one peg in a hole in each column so the shelf will be supported at four points (Figure 15). Leave the flat surface of the peg facing up. Make sure you install the pegs in the same hole in each column.
3. Place the shelf in the cupboard on the pegs (Figure 16). If the shelf rocks, you have placed a peg in the wrong hole. Remove the shelf, relocate the peg, and replace the shelf.

Adjusting the Shelf Position

Remove the shelf, relocate the pegs to the desired height, and replace the shelf.

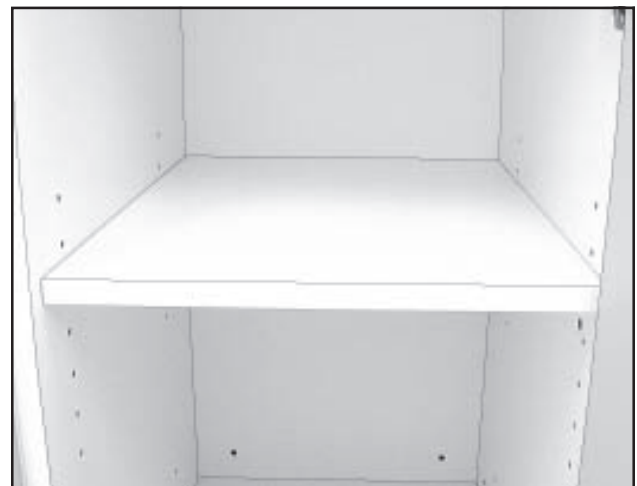


Figure 16. Shelf Mounted in Place

Care & Cleaning of Laminated Surfaces

Introduction

You will no doubt want to clean your exam table frequently. Maintaining high standards of sanitation will be an important priority for your facility.

CAUTION: The warranty for this product is void if the care and cleaning instructions provided here are not followed.

Cleaning Laminated Surfaces

Sharp Objects

Sharp objects can damage the laminate surface, marring its beauty, and lowering wear and stain resistance. Although high pressure laminates are somewhat resistant to scratching and marring, the surface can be damaged even under normal use.

Dusting

The laminate may need occasional dusting. To keep the surface beautiful, use a non-oily furniture spray. (Remember to clean the spray off several times a year to prevent build-up.) Furniture polish can also help hide fine scratches in the surface.

Cleaning Tips

To clean the laminated surface, use a damp cloth or sponge and a mild soap or detergent. Difficult stains such as coffee or tea can be removed using a mild household cleaner and baking soda, mixed to a paste-like consistency. Using a stiff nylon bristle brush, scrub (approx. 15 to 20 strokes) the affected area. Do not scrub so hard as to mar the surface finish.

Stubborn stains that resist the above cleaning method may require the use of undiluted household bleach or nail polish remover. Using a cotton ball saturated with bleach or nail polish remover (acetone), gently rub the stain for up to two minutes. Rinse thoroughly with warm water and wipe dry using a soft cloth. This step may be repeated if the stain appears to be going away, and the color of the laminate has not been affected.

Substances to Avoid

Acidic or abrasive cleaners can damage laminate surfaces - do not use them. Drain cleaners containing lye will permanently damage the laminate surface. If you spill or splash a drain cleaner on the laminate, wipe it up immediately and rinse several times with water.

CAUTION: Prolonged exposure of the laminate surface to bleach will cause discoloration. Always rinse laminate surfaces after cleaning! Failure to rinse after cleaning can cause damage even if only a small amount of cleaning solution remains on the surface. A dry residue may be invisible, however, moisture from cups or drinks can reactivate it, and result in permanently etched scars or stains over time.

Hair, textile, and food dyes can cause permanent stains. If dye should happen to spill or splash on the laminate, wipe it up immediately with dishwashing detergent or an all-purpose cleaner. Wipe spills away promptly and rinse several times with water.

Rust removers contain harsh chemicals which will quickly cause permanent damage. If a spill or splash occurs, wipe off all residue immediately, wash thoroughly with soapy water, and rinse several times.

Steel wool and other abrasive pads will damage the laminate surface. Do not use them for cleaning, and do not leave steel wool pads on the laminate; the metal can rust and leave stains.

Toilet bowl cleaners contain harsh chemicals that can cause permanent damage. If spills or splashes occur, wipe up immediately, wash the surface with soapy water and rinse several times.

Recommended Cleaners

The following commercially available cleaners can be used on the laminated surface:

- Clorox® (avoid prolonged exposure)
- Formula 409®
- Dawn®
- Glass Plus®
- Dow Bathroom Cleaner with Scrubbing Bubbles™
- Fantastik®
- Favor®
- Windex®
- Lestoil®
- Pledge®
- Grease Relief®
- Lysol® Brand Disinfectant Basin/Tub/Tile Cleaner
- Mr. Clean®
- TOP JOB®

Cleaners to Avoid

DO NOT use the following cleaners on the laminate:

Chemical Ingredient	Synonymous Names
■ Hydrochloric Acid	<i>Muriatic Acid</i> <i>Hydrogen Chloride</i> <i>Oleic Acid</i> <i>Oil of Vitriol</i> <i>Oleum</i>
■ Hydrofluoric Acid	<i>Rust Remover</i>
■ Phosphoric Acid	<i>Rust Remover</i>
■ Sodium Hydroxide	<i>Caustic Soda</i> <i>Caustic</i> <i>Lye</i> <i>Soda Lye</i>
■ Pumice (abrasive)	

Care & Cleaning of Stainless Steel

Introduction

Stainless steel is steel alloyed with chromium to make it highly resistant to stain, rust, and corrosion. **Note:** This does NOT mean that stainless steel will *never* rust or corrode. Science has not yet developed a steel which is completely stainless or corrosion PROOF.

The type of stainless steel and finish selected by SSCI for this product is the best available for the intended use.

Cleaning & Cleansers

The basic rule of thumb is to use the mildest cleaning agent that will do the job effectively. After cleaning, always rinse thoroughly with clear water, and dry completely. Frequent cleaning will prolong the service life of stainless steel equipment and will help maintain a bright, pleasing appearance.

Ordinary deposits of waste and fluids can usually be removed with soap and water. More stubborn deposits or tightly adhering debris may require harder scrubbing and possibly the use of commercial cleaning products acceptable for use on metal surfaces. When using any cleaning agent, rub in the direction of the polish lines or “grain” of the metal. For high luster finishes, clean soft cloths or pads should be used.

If especially rough cleaning is necessary, use “stainless steel” wool, nylon, or plastic scrubbers. Test these scrubbers in an inconspicuous area first to be sure they do not mar or scratch the stainless steel finish.

Minor scale build-up and some hard water spotting may be removed by washing with vinegar, followed by a neutralizing rinse with clear water, and a thorough drying with a soft cloth. For heavy deposits of scale, 5% oxalic acid (use warm), 5-15% sulfamic acid, or 5-10% phosphoric acid may be used. Always follow with a neutralizing rinse of clean water and a thorough drying.

Deodorizing Agents, Disinfectants, & Sanitizers

The large selection of brands and combinations of chemicals available for deodorizing, disinfecting, and sanitizing is staggering. Select one or more agents for use in your facility only after weighing all the benefits claimed by each product. Often this choice is made without adequate consideration of the effects these agents may produce on equipment or furnishings.

CAUTION: Before selecting a chemical to employ in your facility, review label statements regarding use with metals (stainless steel). Always consult the chemical supplier if there are any doubts.

Avoid prolonged use of chlorides (such as chlorine bleach), bromides, iodides, and thiocyanates on stainless steel surfaces as these chemicals will cause pitting, corrosion, and metal discoloration. Allowing salty solutions to evaporate and dry on stainless steel may also contribute to corrosive conditions.

In summary, select chemical deodorizers, disinfectants, and/or sanitizers only after weighing all possible benefits and known adverse effects.

Effect on Warranty

The warranty for this product is void if the care and cleaning instructions provided in this manual are not followed.

Parts Replacement

Replacement Parts

Table 3 lists replacement parts for the Premier Exam Table. For parts not listed, contact SSCI Customer Service at (800) 323-7366. To order, refer to *Parts Ordering Procedure* on Page 21.

Description	SSCI Part Number	Quantity	Replacement Instructions
Door, Hinged Left or Hinged Right	Contact SSCI	0 or 1, depending on table configuration	Page 22
Door Hinge	851910	2 per door	Page 23
Door Hinge Mounting Plate	851912	1 per hinge	Page 24
Door Hinge <i>Suburban Surgical</i> Logo Cap	854613	1 per hinge	Page 25
Drawer	Contact SSCI	varies, depending on table configuration	Page 25
Stainless Steel Exam Top	202344	1	Page 26
Laminated Exam Top	FCARHAAGYAAAADH-_____*	1	Page 27
Electronic Scale Top	For information on the SSCI Regal 300 Electronic Scale, refer to SSCI <i>Owner's Manual, Regal 300 Table-mounted Electronic Platform Scale, 702729.</i>		

* - ____ = 4-digit Laminate Color Code - refer to SSCI Product Catalog

Table 3. Replacement Parts - Premier Exam Tables

General Information

- During disassembly, retain all hardware items such as screws, nuts, lockwashers, etc. for reassembly.
- If you have problems with any procedure, please feel to call SSCI Customer Service at (800) 323-7366.

Parts Ordering Procedure

Order new equipment, accessories, and/or replacement parts directly through SSCI Customer Service. You can order by mail, telephone, or fax. Refer to *SSCI Contact Information on Page 9* for address, telephone, and fax numbers. When ordering parts, please provide the following information:

- Your name
- Company name
- Company account number
- Telephone number
- Fax number
- e-mail address
- Shipping address
- Billing address (if different from shipping address)
- Names, part numbers, and quantities of items being ordered
- Credit card number and expiration date, or other payment information
- Preferred method of shipment
- Information on whether the items are required on a normal or urgent basis

Parts Replacement Procedures

The following sections guide you in replacing worn, damaged, or missing parts on an SSCI Premier Laminated Exam Table.

Door, Hinged Left or Hinged Right
Contact SSCI for Part Numbers

Overview

The following instructions guide you in replacing a door hinged either left or right. For detailed information on the operation of the hinges, refer to *Page 12*.

Tool Required

- Phillips screwdriver

Procedure

1. Squeeze on the release catches of both hinges (Figure 17), and remove the door from the table. Refer to *Removing a Door* on *Page 12*.
2. With a Phillips screwdriver, remove the two screws on each hinge and remove both hinges from the old door (Figure 18). Keep the screws and hinges, and discard the old door.
3. With the screws removed above, mount the hinges on the new door.
4. Mount the door in the cabinet - engage the hinges into the hinge mounting plates, and snap the hinges into place. Refer to *Installing a Door* on *Page 13*.
5. Observe how the door fits in the cabinet. If necessary, adjust the door; refer to *Adjusting the Door Left-to-Right* on *Page 13* and *Adjusting the Door In-&-Out* on *Page 14*.



Figure 17. Door Hinge Release Catch

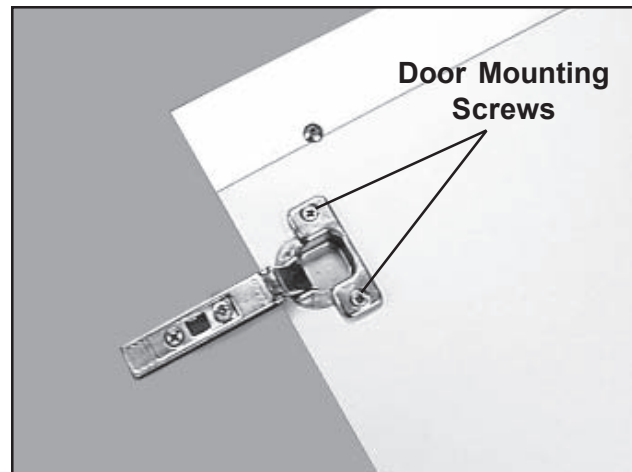


Figure 18. Hinge Mounted on Door

Door Hinge
P/N 851910**Overview**

The following instructions guide you in replacing a door hinge. All door hinges are identical whether the door is hinged-left or hinged-right. For detailed information on hinges, refer to *Page 12*.

Tool Required

- Phillips screwdriver

Procedure

1. Open the cabinet door.
2. Squeeze on the release catches of both hinges (Figure 17), and remove the door from the cabinet. Refer to *Removing a Door on Page 12*.
3. With a Phillips screwdriver, remove the two mounting screws on the hinge and remove the hinge from the door (Figure 18). Keep the screws and the door, and discard the old hinge.
4. Pull the *Suburban Surgical* logo cap off the new hinge.
5. Using the two screws removed above, mount the new hinge to the door.
6. Snap the *Suburban Surgical* logo cap onto the new hinge.
7. Mount the door in the cabinet - engage the hinges into the hinge mounting plates, and snap the hinges into place. Refer to *Installing a Door on Page 13*.
8. Observe how the door fits in the cabinet. If necessary, adjust the door; refer to *Adjusting the Door Left-to-Right on Page 13* and *Adjusting the Door In-&-Out on Page 14*.

Door Hinge Mounting Plate

P/N 851912

Overview

The following instructions guide you in replacing a door hinge mounting plate. There is one mounting plate behind each door hinge. All door hinge mounting plates are identical. For detailed information on hinges, refer to *Page 12*.

Tools Required

- Phillips screwdriver
- Pliers

Procedure

1. Open the cabinet door.
2. Squeeze on the release catches of both hinges (Figure 17), and remove the door from the cabinet. Refer to *Removing a Door* on *Page 12*.

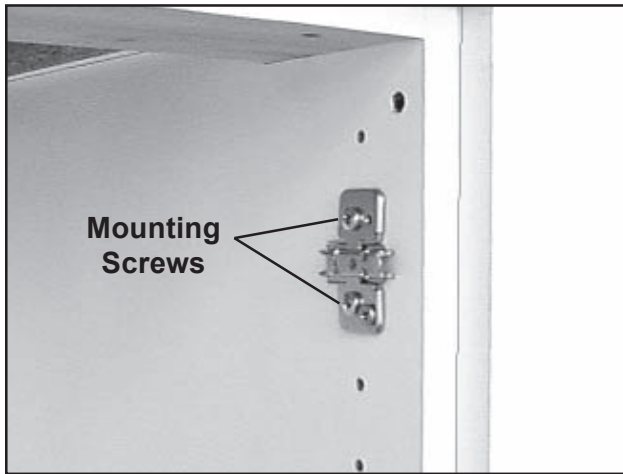


Figure 19. Door Hinge Mounting Plate

3. With a Phillips screwdriver, remove the two screws on the hinge mounting plate and remove the plate from the cabinet wall (Figure 19).
4. On the new mounting plate, two white plastic thread guards protect the plate mounting screws. Hold each guard with a pliers and, with a Phillips screwdriver, undo the screws. Discard the guards.
5. Mount the new mounting plate to the cabinet wall using the new screws supplied with the plate.
6. Mount the door in the cabinet - engage the hinges into the hinge mounting plates, and snap the hinges into place. Refer to *Installing a Door* on *Page 13*.
7. Observe how the door fits in the cabinet. If necessary, adjust the door; refer to *Adjusting the Door Left-to-Right* on *Page 13* and *Adjusting the Door In-&-Out* on *Page 14*.

Door Hinge
Suburban Surgical
Logo Cap
P/N 854613

Overview

The following instructions guide you in replacing a *Suburban Surgical* logo cap on a door hinge. The cap protects the door adjustment screws on the hinge from inadvertent tampering. There is one cap on each door hinge.

Procedure

1. Open the cabinet door.
2. Pull the old *Suburban Surgical* logo cap off the hinge.
3. Snap the new *Suburban Surgical* logo cap onto the hinge.

Drawer
Contact SSCI for
Part Numbers

Overview

The following instructions guide you in replacing a drawer. They are appropriate for all Premier Exam Table drawers.

Procedure

1. Remove all items from the drawer.
2. Pull the drawer out, then tilt the front of the drawer up, and pull the drawer out of the cabinet.
3. To replace the drawer, slide the rollers at the rear of the drawer (Figure 20) over the rollers on the cabinet wall (Figure 21), then gently push the drawer in.

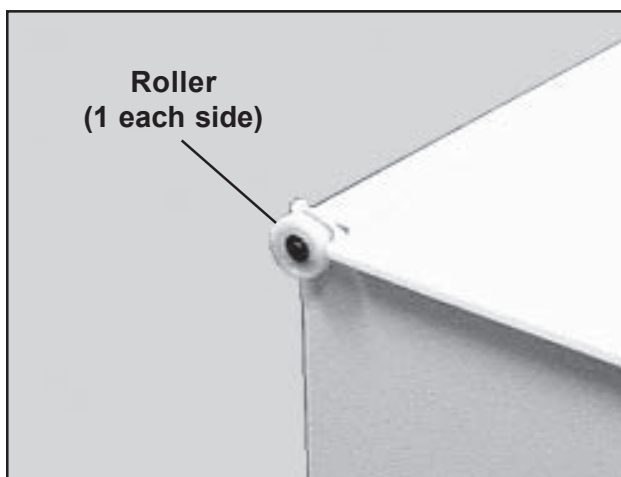


Figure 20. Roller on Rear of Drawer.

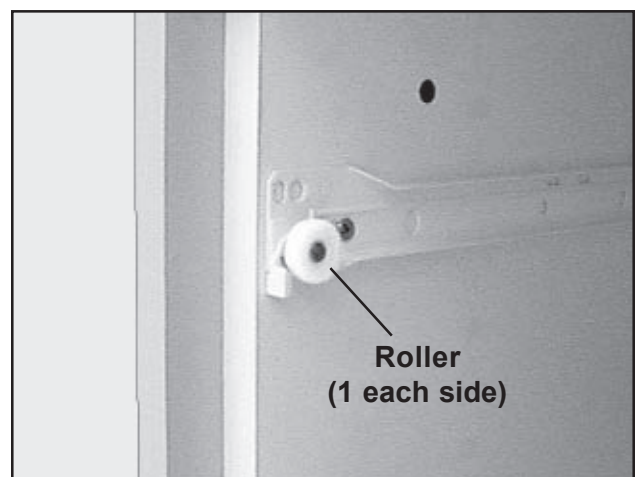


Figure 21. Roller on Cabinet Wall

**Stainless Steel
Exam Top
P/N 202344**

Overview

The stainless steel exam top can easily be replaced if it becomes damaged or otherwise unserviceable.

Tool Required

- 1/2 in. wrench

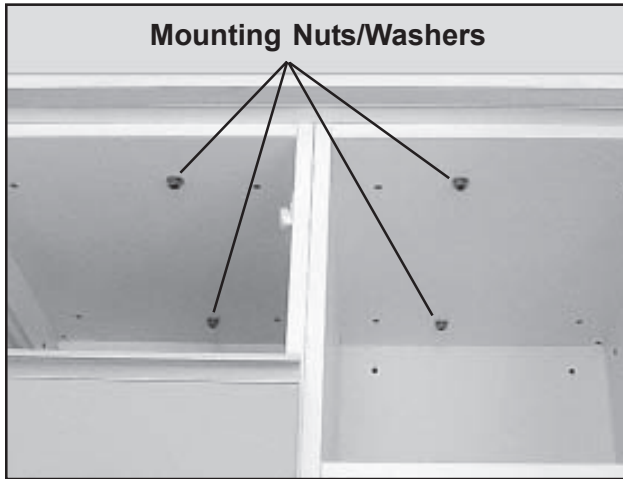


Figure 22. Interior View of Exam Table

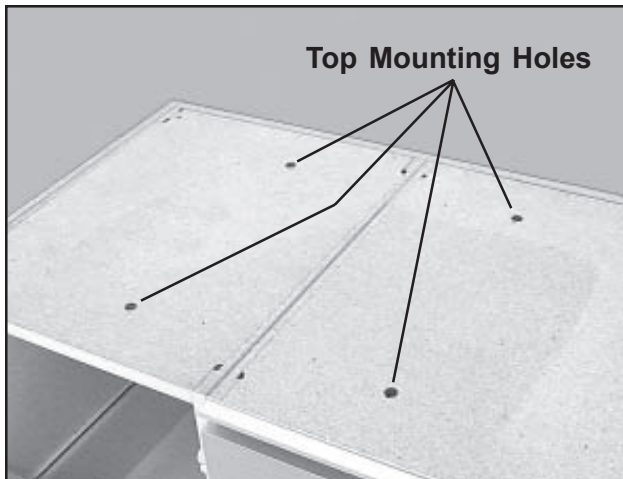


Figure 23. Stainless Steel Top Mounting Holes

Procedure

1. Remove the drawers, remove the shelves, and remove the refrigerator (if present) to gain access to the exam top mounting nuts.
2. With a 1/2 in. wrench, remove the four exam top mounting nuts and washers (Figure 22).
3. Lift off the old exam top.
4. Turn the new exam table top over and remove the four 1/2 in. nuts and washers from the four mounting screws on the underside.
5. Place the new exam top on the table with the four mounting screws into the four holes (Figure 23) on the upper surface of the table.
6. Use the four new nuts and washers removed in *Step 4* to secure the exam top to the table.
7. Replace all shelves and drawers, and reinstall the refrigerator, as required.

Laminated Exam Top
P/N FCARHAAGYAAAADH
 - _ _ _ _ *

Overview

The laminated exam top can easily be replaced if it becomes damaged or otherwise unserviceable.

Tool Required

- Phillips screwdriver

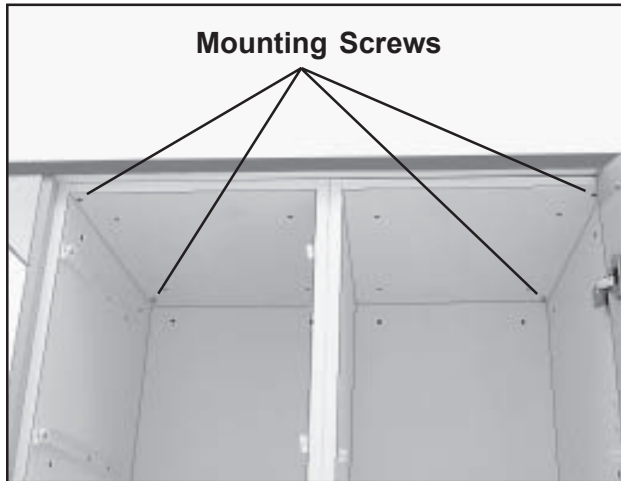


Figure 24. Interior View of Exam Table

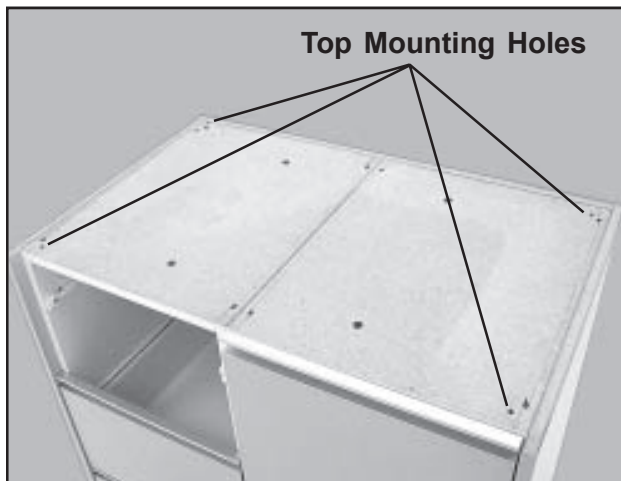



Figure 25. Laminated Top Mounting Holes

Procedure

1. Remove the drawers, remove the shelves, and remove the refrigerator (if present) to gain access to the exam top mounting nuts.
2. With a Phillips screwdriver, remove the four exam top mounting screws (Figure 24).
3. Lift off the old exam top.
4. Place the new exam top on the table.
5. Align the four mounting holes on the underside of the table top with the four holes in the upper surface of the table (Figure 25).
6. Using the four wood screws supplied, secure the exam top to the table.
7. Replace all shelves and drawers, and reinstall the refrigerator, as required.

* - _ _ _ _ = 4-digit Laminate Color Code - refer to SSCI Product Catalog

***For more information on SSCI's fine line of products
and accessories, talk to your SSCI sales representative.***

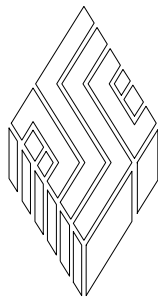

SSCI

Suburban Surgical Company, Inc.

275 Twelfth Street

Phone: (847) 537-9320

Toll Free: (800) 323-7366

Wheeling, IL 60090

Fax: (847) 537-9061

Web: www.suburbansurgical.com