


*Congratulations on your purchase of
the Classic Vet-Mate Lift Gurney with
Scale!*

Regal 300i Scale indicator bracket and scale mounting
hardware kit for Classic Vet-Mate Lift Gurney Owner's
Manual/Instructions

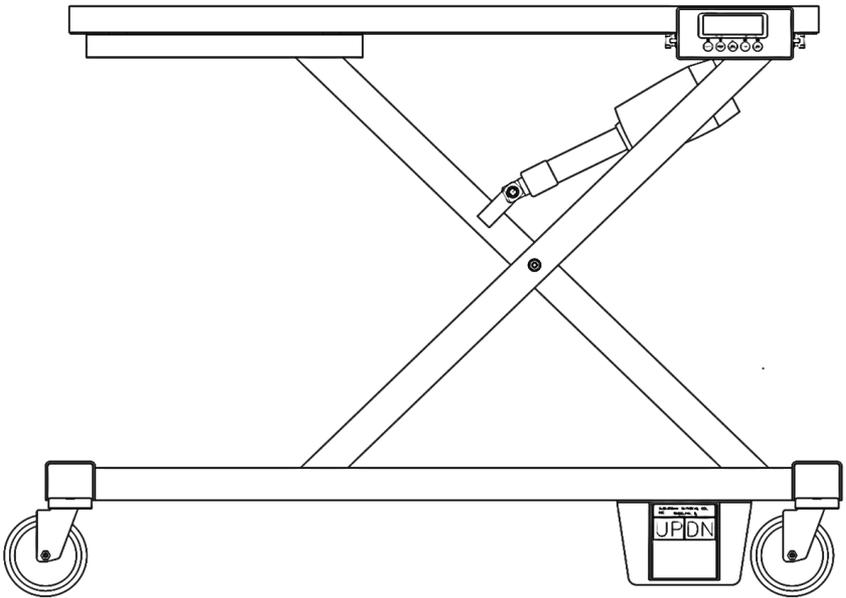

SSCI Contact Information

Suburban Surgical Co., Inc. 275 Twelfth Street Wheeling, Illinois 60090

Telephone: Illinois-(847)537-9320 Toll Free-(800)323-7366


Web: www.suburbansurgical.com

Removing Gurney Exam Top

1. Fully extend the gurney lift.
2. Using a Phillips screwdriver remove four (4) screws from the sides of the exam top.
3. Remove top.

Installing Scale Indicator

1. Insert four (4) AA alkaline batteries (included) before mounting bracket unless using power cord.
2. Mount bracket to underside of gurney (see below) using two (2) short ½-inch screws (included).

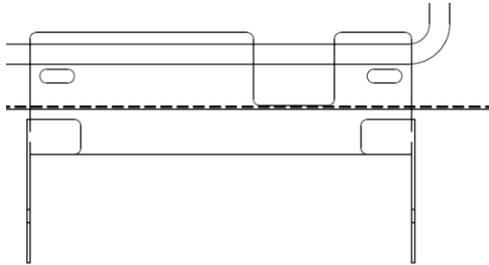


3. Mount indicator to bracket with attached hardware.

Installing Scale Top

1. Fully extend the gurney lift.
2. Place four (4) large 1-inch washers centered on top of scale spacers.
3. Hold down with masking tape (see right).
4. Unfold blue cable on bottom of scale and route through top of gurney frame.

HOLD WASHER IN PLACE
WITH MASKING TAPE
BEFORE PLACING SCALE

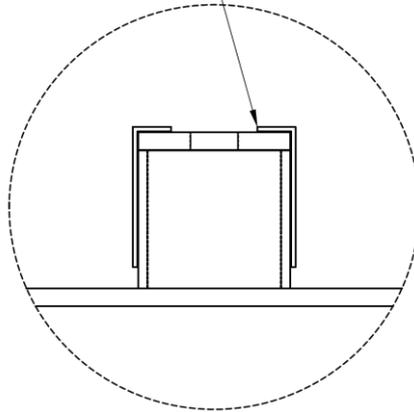

5. Gently place scale onto gurney, with scale feet directly on top of washers.
6. Using a Phillips screwdriver insert two (2) long 1 ¾-inch screws using two (2) small ¾-inch washers under the rear side of the gurney, over the foot pedal, into scale feet.
7. Using a Phillips screwdriver insert two (2) long 1 ¾-inch screws into the other scale feet. Tighten all four (4) screws.
8. Check gurney and scale at all corners and sides for a tight fit. If loose unscrew and check washer centering, repeat if necessary.

Re-Installing Gurney Exam Top

1. Fully extend the gurney lift.
2. Unscrew scale and gently remove.
3. Place top onto gurney.
4. Loosely insert four (4) screws with Philips screwdriver.
5. Tighten all four (4) screws.