


Congratulations on your purchase of the Access-A-Rail!

Access-A-Rail kit for Classic Vet-Mate Lift Gurney Installation Instructions

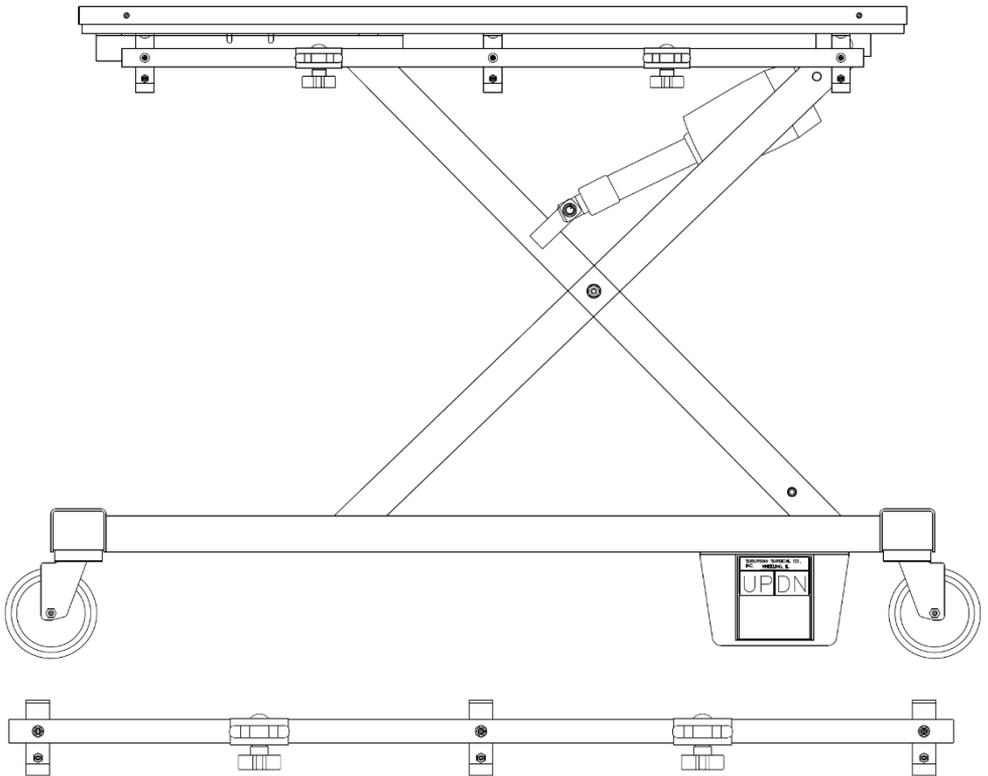

SSCI Contact Information

Suburban Surgical Co., Inc. 275 Twelfth Street Wheeling, Illinois 60090

Telephone: Illinois-(847)537-9320 Toll Free-(800)323-7366

Web: www.suburbansurgical.com



701613

If you have a Regal 300i Scale indicator installed on your lift gurney, the Access-A-Rail should be installed as below with no change. You may need to remove the Access-A-Rail to remove the Scale indicator.



The Access-A-Rail is easily installed with a screwdriver and a tool to hold the 7/16-inch lock nut such as an adjustable wrench or socket wrench. When disassembling, note that these nuts will lock in tightly.

Included in this kit are:

- Six (6) 1/2-inch long screws
- Six (6) 3/4-inch long screws
- Six (6) 7/16-inch lock nuts
- Four (4) tie-downs

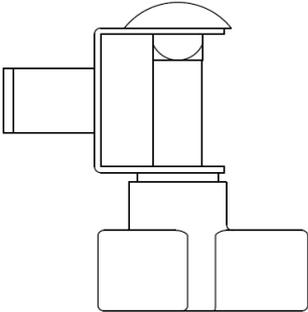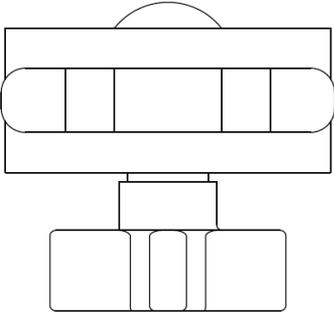


Six (6) brackets with attached spacers

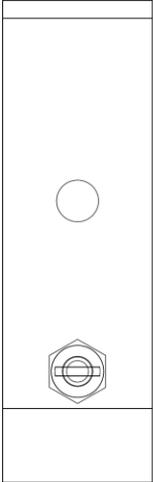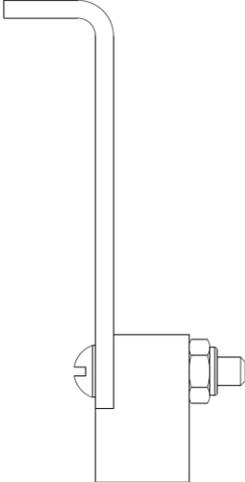

Access-A-Rail Installation

1. Lock all gurney casters
2. Screw each bracket into the underside of the gurney with the $\frac{1}{2}$ -inch screws. There are multiple holes near the rear bracket location on one side. This bracket must be screwed into the middle hole as shown below

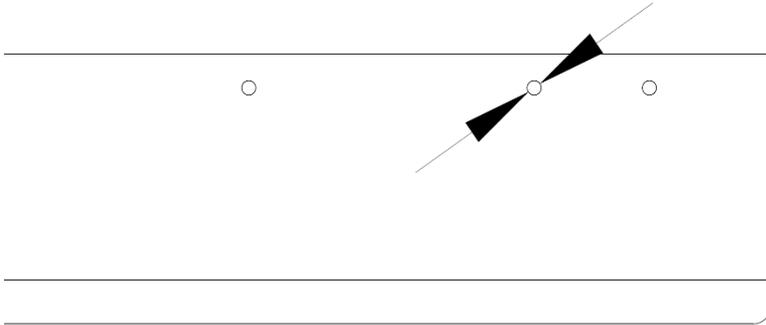

3. With a $\frac{3}{4}$ -inch screw, loosely mount the rail to the middle bracket, holding a lock nut behind the rail and bracket
4. Slide the tie-downs onto the rail next to the middle bracket on both sides, tighten until installation is done
5. Mount rail to two (2) outside brackets with $\frac{3}{4}$ -inch screws, holding a lock nut behind the rail and bracket
6. Tighten all $\frac{3}{4}$ -inch screws and lock nuts
7. Make sure both rails are tightly installed at all screws

Access-A-Rail Removal

1. Unscrew the rail from the three (3) brackets. Hold the lock nut behind the rail tightly until screw is loose
2. Unscrew each bracket