


Congratulations on your purchase of the Regal Vet-Mate Lift Gurney!

Regal Vet-Mate Lift Gurney Owner's Manual

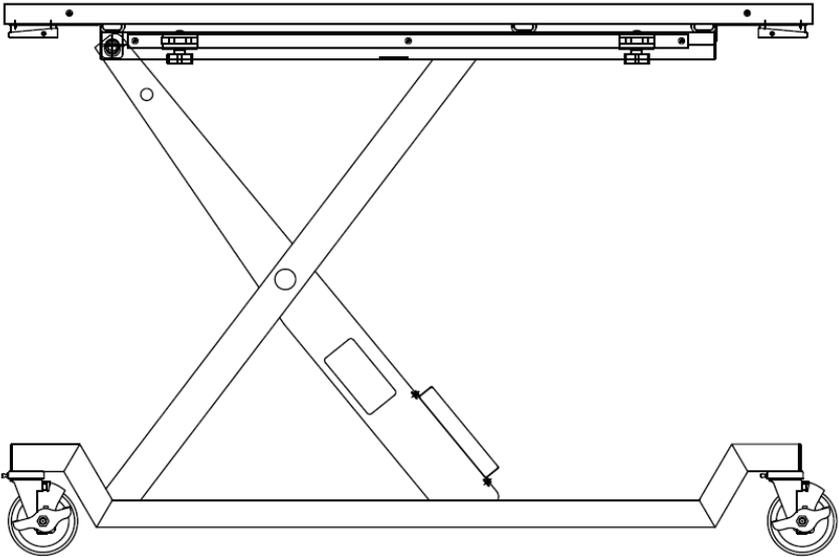

SSCI Contact Information

Suburban Surgical Co., Inc. 275 Twelfth Street Wheeling, Illinois 60090

Telephone: Illinois-(847)537-9320 Toll Free-(800)323-7366

Web: www.suburbansurgical.com

702700


Operation

The lift gurney is designed to carry weights up to 300 pounds (136 kg).

Raising and Lowering the Gurney

Two switches under each end of the gurney allow you to raise and lower the gurney. From either end:

- To raise the gurney, press the right-hand switch.
- To lower the gurney, press the left-hand switch.

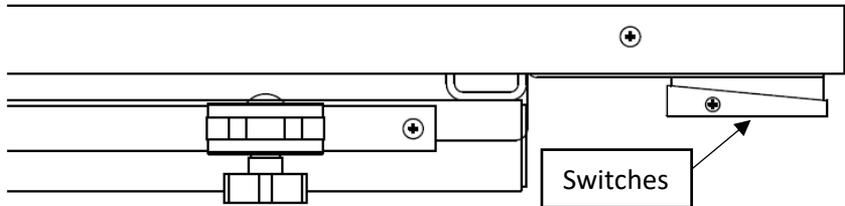


Hold either switch down until the gurney reaches the desired height, then release the switch.

Adjusting the Tie-Downs

There are two adjustable tie-downs on each side of the gurney. To reposition the tie-downs:


1. Loosen the adjustment knob on the tie-down by rotating it counterclockwise.
2. Slide the tie-down along the rail to the desired position.
3. Tighten the adjustment knob firmly by rotating it clockwise to secure the tie-down in place.

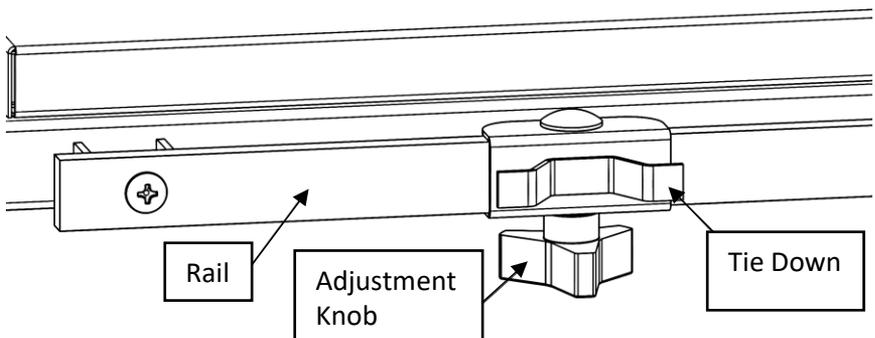

Maintenance

Charging the Battery

The battery is charged by a built-in charger in the large scissor leg, with its plug extended up under the gurney. Connect the plug to a power source. When the LED indicator turns green, the battery is fully charged.

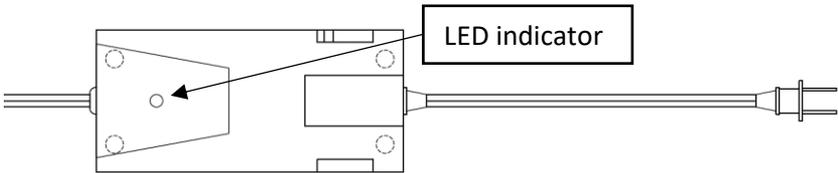

Inspecting the Translation Tube Mounting Points

The two translation tube mounting points should be inspected every six months for tightness. If either point is noticeably loose, tighten with the indicated tools.

- Translation tube nut

Check this point at the end of the translation tube.

Tool required: 3/4-in wrench



- Rod end retainer

Check this point under the gurney top.

Tools required:

9/16-in wrench,

1/4-in. Allen wrench

