


# *Congratulations on your purchase of the V-Style Operating Table Top!*

---

## V-Style Operating Table Owner's Manual

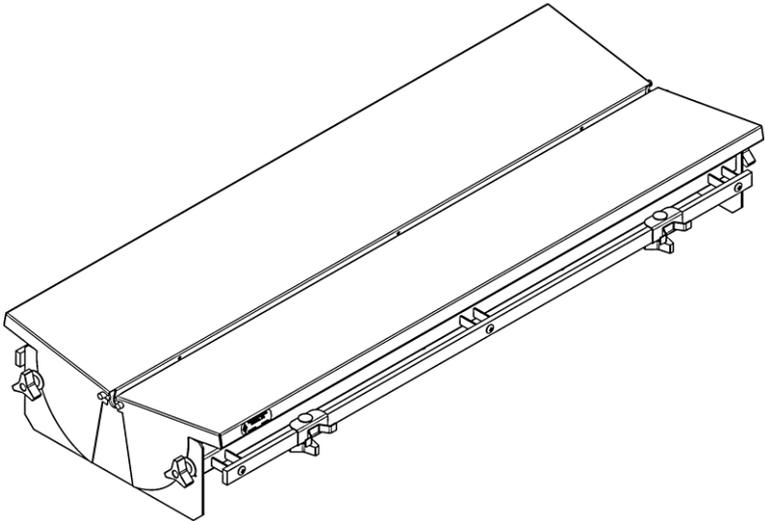

### **SSCI Contact Information**

Suburban Surgical Co., Inc. 275 Twelfth Street Wheeling, Illinois 60090

Telephone: Illinois-(847)537-9320 Toll Free-(800)323-7366


Web: [www.suburbansurgical.com](http://www.suburbansurgical.com)

702705

## Adjusting the Tabletop Panels

The tabletop consists of two adjustable V-panels designed to support a variety of surgical positions. To adjust the panels:


1. Loosen the adjustment knob at one end of the table by rotating counterclockwise while supporting the panel by hand. It is not necessary to loosen the knobs at both ends of the table.
2. Raise or lower the panel to the desired angle.
3. Tighten the adjustment knob firmly by rotating clockwise to lock the panel in position
4. Repeat the procedure for the opposite side until the desired V-angle is achieved.

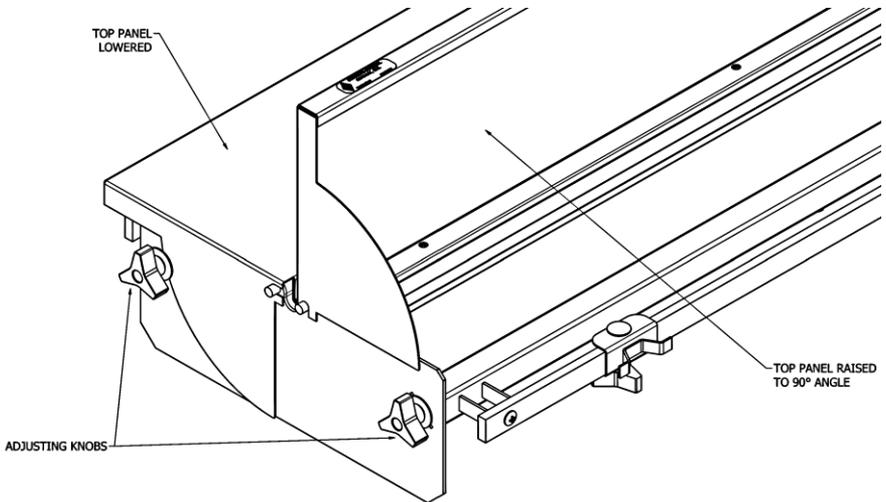

## Adjusting the Tie-Downs

The tabletop includes two adjustable tie-downs, located on each side of the table. To reposition the tie-downs:


1. Loosen the adjustment knob on the tie-down by rotating it counterclockwise.
2. Slide the tie-down along the rail to the desired position.
3. Tighten the adjustment knob firmly by rotating it clockwise to secure the tie-down in place.

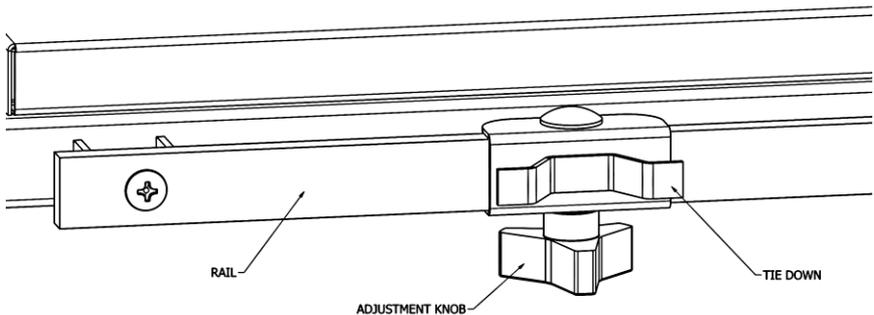

## Mounting the Tabletop to a Base


*Requires a ½ in. wrench*

1. Remove both V-panels from the table by loosening all four adjustment knobs, rotating the panels 90 degrees to a vertical position, and lifting them out of the frame.
2. Remove the collection pan from the frame.
3. Turn the tabletop frame over so that it is upside down.
4. With a ½ in. wrench, remove the four cap screws and eight washers from the frame.

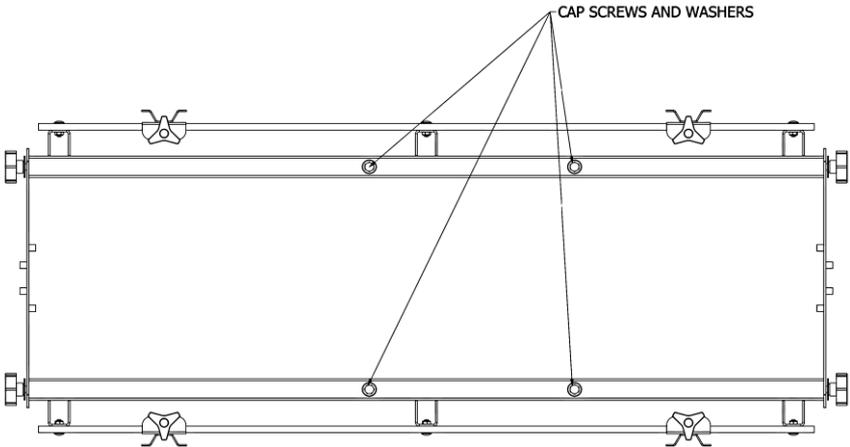


5. Place the table frame on the base and align the mounting holes. Note that the mounting holes are not centered on the table, which creates a long end and a short end.

For mounting the table to a **hydraulic base**, verify that the long end of the table is facing away from the base tilt mechanism.

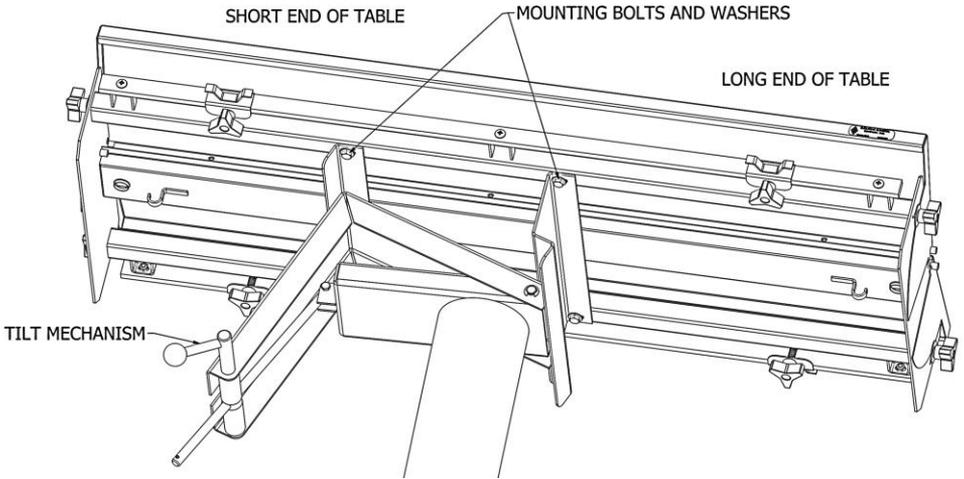

For mounting the table to an **electric base**, verify that the short end of the table is on the same side as the extended end of the tilt mechanism.

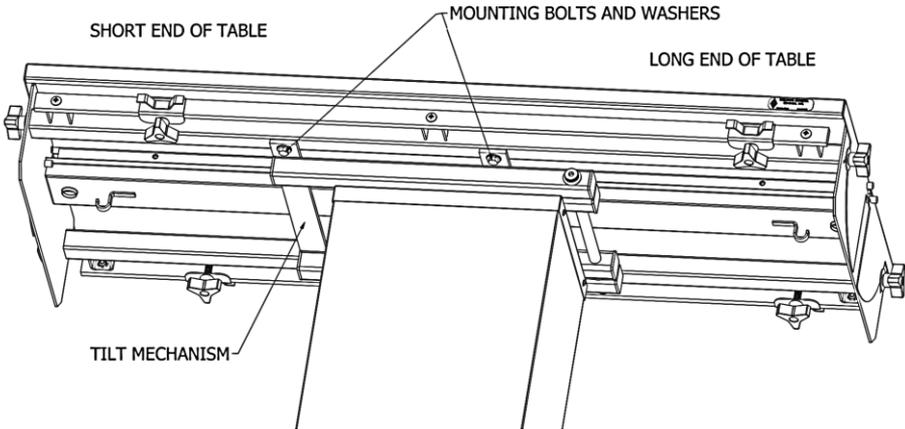

6. Install two washers on each of the four cap screws.
7. From underneath, secure the table frame to the base using the four screws and washers.
8. Place the drain trough back onto the frame by resting it over the inner support studs.
9. Reinstall the two V-panels onto the frame by loosening the lock handles and aligning the panel notches with the mounting studs. Verify that the large washer on each locking knob is positioned in front of the V-panel leaf.

