


Congratulations on your purchase of the Flat Operating Table Top!

Flat Operating Table Owner's Manual

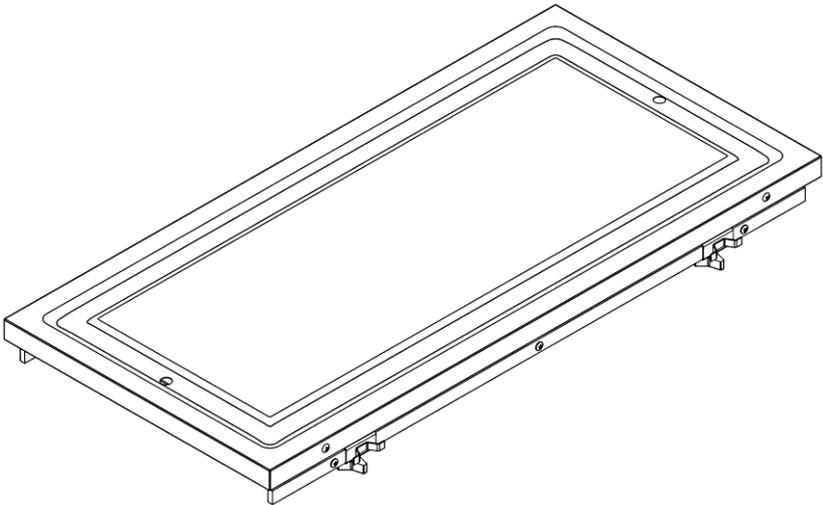

SSCI Contact Information

Suburban Surgical Co., Inc. 275 Twelfth Street Wheeling, Illinois 60090

Telephone: Illinois-(847)537-9320 Toll Free-(800)323-7366


Web: www.suburbansurgical.com

702706

Adjusting the Tie-Downs

The tabletop includes two adjustable tie-downs, located on each side of the table. To reposition the tie-downs:


1. Loosen the adjustment knob on the tie-down by rotating it counterclockwise.
2. Slide the tie-down along the rail to the desired position.
3. Tighten the adjustment knob firmly by rotating it clockwise to secure the tie-down in place.

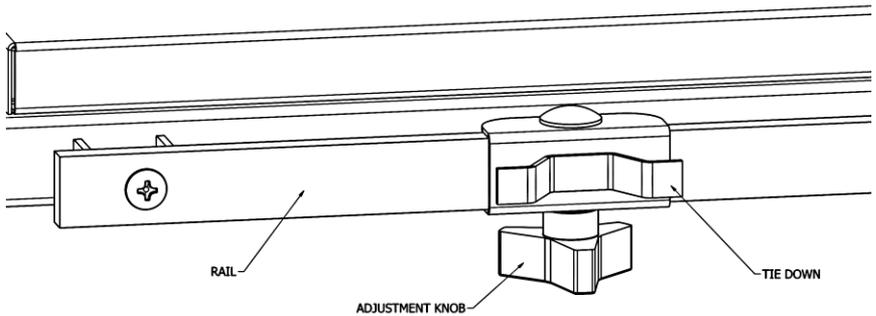

Mounting the Tabletop to a Base


Requires a ½ in. wrench

1. Turn the tabletop frame over so that it is upside down.
2. With a ½ in. wrench, remove the four cap screws and eight washers from the frame.

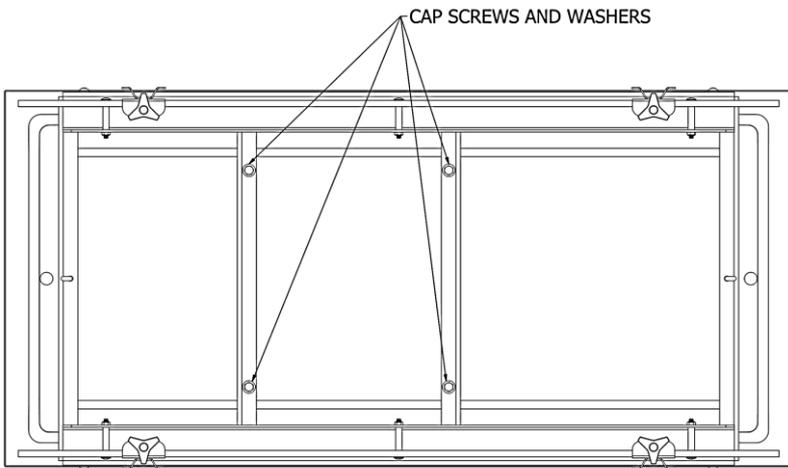


3. Place the table frame on the base and align the mounting holes. Note that the mounting holes are not centered on the table, which creates a long end and a short end.

For mounting the table to a **hydraulic base**, verify that the long end of the table is facing away from the base tilt mechanism.

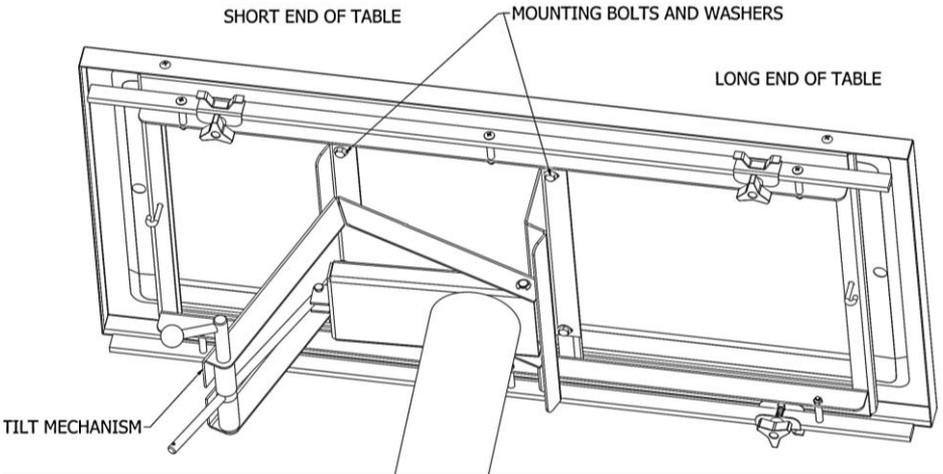

For mounting the table to an **electric base**, verify that the short end of the table is on the same side as the extended end of the tilt mechanism.

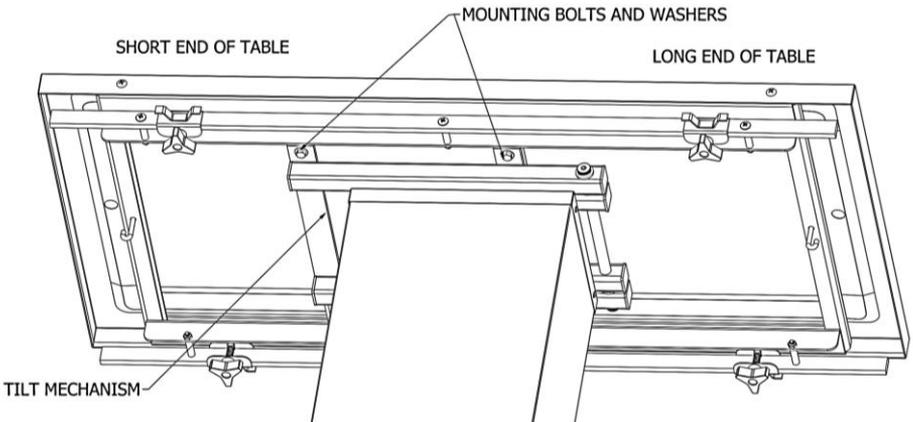

4. Install two washers on each of the four cap screws.
5. From underneath, secure the table frame to the base using the four screws and washers.