


Congratulations on your purchase of the Classic Folding Wall-Mounted Exam Table!

Classic Folding Wall-Mounted Exam Table Owner's Manual

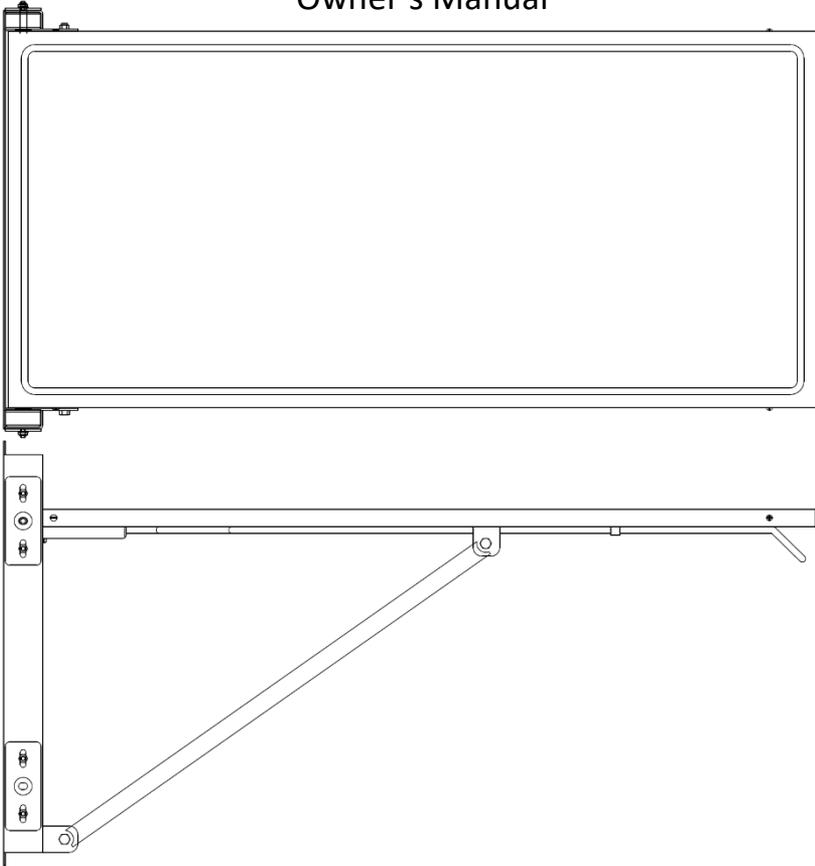

SSCI Contact Information

Suburban Surgical Co., Inc. 275 Twelfth Street Wheeling, Illinois 60090
Telephone: Illinois-(847)537-9320 Toll Free-(800)323-7366
Web: www.suburbansurgical.com

702710

Pre-Assembly

For new construction: we recommend that, prior to finishing the wall to which this table will be mounted, you determine exactly where on the wall it will be located. Back up the mounting screw locations with 2x6s.

For an existing wall: you may want to open the wall and install these supports. These steps are not required for masonry walls.

Table-to-wall mounting hardware is not included. Use lag screws for mounting into wood or toggle bolts for mounting into block.


Assembly

Tools Required

Pen / pencil, Tape measure, Carpenter's Level, Power drill, Mounting hardware suitable for wall with wrenches or screwdrivers

Preparing the Wall

1. Mark the ten mounting holes as shown using pen/pencil, tape measure, and level. To achieve a table height of 42 in., mark the bottom holes 20 in. from floor. Mark holes higher or lower for desired table height.
2. Drill ten holes with suitable drill bit for appropriate mounting hardware.

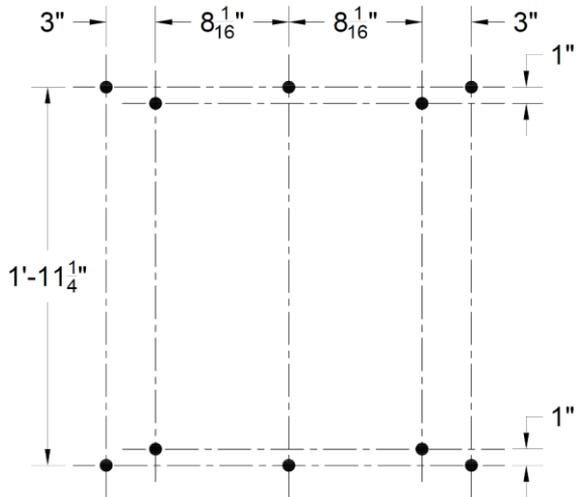


Mounting the Table to the Wall

1. Place sheets or other soft materials on the floor to protect the table top. Remove the table from the shipping carton and lay it on the floor with the table top surface down.

2. Pull outward on the locking handle and unfold the table.



3. Using hardware suitable to the type of wall, mount the table to the wall with screws or bolts through the ten holes.

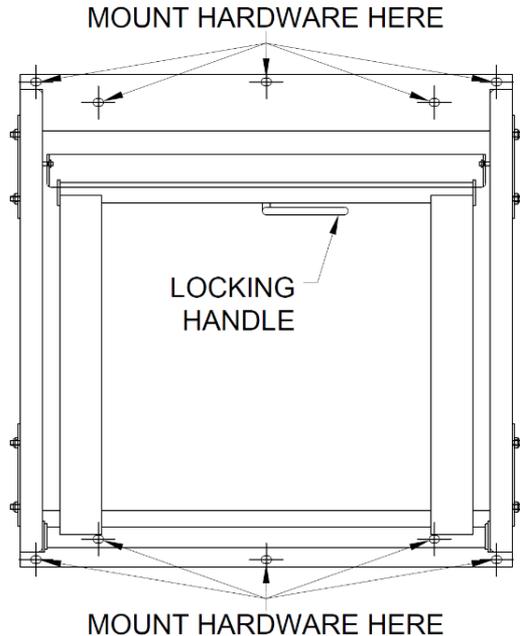

Folding and Unfolding the Table

CAUTION: This table is designed to carry weights up to 200-lbs.

Folding the Table

To fold the table and place in the stored position, pull out on the locking handle and gently lift the table up against the wall until it locks.

Unfolding the Table

To unfold the table and place it in the lowered position, pull up on the locking handle and gently lower the table.