


Congratulations on your purchase of the Double–Station Surgeon Scrub Sink!

Double–Station Surgeon Scrub Sink Owner's Manual

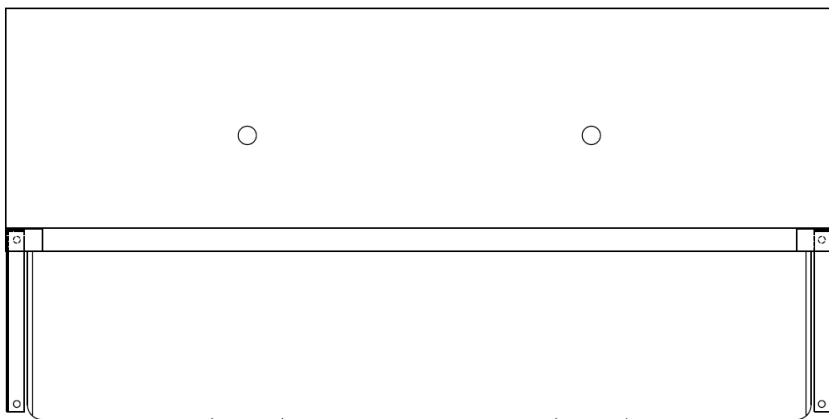

SSCI Contact Information

Suburban Surgical Co., Inc. 275 Twelfth Street Wheeling, Illinois 60090


Telephone: Illinois-(847)537-9320 Toll Free-(800)323-7366

Web: www.suburbansurgical.com


Parts Included

Note: Sink-to-wall mounting hardware is not included.


Scrub sink (1)

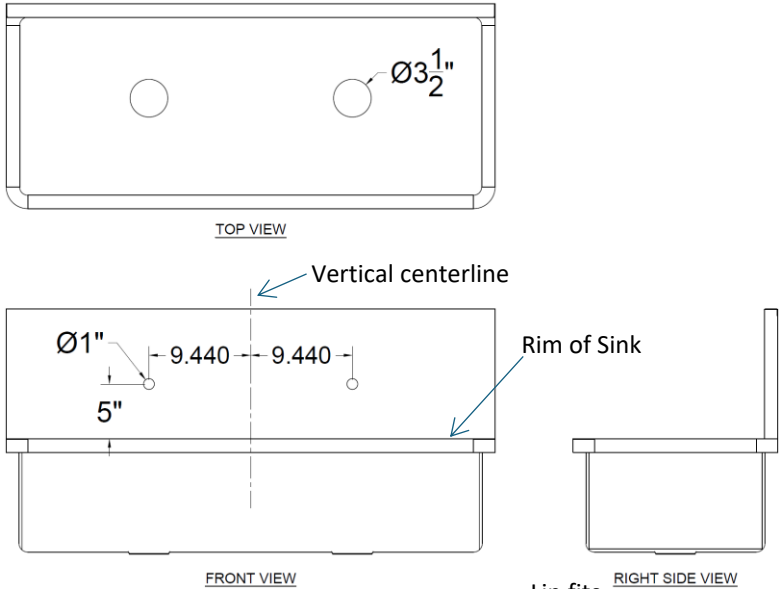



Top bracket (1)

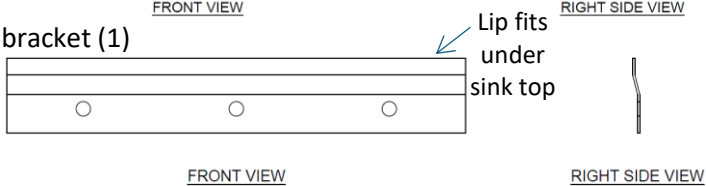

Left side bracket (1)

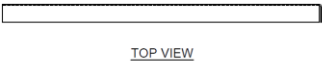

Right side bracket (1)

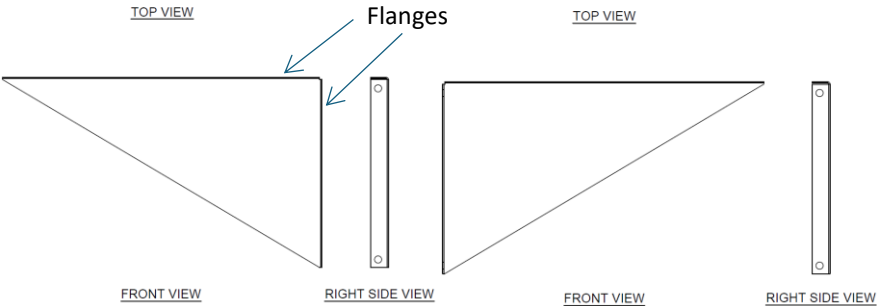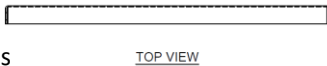


Installation


Plumbing should be complete and wall should be finished before installation.

Tools needed: Pen or pencil, measuring tape, carpenter's level, power drill, mounting hardware and tools


1. Hold the sink in position level. Measure 12 in. up from sink rim and mark a level horizontal line on the wall.
2. Mark the vertical center of the sink across this horizontal line.
3. Align the top edge of the top bracket along the horizontal line and center it on the vertical line. Mark the centers of the three vertical holes as shown.

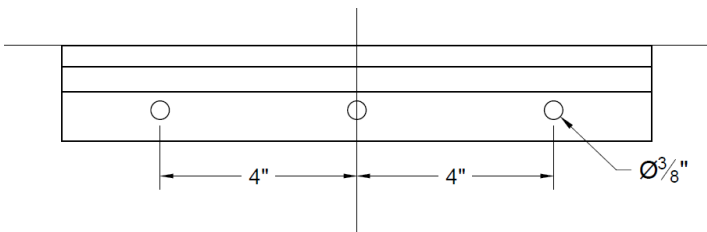

4. Drill the holes and mount the top bracket.
5. Hang the sink on the bracket centered and level.
6. Hold one side bracket against the wall with flanges facing inward. Make firm contact with the top flange under the sink rim. Mark its location on the wall. Repeat for the other side.
7. Remove the sink and hold each side bracket to its marked location. Mark two mounting holes for each bracket as shown on Page 3.

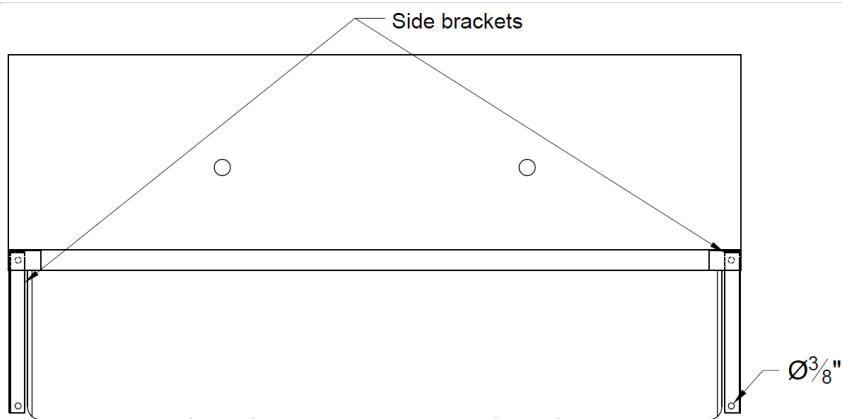

FRONT VIEW

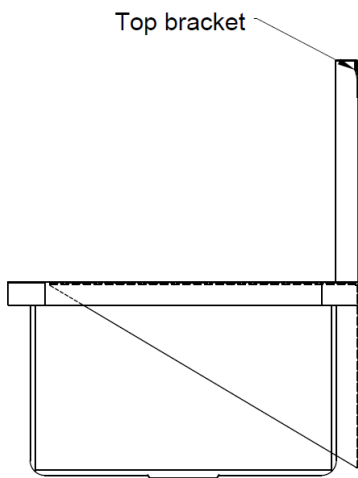

RIGHT SIDE VIEW

- 8. Drill the holes and mount the side brackets.
- 9. Hang the sink on the brackets.

Install plumbing fixtures according to their own instructions.