


Congratulations on your purchase of the Over Instrument Table!

Over Instrument Table Owner's Manual

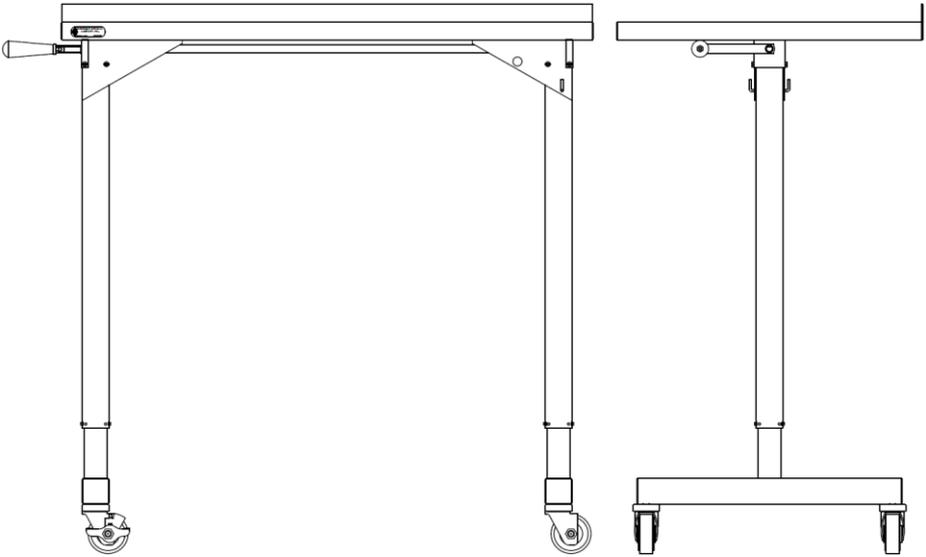


SSCI Contact Information

Suburban Surgical Co., Inc. 275 Twelfth Street Wheeling, Illinois 60090
Telephone: Illinois-(847)537-9320 Toll Free-(800)323-7366
Web: www.suburbansurgical.com


702713

Parts Included


Table Top Assembly

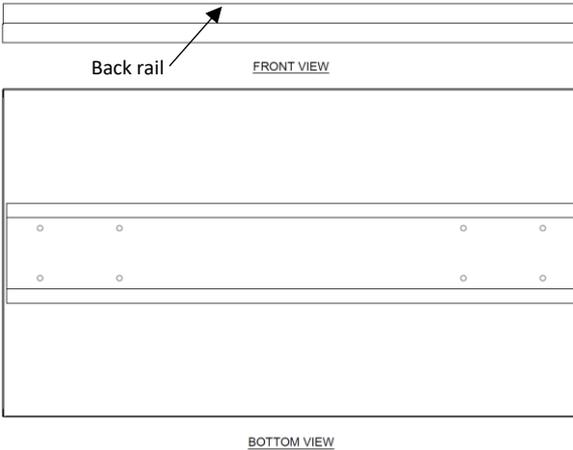


Raise/Lower Handle

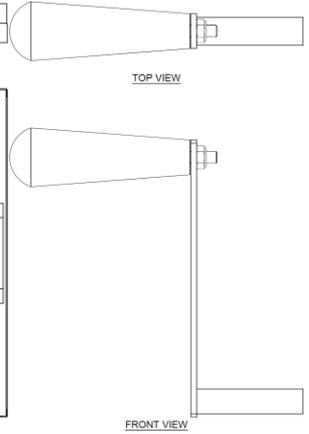


Left and Right Leg Assemblies

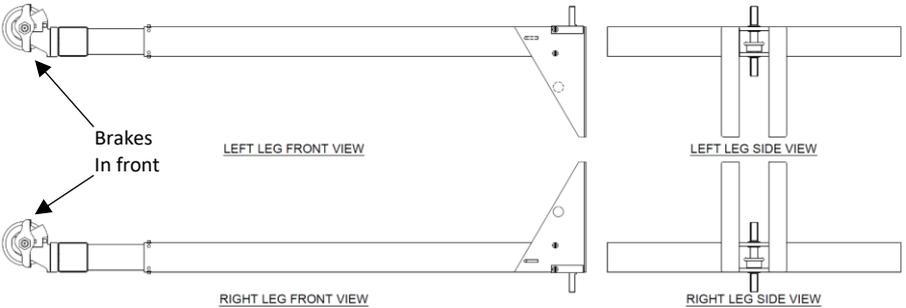

Eight (8) of each:
Cap Screw
Flat Washer
Lock Washer

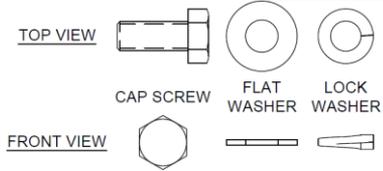



Connecting rod

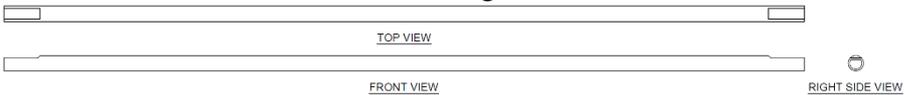


Assembly

Tool required: ½-in. wrench

1. Put the raise/lower handle on each leg and lower the leg to its lowest position. Align the flat on the handle with the flat on the gear shaft, then push the handle in. Both legs must be *in the same position* to prevent a misalignment during assembly. Set aside the handle when both legs have been lowered.
2. Using a carpet or other soft surface to protect the table top, place the table top assembly upside-down on the floor.
3. Fasten the **RIGHT** leg assembly to the right side of the table top assembly as shown on Page 3 with four (4) cap screws, lock-washers, and flat washers. The lock-washers go between the cap screws and flat washers. The handle hook and caster with a brake should be toward the *front* of the table.

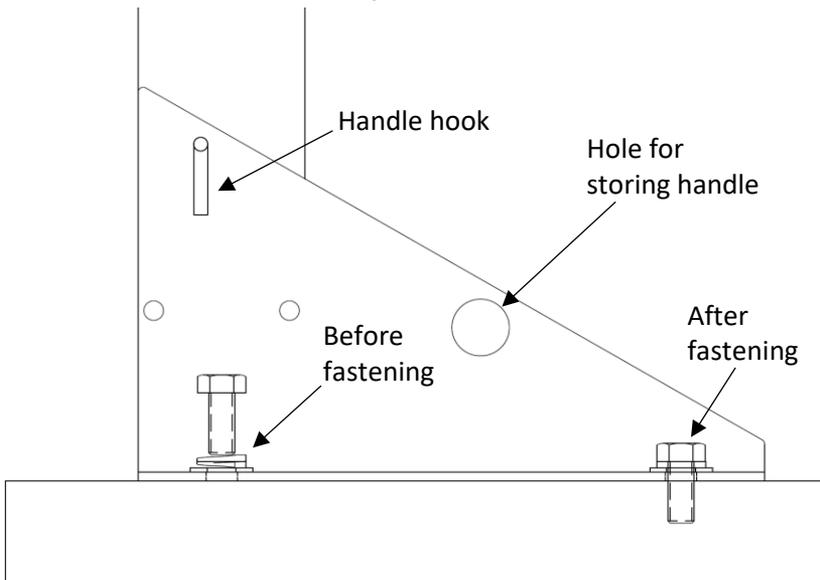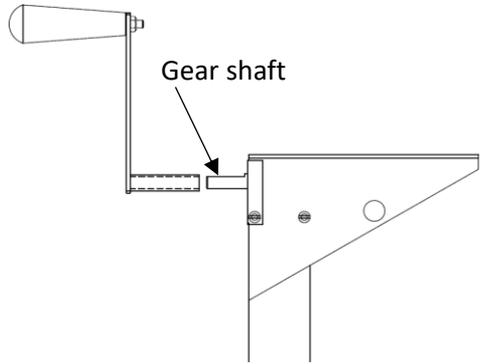


4. Push the connecting rod onto the gear shaft of the installed leg assembly with the flat on the rod aligned with the flat on the gear shaft. Push the rod all the way onto the shaft.

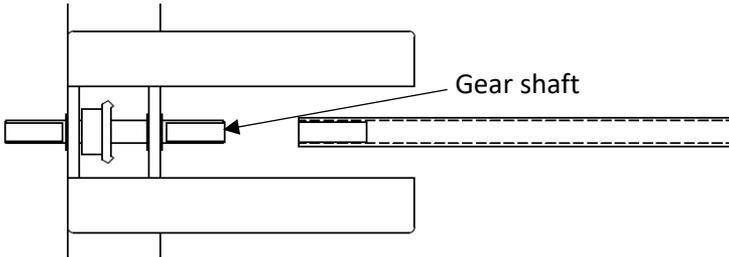

5. Connect the other end of the connecting rod to the gear shaft on the **LEFT** (not installed) leg assembly as with the right leg.
6. Fasten this leg to the bottom of the table with four (4) cap screws, lock washers, and flat washers as with the right leg in Step 3. The handle hook on the **LEFT** leg will be toward the *rear* of the table and the caster with a brake will be toward the front of the table.
7. Turn the table right-side-up.
8. Insert the raise/lower handle onto the gear shaft on either side of the table.

Storing the Raise/Lower Handle

The raise/lower handle can be stored away when not in use. Place the handle shaft into the hole in the leg mounting bracket at either end of the table and rest the handle in the handle hook.

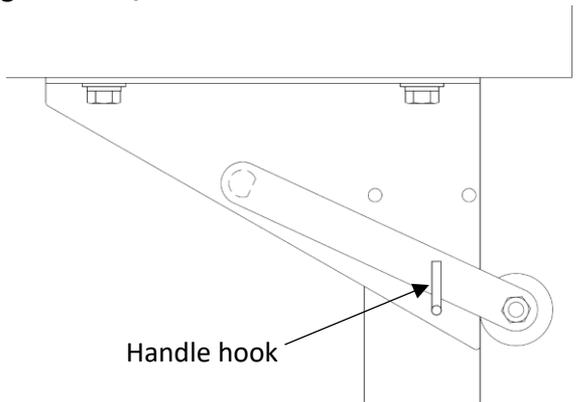

Lubrication

If gears ever require additional lubrication, unscrew legs from table top and add Lubriplate grease, rotate several times, and re-assemble.