


Congratulations on your purchase of the Pedestal Table Base!

Pedestal Table Base Owner's Manual

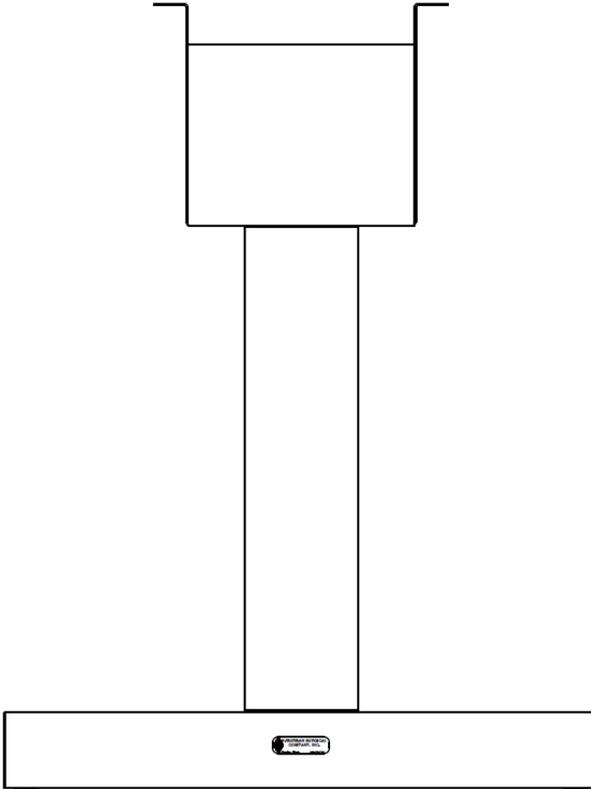

SSCI Contact Information


Suburban Surgical Co., Inc. 275 Twelfth Street Wheeling, Illinois 60090
Telephone: Illinois-(847)537-9320 Toll Free-(800)323-7366
Web: www.suburbansurgical.com

702719

Pre-Assembly

Parts Included

- Pedestal Table Base
- Four (4) leveler caps, shown right

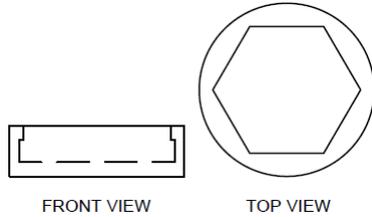

Tools Required



7/8-inch wrench, 9/16-inch wrench, 1/2-inch wrench, carpenter's level

Assembly


First, level the base, then mount the Table Top. Level the table base **ONLY** at the location and orientation where it will be used. If the table base is moved to another location or turned to a different orientation, the leveling procedure may be invalid and should be redone.

Leveling the Base

1. Place a carpenter's level length-wise on the top frame and check the level of the base left-to-right.
2. Place the level cross-wise on the top frame and check the level of the base front-to-rear.

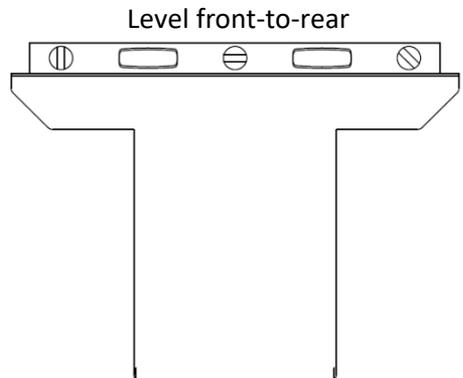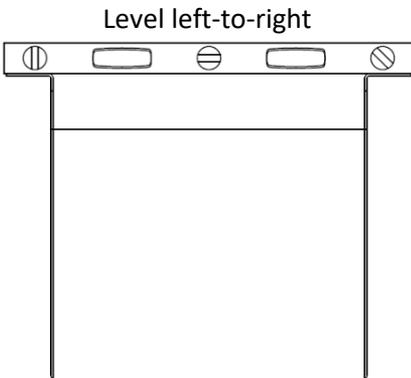



- If the base is level or very close, press the four (4) leveler caps onto the leveler legs and proceed to **Mounting the Table Top** on Page 3.
 - If the base is substantially out of level, proceed to Step 3.
3. Lay the base down carefully to reach the leveler legs.

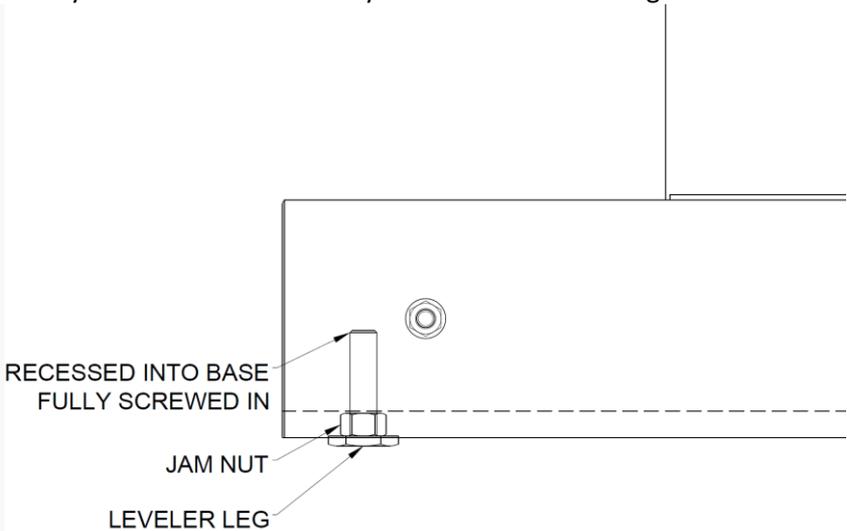

4. Loosen jam nuts on all four (4) leveler legs with 9/16-inch wrench.
5. Unscrew all four (4) leveler legs out about ½ inch.
6. Turn the base right-side-up.
7. Place a level on the top frame in the left-to-right position.
8. With a 7/8-inch wrench, turn the leveler legs as necessary to level the base, left-to-right.
9. Place a level on the top frame in the front-to-rear position.
10. Turn the leveler legs as necessary to level the base, front-to-rear.
11. Re-check the level both ways and re-adjust the leveler legs until the base is completely level left-to-right and front-to-rear.
12. When the base is level (and all legs firmly touch the floor), hold each leveler leg steady with the 7/8-inch wrench, and then tighten the jam nut firmly with the 9/16-inch wrench up against the bottom of the base.
13. When leveling is complete, press the four (4) leveler caps onto the leveler legs.

Mounting the Table Top

1. Place sheets or other soft materials on a smooth, clean floor to avoid scratching the table top.
2. Remove the table top from the shipping carton, then place the top upside-down on the soft surface.
3. With a ½-inch wrench, remove the nuts and flat washers from the four (4) threaded studs under the table top.

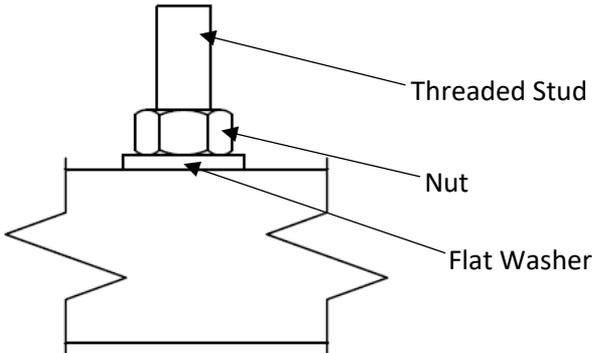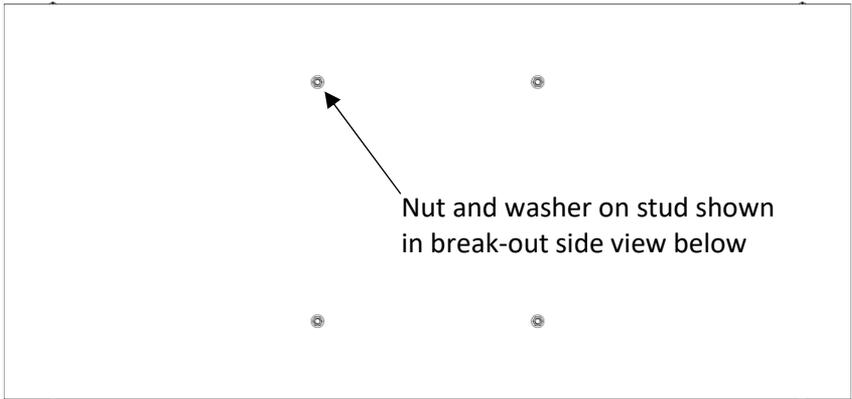

4. Carefully place the table top on the base's top frame so that all threaded studs under the table enter the mounting holes in the top frame.
5. Secure the table top to the top frame with the four (4) nuts and washers removed in Step 3.