


Congratulations on your purchase of the THERM-O-MATIC II Heat Controller!

THERM-O-MATIC II Heat Controller Owner's Manual

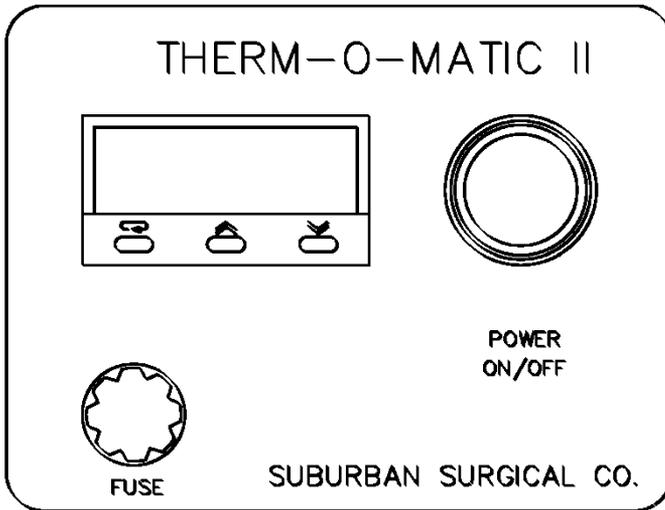

SSCI Contact Information

Suburban Surgical Co., Inc. 275 Twelfth Street Wheeling, Illinois 60090

Telephone: Illinois-(847)537-9320 Toll Free-(800)323-7366

Web: www.suburbansurgical.com

Power Button (On/Off)

- Press the **On/Off** button to turn the heat controller **ON** or **OFF**.
- When the controller is **ON**, the button will light up.

Setting the Temperature

1. Press the **left cycle** button once — the display will show **“SP1.”**
2. Press the **middle UP arrow** once.
3. Use the **UP and DOWN arrows** to set your desired temperature.
4. Once set, press the **left cycle** button again to confirm and lock the temperature.

Temperature Range:

- **Minimum:** 75°F
- **Maximum:** 105°F

LCD Display

- When the controller is **ON**, the screen shows the **current temperature** in degrees Fahrenheit (°F).

Fuse Information

- **All standard heated products** use a 2 Amp fuse (3AG, Fast-Acting, 250V 2A)
- **Double door dog cages** use a 3.5 Amp fuse (AGC, Fast-Acting, 250V 3.5A)

These are common fuses and can be purchased from SSCI or any store that sells electrical components.

How to Replace the Fuse

Removing the Fuse

1. **Unplug** the power cord.
2. Press the **fuse cap in** and turn it $\frac{1}{4}$ **turn counterclockwise** until it releases.
3. Pull out the **fuse cap** and remove the **fuse** from the cap.

Inspecting the Fuse

- Look inside the fuse. If the **thin wire inside the glass is intact**, the fuse is good.
- If the wire is **broken**, the fuse is blown and must be replaced.

The controller will not operate without a working fuse.

Installing a New Fuse

1. Insert the **new fuse** into the **fuse cap**.
2. Line up the two **ears** on the cap with the **gaps** in the fuse holder.
3. Push the fuse into the holder and turn it $\frac{1}{4}$ **turn clockwise** until it locks in place.
4. Plug in the **power cord**.