


Congratulations on your purchase of the Classic Bathing Table with Access Door!

Classic Bathing Table with Access Door Owner's Manual

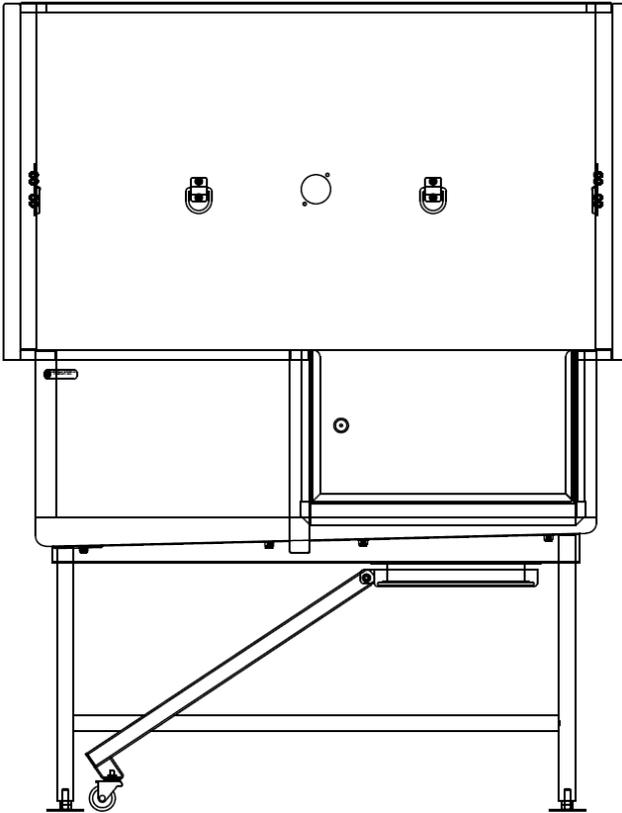


SSCI Contact Information

Suburban Surgical Co., Inc. 275 Twelfth Street Wheeling, Illinois 60090
Telephone: Illinois-(847)537-9320 Toll Free-(800)323-7366
Web: www.suburbansurgical.com

Leveling the Table:


Tools Required: 2 $\frac{3}{4}$ -in. open-end wrenches, carpenter's level

1. With a $\frac{3}{4}$ -in. wrench, rotate the levelers on the legs until table is level and all legs firmly touch the floor.

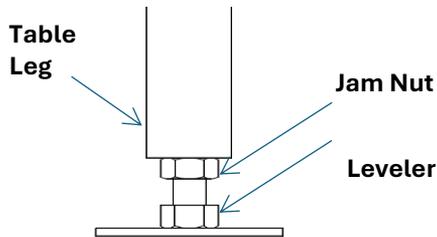

2. Holding each leveler steady with a $\frac{3}{4}$ -in. wrench, tighten each jam nut firmly up against the bottom of the table with the other $\frac{3}{4}$ -in. wrench.

Using the Rotating Walk-up Ramp:

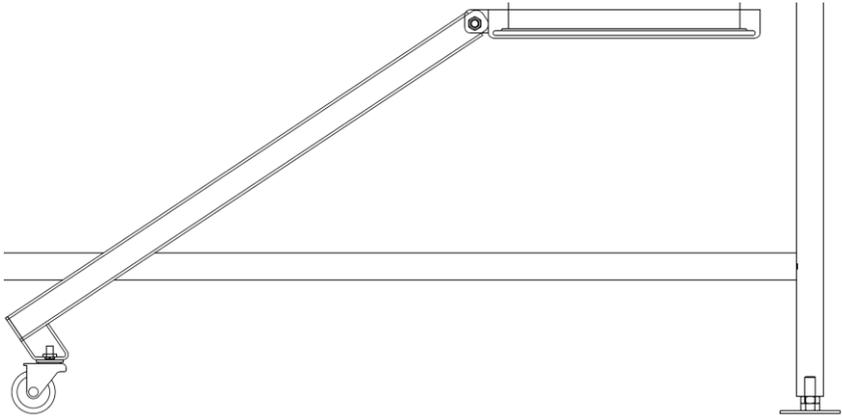

The rotating walk-up ramp parks under the table and can be rotated outward to facilitate access.

To open or park the walk-up ramp, grasp the ramp at any convenient point and pull or push the ramp into position. The lower end of the ramp is wheeled and allows the ramp to turn easily in either direction.