


Congratulations on your purchase of the Classic Vet-Mate Lift Gurney!

Classic Vet-Mate Lift Gurney Owner's Manual

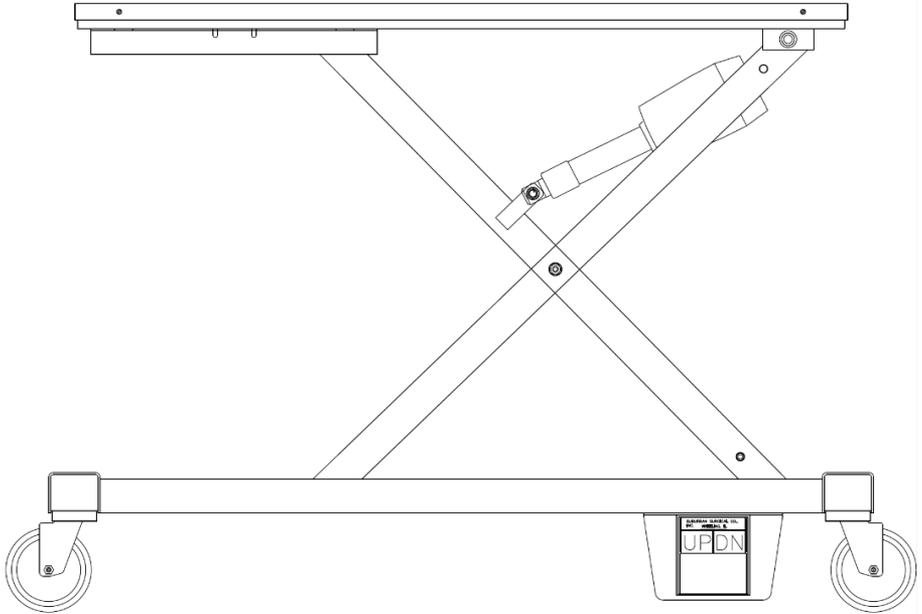

SSCI Contact Information

Suburban Surgical Co., Inc. 275 Twelfth Street Wheeling, Illinois 60090

Telephone: Illinois-(847)537-9320 Toll Free-(800)323-7366

Web: www.suburbansurgical.com

702777



The lift gurney is designed to carry up to 250 pounds.

Gurney must be unplugged to be able to raise and lower.

When charging, the battery's charging LED blinks yellow until fully charged. Once charged, the LED is solid yellow and the gurney may be unplugged.


A foot switch will allow you to raise and lower the gurney. Hold down either end of the switch until the gurney reaches the desired height, then release the switch.

The foot switch can be mounted on the end or the side of the frame.

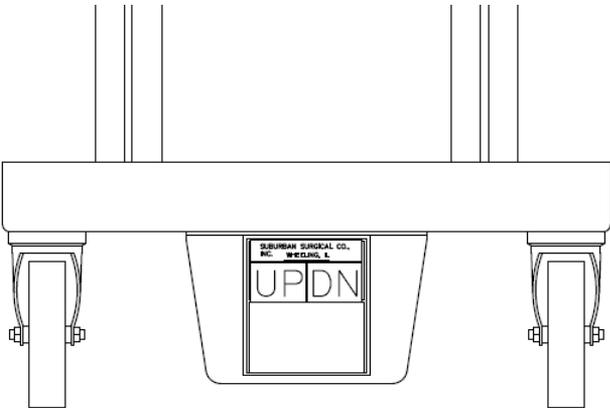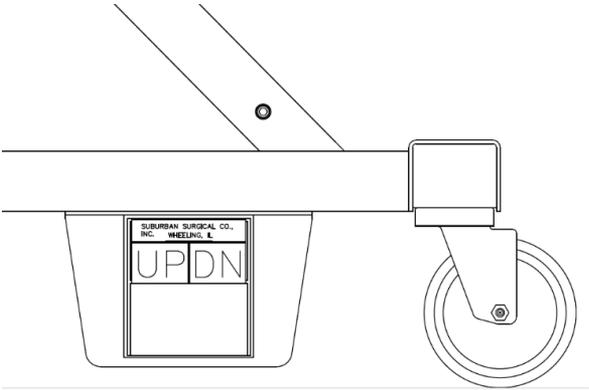

Safety

Keep fingers and other body parts clear of marked pivot point while gurney lift is in use.

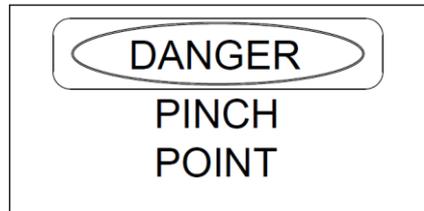

Crush Warning

The gurney is raised and lowered by means of a powerful electric actuator. Use caution when raising or lowering the gurney to ensure that feet or other body parts are not trapped under the gurney. Make sure that objects are not caught under the gurney as they can be damaged or cause damage to the gurney lifting mechanism.