




## Classic Bathing Table with Access Door

- *Convenient Access Door & Rotating Ramp*
- *Select Left, Back, or Right Faucet Location*
- *Pick 48-in or 60-in Table Width*
- *Choice of Integral Splash Shields*

### Model Numbers:

13022-LA-HJJRDR	13022-LA-JFKDDR	13022-LE-HJJRDR	13022-LE-JFKDDR
13022-LB-HJJRDW	13022-LB-JFJRDW	13022-LF-HJJRDW	13022-LF-JFJRDW
13022-LC-HJJRDW	13022-LC-JFJRDW	13022-LG-HJJRDW	13022-LG-JFJRDW
13023-RA-HJJRDR	13023-RA-JFKDDR	13023-RE-HJJRDR	13023-RE-JFKDDR
13023-RB-HJJRDW	13023-RB-JFJRDW	13023-RF-HJJRDW	13023-RF-JFJRDW
13023-RC-HJJRDW	13023-RC-JFJRDW	13023-RG-HJJRDW	13023-RG-JFJRDW
13022-BA-HJJRDW	13022-BA-JFJRDW	13022-BE-HJJRDW	13022-BE-JFJRDW
13022-BB-HJJRDW	13022-BB-JFJRDW	13022-BF-HJJRDW	13022-BF-JFJRDW
13022-BD-HJJRDW	13022-BD-JFJRDW	13022-BH-HJJRDW	13022-BH-JFJRDW
13023-BA-HJJRDW	13023-BA-JFJRDW	13023-BE-HJJRDW	13023-BE-JFJRDW
13023-BC-HJJRDW	13023-BC-JFJRDW	13023-BG-HJJRDW	13023-BG-JFJRDW
13023-BD-HJJRDW	13023-BD-JFJRDW	13023-BH-HJJRDW	13023-BH-JFJRDW

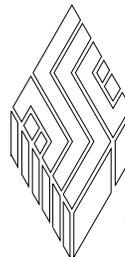

# SSCI

Wheeling, IL (800) 323 7366

# Table of Contents

<b>General Information .....</b>	<b>3</b>
Introduction .....	3
About this Manual .....	3
Information & Safety Notices .....	4
Notes .....	4
CAUTIONS .....	4
SSCI Contact Information .....	4
Models .....	5
Accessories .....	7
Parts Ordering Procedure .....	7
Care & Cleaning of Stainless Steel .....	7
Introduction .....	7
Cleaning & Cleansers .....	8
Deodorizing Agents, Disinfectants & Sanitizers .....	8
Effect on Warranty .....	9
Cleaning Requirements .....	9
Warranty .....	9
<b>Installation .....</b>	<b>10</b>
General .....	10
Installation Procedure .....	10
Unpacking & Inspection .....	10
Tools & Supplies Required .....	10
Installation Sequence .....	10
Determine the Bathing Table Location .....	10
Open the Wall .....	11
Install the Plumbing Piping for the Faucet & Drain .....	11
Finish the Wall .....	11
Install the Bathing Table .....	11
Connect the Drain .....	11
Level the Bathing Table .....	11
Install the Faucets .....	12
<b>Use &amp; Care of Your Bathing Table .....</b>	<b>13</b>
Using Your Bathing Table .....	13
Introduction .....	13
Opening & Closing the Access Door .....	13
Using the Rotating Walk-up Ramp .....	13
Caring for Your Bathing Table .....	14
Introduction .....	14
Cleaning Procedures .....	14

© Copyright 2008 by **Suburban Surgical Co., Inc.** All rights reserved.

No part of this document may be reproduced or utilized in any form or by any means, electronic or mechanical, including photocopying, recording, or by any information storage and retrieval system without written permission.

Inquiries should be addressed to **Suburban Surgical Co., Inc.**

Wheeling, Illinois 60090, USA

# General Information



## Introduction

Efficient bathing equipment is an essential component of virtually every pet care facility. Whether your business specializes in regular preventive care, emergency services, boarding, or grooming, you'll build customer satisfaction when you send your clients home from your facility bathed, groomed, and sweet smelling.

SSCI's unique new Bathing Table features a moveable ramp and a leakproof entry door to make placing the animal into the table easier, more convenient and much safer.

SSCI offers a wide variety of other options for your Bathing Table. You can choose the exact Bathing Table configuration you need to efficiently bathe the animals in your care. Select a table with the faucet on the left, back, or right. You can even specify the splash shields you need to minimize water damage from overspray. Choose either 48 in. or 60 in. widths. SSCI's Classic series Bathing Tables are designed to the most comfortable working height.

The entire table: tub, splash shields, comfort rails, and base, is made of Type 304 stainless steel, heli-arc welded for superior strength and durability. All welds are ground smooth, and the entire assembly features a uniform, non-glare finish.

## About this Manual

Every attempt has been made to insure that the information in this manual is correct and complete. SSCI, however, always welcomes our customer's suggestions for improvements to our products and associated publications.

## Information & Safety Notices

In this manual you will find text under the headings **Note:**, and **CAUTION:**.

### Notes

Under **Note:** headings, you will be given additional information pertinent to the subject discussed in that paragraph or step.

#### Example:

Stainless steel is steel alloyed with chromium to make it highly resistant to stain, rust and corrosion. **Note:** This does NOT mean that stainless steel will *never* rust or corrode.

### CAUTIONS

Under **CAUTION:** headings, you will be alerted to potentially hazardous conditions which, if ignored or mishandled, could result in injury to yourself or damage to the equipment.

#### Example:

**CAUTION: Unpacking & assembling the Bathing Table is not difficult. The table is heavy, however, & handling it can be awkward for one person. We recommend that unpacking & installation be done by at least two people.**

## SSCI Contact Information

Contact SSCI Customer Service by mail, telephone or fax. The department is available from 8:30am to 5:00pm, Central Time, Monday through Friday. Closed holidays.

**Address:** Suburban Surgical Co., Inc.  
275 Twelfth Street  
Wheeling, Illinois 60090

**Telephone:** Illinois - (847) 537-9320, ext. 3518  
Toll Free - (800) 323-7366

**Fax:** (847) 537-9061

**Web:** [www.suburbansurgical.com](http://www.suburbansurgical.com)

**Models**

The table below gives the model numbers and important identifying features for the various Classic Bathing Tables with Access Doors.

Table Part Number	Table Length	*Door & Ramp Location	*Drain Location	**Splash Shields	Faucet Location	Faucet Types
13022-LA-HJJRDR	48"	Right	Left	L	Left	Two Holes on 8" Centers
13022-LA-JFKDDR	60"	Right	Left	L	Left	Two Holes on 8" Centers
13022-LE-HJJRDR	48"	Right	Left	L	Left	Single Lever Faucet
13022-LE-FJKDDR	60"	Right	Left	L	Left	Single Lever Faucet
13022-LB-HJJRDW	48"	Right	Left	L, B	Left	Two Holes on 8" Centers
13022-LB-JFJRDW	60"	Right	Left	L, B	Left	Two Holes on 8" Centers
13022-LF-HJJRDW	48"	Right	Left	L, B	Left	Single Lever Faucet
13022-LF-JFJRDW	60"	Right	Left	L, B	Left	Single Lever Faucet
13022-LC-HJJRDW	48"	Right	Left	L, B, R	Left	Two Holes on 8" Centers
13022-LC-JFJRDW	60"	Right	Left	L, B, R	Left	Two Holes on 8" Centers
13022-LG-HJJRDW	48"	Right	Left	L, B, R	Left	Single Lever Faucet
13022-LG-JFJRDW	60"	Right	Left	L, B, R	Left	Single Lever Faucet
13023-RA-HJJRDR	48"	Left	Right	R	Right	Two Holes on 8" Centers
13023-RA-JFKDDR	60"	Left	Right	R	Right	Two Holes on 8" Centers
13023-RE-HJJRDR	48"	Left	Right	R	Right	Single Lever Faucet
13023-RE-JFKDDR	60"	Left	Right	R	Right	Single Lever Faucet
13023-RB-HJJRDW	48"	Left	Right	B, R	Right	Two Holes on 8" Centers
13023-RB-JFJRDW	60"	Left	Right	B, R	Right	Two Holes on 8" Centers
13023-RF-HJJRDW	48"	Left	Right	B, R	Right	Single Lever Faucet
13023-RF-JFJRDW	60"	Left	Right	B, R	Right	Single Lever Faucet
13023-RC-HJKDDR	48"	Left	Right	L, B, R	Right	Two Holes on 8" Centers
13023-RC-JFKDDR	60"	Left	Right	L, B, R	Right	Two Holes on 8" Centers
13023-RG-HJKDDR	48"	Left	Right	L, B, R	Right	Single Lever Faucet
13023-RG-JFKDDR	60"	Left	Right	L, B, R	Right	Single Lever Faucet

\* The drain is always on the opposite side from the door and ramp.

\*\* Under Splash Shields: L = Left, B = Back, R = Right

**Part Numbers - Classic Bathing Tables with Access Doors (cont'd on Page 6)**

Table Part Number	Table Length	*Door & Ramp Location	*Drain Location	**Splash Shields	Faucet Location	Faucet Types
13022-BA-HJJRDW	48"	Right	Left	B	Back	Two Holes on 8" Centers
13022-BA-JFJRDW	60"	Right	Left	B	Back	Two Holes on 8" Centers
13022-BE-HJJRDW	48"	Right	Left	B	Back	Single Lever Faucet
13022-BE-JFJRDW	60"	Right	Left	B	Back	Single Lever Faucet
13022-BB-HJJRDW	48"	Right	Left	L, B	Back	Two Holes on 8" Centers
13022-BB-JFJRDW	60"	Right	Left	L, B	Back	Two Holes on 8" Centers
13022-BF-HJJRDW	48"	Right	Left	L, B	Back	Single Lever Faucet
13022-BF-JFJRDW	60"	Right	Left	L, B	Back	Single Lever Faucet
13022-BD-HJJRDW	48"	Right	Left	L, B, R	Back	Two Holes on 8" Centers
13022-BD-JFJRDW	60"	Right	Left	L, B, R	Back	Two Holes on 8" Centers
13022-BH-HJJRDW	48"	Right	Left	L, B, R	Back	Single Lever Faucet
13022-BH-JFJRDW	60"	Right	Left	L, B, R	Back	Single Lever Faucet
13023-BA-HJJRDW	48"	Left	Right	B	Back	Two Holes on 8" Centers
13023-BA-JFJRDW	60"	Left	Right	B	Back	Two Holes on 8" Centers
13023-BE-HJJRDW	48"	Left	Right	B	Back	Single Lever Faucet
13023-BE-JFJRDW	60"	Left	Right	B	Back	Single Lever Faucet
13023-BC-HJJRDW	48"	Left	Right	B, R	Back	Two Holes on 8" Centers
13023-BC-JFJRDW	60"	Left	Right	B, R	Back	Two Holes on 8" Centers
13023-BG-HJJRDW	48"	Left	Right	B, R	Back	Single Lever Faucet
13023-BG-JFJRDW	60"	Left	Right	B, R	Back	Single Lever Faucets
13023-BD-HJJRDW	48"	Left	Right	L, B, R	Back	Two Holes on 8" Centers
13023-BD-JFJRDW	60"	Left	Right	L, B, R	Back	Two Holes on 8" Centers
13023-BH-HJJRDW	48"	Left	Right	L, B, R	Back	Single Lever Faucet
13023-BH-JFJRDW	60"	Left	Right	L, B, R	Back	Single Lever Faucet

\* The drain is always on the opposite side from the door and ramp.

\*\* Under Splash Shields: L = Left, B = Back, R = Right

**Part Numbers - Classic Bathing Tables with Access Doors (cont'd from Page5)**

## Accessories

SSCI provides a variety of accessories to the Bathing Tables to fit them to your requirements. Find descriptions in our catalog, and on our website at [www.suburbansurgical.com](http://www.suburbansurgical.com). To order accessories, refer to *Parts Ordering Procedure* below.

- Stainless steel tub rack, 48 in. - P/N 13100-00-GLAEDG
- Stainless steel tub rack, 60 in. - P/N 13100-00-IHAEDG
- PVC coated tub rack, 48 in. - P/N 13101-00-GLAEDG
- PVC coated tub rack, 60 in. - P/N 13101-00-IHAEDG
- Coated Raised Floor - P/N 13102-00 DGAEDG

## Parts Ordering Procedure

Order new equipment or parts from SSCI by mail, telephone, or fax. Refer to *Contact Information* on *Page 4* for address, telephone and fax numbers. Please provide the following:

- Your name
- Company name
- Company account number
- Telephone number
- Fax number
- e-mail address
- Shipping address
- Billing address (if different from shipping address)
- Names, part numbers, and quantities of items being ordered
- Credit card number and expiration date, or other payment information
- Preferred method of shipment
- Information on whether the items are required on a normal or urgent basis

## Care & Cleaning of Stainless Steel

### Introduction

Stainless steel is steel alloyed with chromium to make it highly resistant to stain, rust and corrosion. **Note:** This does NOT mean that stainless steel will *never* rust or corrode. Science has not yet developed a steel which is completely stainless or corrosion PROOF.

The type of stainless steel and finish selected by SSCI for the Bathing Table is the best available for the intended use.

## Cleaning & Cleansers

The basic rule of thumb is to use the mildest cleaning agent that will do the job effectively. After cleaning, always rinse thoroughly with clear water, and dry completely. Frequent cleaning will prolong the service life of stainless steel equipment and will help maintain a bright, pleasing appearance.

Ordinary deposits of waste and fluids can usually be removed with soap and water. More stubborn deposits or tightly adhering debris may require harder scrubbing and possibly the use of commercial cleaning products acceptable for use on metal surfaces. When using any cleaning agent, rub in the direction of the polish lines or "grain" of the metal. For high luster finishes, clean soft cloths or pads should be used.

If especially rough cleaning is necessary, use "stainless steel" wool, nylon or plastic scrubbers. Test these scrubbers in an inconspicuous area first to be sure they do not mar or scratch the stainless steel finish.

Minor scale build-up and some hard water spotting may be removed by washing with vinegar, followed by a neutralizing rinse with clear water and a thorough drying with a soft cloth. For heavy deposits of scale, 5% oxalic acid (use warm), 5-15% sulfamic acid, or 5-10% phosphoric acid may be used. Always follow with a neutralizing rinse of clean water and a thorough drying.

## Deodorizing Agents, Disinfectants, & Sanitizers

The large selection of brands and combinations of chemicals available for deodorizing, disinfecting and sanitizing is staggering. Select one or more agents for use in your facility only after weighing all the benefits claimed by each product. Often this choice is made without adequate consideration of the effects these agents may produce on equipment or furnishings.

**CAUTION: Before selecting a chemical to employ in your facility, review label statements regarding use with metals (stainless steel). Always consult the chemical supplier if there are any doubts.**

Avoid prolonged use of chlorides (such as chlorine bleach), bromides, iodides and thiocyanates on stainless steel surfaces as these chemicals will cause pitting, corrosion and metal discoloration. Allowing salty solutions to evaporate and dry on stainless steel may also contribute to corrosive conditions.

In summary, select chemical deodorizers, disinfectants and/or sanitizers only after weighing all possible benefits and known adverse effects.

## Effect on Warranty

The warranty for this product is void if the care and cleaning instructions provided in this manual are not followed.

## Cleaning Requirements

Clean the Bathing Table exactly in accordance with the cleaning instructions provided on *Page 14* of this manual.

*Failure to follow these instructions can void your warranty.*

## Warranty

Suburban Surgical Company, Inc. warrants the original purchaser that our products are of the highest standards in material and workmanship. Our stainless steel components are guaranteed to last a lifetime assuming they are used as intended, properly maintained and cared for. Mechanical, electrical, electronic, hydraulic, and any product's devices carry a one year warranty.

Items purchased by Suburban Surgical Company, Inc. from other manufacturers and incorporated into our equipment are covered by the respective manufacturer's warranties.

Warranties will not apply if it is determined by Suburban Surgical Company, Inc. that the equipment became defective due to an accident, misuse, abuse, improper maintenance or alteration. Warranty freight charges are covered for the first year only.

# Installation

## General

This section guides you in installing the SSCI Classic Bathing Table with Access Door. If you have problems or require additional assistance, please feel free to call SSCI Customer Service at (800) 323-7366.

## Installation Procedure

**CAUTION:** Unpacking and installing the Bathing Table is not difficult. The table is heavy, however, and handling it can be awkward for one person. We recommend that unpacking and installation be done by at least two people.

### Unpacking & Inspection

If the shipping container appears damaged in any way, contact the shipping company immediately. Save all damaged packing materials to assist in proving liability for damage.

Carefully inspect your Bathing Table while you unpack it. If any damage is noted, or if parts appear to be missing, call SSCI Customer Service at (800) 323-7366 immediately.

### Tools & Supplies Required

To install the Bathing Table, you will require only the basic tools and plumbing supplies.

### Installation Sequence

The Classic Bathing Table is easy to install. Since every installation will be a little different, however, we suggest that the following installation instructions serve as a general guide:

1. Determine the table location.
2. Open the wall (only if necessary to pipe in the faucet and drain - frequently not required on new construction).
3. Install the piping for the faucet and drain.
4. Finish the wall.
5. Install the Bathing Table.
6. Connect the drain.
7. Level the table.
8. Install the faucets.

### Determine the Bathing Table Location

Determine where you want the Bathing Table to be mounted. The location should provide a convenient working space and efficient and cost-effective routing for the faucet and drain plumbing.

## Open the Wall

If this is not new construction, open the wall as necessary to install piping for the Bathing Table faucet and drain.

## Install the Plumbing Piping for the Faucet & Drain

**Note:** Use only wall-mounted faucets. The Bathing Table is not designed for use with deck-mounted faucets.

Install the pipes and any other items required for the faucet and drain.

## Finish the Wall

After all plumbing is complete, the wall can be finished.

## Install the Bathing Table

Place the Bathing Table in place against the wall(s) with the drain hole aligned with the drain piping.


## Connect the Drain

Connect the drain piping to the table drain.

## Level the Bathing Table

Rotating the levelers on the legs (Figure 1) allows you to adjust the level of the table.

**Note:** The tub has a built-in 1-inch downward slope toward the drain for proper drainage, therefore, adjusting the leveler pads may not be necessary.

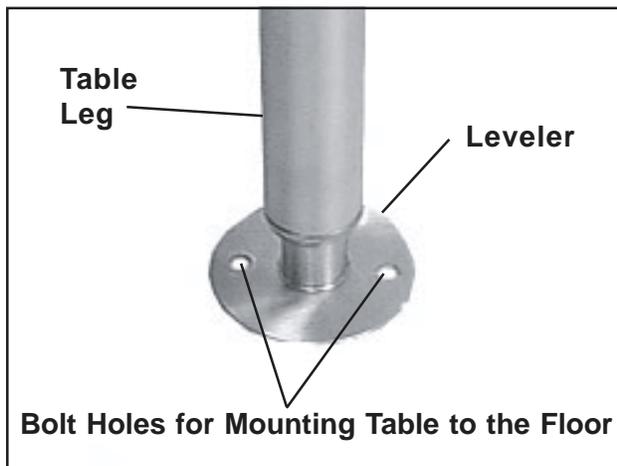

**Figure 1. Leveler on the Bottom of a Table Leg.**

1. Pour water into the tub and observe the flow. If it flows easily and completely to the drain, the table is just fine as it is. Stop here and proceed to *Install the Faucets* on *Page 12*. If the water does not flow to the drain, go to *Step 2* to raise the non-drain end of the tub slightly.
2. Raise the non-drain end of the table as necessary for the legs to clear the floor.
3. By hand, turn each leveler clockwise to lower the leg, or counterclockwise to raise.
4. Lower the table to the floor.
5. Pour more water into the tub and observe the flow. If the flow is OK, proceed to *Install the Faucets* on *Page 12*. If not, repeat *Steps 2* through *5* until the flow is adequate.

## **Install the Faucets**

1. Install the faucets according to the instructions enclosed with the units.
2. Run water into the Bathing Table and make sure the plumbing is correct and free of leaks.
3. Place the basket strainer into the drain opening.

# Use & Care of Your Bathing Table

## Using Your Bathing Table

### Introduction

Other than the faucet(s), the Bathing Table has only two moving parts: the access door and the rotating walk-up ramp.

### Opening & Closing the Access Door

The table has a stainless steel hinged door that swings inward to allow the animal easy entrance and exit. The door includes a heavy-duty, spring-loaded, self-latching positive door catch that automatically seals the door to prevent leakage.

To open the access door, simply push the circular knob away from the door opening, and push the door open (Figure 2). The knob is spring-loaded and will return to its normal position. The access door only opens inward. When closing the door, make sure it closes and locks securely to provide a sure seal against water leakage.



Figure 2. Opening & Closing the Access Door



Figure 3. Using the Walk-up Ramp

### Using the Rotating Walk-up Ramp

The rotating walk-up ramp is parked conveniently under the table but can be rotated outward to facilitate access. The ramp has a non-slip, dura-grid surface that provides traction wet or dry.

To open or park the walk-up ramp, grasp the ramp at any convenient point and pull or push the ramp to the position you desire (Figure 3). The lower end of the ramp is wheeled and allows the ramp to turn easily in either direction.

## Caring for Your Bathing Table

### Introduction

You will no doubt want to clean your Bathing Table frequently. Maintaining high standards of sanitation will be an important priority for your facility.

### Cleaning Procedures

Rinse the Bathing Table with clear water and dry thoroughly with clean, soft cloths.

Ordinary deposits of waste fluids can usually be removed with soap and water. Stubborn deposits may require scrubbing with “stainless steel” wool, nylon or plastic scrubbers and/or the use of commercial cleaning products. Always scrub in the direction of the “grain” of the metal. Rinse with clear water and dry thoroughly with clean, soft cloths.

Minor scale build-up and some hard water spotting may be removed by washing with vinegar, followed by a neutralizing rinse of clear water and a thorough drying with clean, soft cloths.

For heavy deposits of scale, 5% oxalic acid (use warm), 5-15% sulfamic acid, or 5-10% phosphoric acid may be used. As always, rinse with clear water and dry thoroughly with clean soft cloths.


Avoid prolonged use of chlorides (such as chlorine bleach), bromides, iodides and thiocyanates. Never allow salty solutions to dry on the stainless steel.

**CAUTION: The warranty for this product is void if the care and cleaning instructions provided above are not followed.**

Inside back cover.

This page left blank intentionally.

***For more information on SSCI's fine line of products  
& accessories, talk to your SSCI sales representative.***

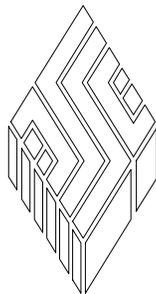

**SSCI**

**Suburban Surgical Company, Inc.**

**275 Twelfth Street**

**Phone: (847) 537-9320**

**Toll Free: (800) 323-7366**

**Wheeling, IL 60090**

**Fax: (847) 537-9061**

**Web: [www.suburbansurgical.com](http://www.suburbansurgical.com)**